INSTRUCTIONS FOR USE

Forearm folding crutch


Product code: **CompactTim**

1. Characteristics

The forearm folding crutch consists of a comfortable, durable handgrip and an aluminum shaft finished with a high-quality rubber tip. Because of stepped length adjustment and folding function, it is very convenient for daily use. The crutch has an ergonomic shape of the cuff and properly adjusted elbow angle, which ensures optimal distribution of the pressure on the handgrip.

- a. cuff
- b. handgrip
- c. folding location
- d. length adjustment
(locking pin)
- e. adjustment lock
- f. rubber tip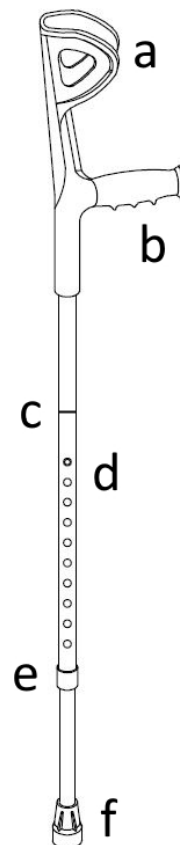

2. Application

The forearm crutch is ancillary equipment which is used to increase the area of support in case of patients with reduced mobility. It is also recommended to patients who need help and support while walking and it can also be used for gait re-education as an aid in keeping balance after the injuries of lower limbs or trunk.

Contraindications: coordination and balance disorders, advanced rheumatic changes, injuries and dysfunctions of upper limbs which may hinder the user to support on the crutch.

3. Assembly and adjustment

- To make transportation easier, the crutch has a folding function. Before adjusting the length, the crutch should be unfolded by overlapping the two parts of the shaft.
- To adjust the length of the crutch, unscrew the adjustment lock.
- Adjust the crutch, press the locking pin and select appropriate position of the inner tube. Lock the position of the tubes relative to each other with the use of locking pin. The crutch is adjustable in 2.5 cm increments.
- Once the length adjustment is complete, screw the crutch adjustment lock.
- To set the proper length of the crutch, stand upright and bent upper limb in elbow joint at the angle of 20°.
- The crutch should be adjusted individually to the user so that the handgrip is at hip joint level and the cuff is at the height of upper part of the forearm.
- Before use, ensure that the crutch is stable and the locking pin fully protrudes through adjustment hole of the crutch.
- It is recommended to use two crutches simultaneously.

NOTE! In case of doubts concerning the technique of moving with the use of the crutches, consult a person who has appropriate knowledge and qualifications.

4. Technical data

Leg: aluminium

Height adjustment: 94-116 cm

Height to the handgrip: 70-92 cm

Maximum capacity: 136 kg

Crutch weight: 0.60 kg

Colour: black

5. Notes

Timago International Group does not bear responsibility of improper use of the product, failure to comply safety regulations and misuse.

6. Cleaning and maintenance

- Clean the crutch using only water and mild detergents, wipe dry after each use.
- **Do not use:** strong detergents, abrasive agents, bleach, petrol nor solvents (it may cause frame damage, visible defects, and in consequence – loss of warranty rights).
- It is recommended to check rubber tip regularly. Replace worn tip immediately.

7. Storage and transport conditions

Device should not be exposed to direct sunlight, water or moisture.

8. Warranty information

All products distributed by our company are covered by a warranty, the terms of which are described in the warranty card available on our website. Please contact the retailer from whom you purchased the product. Please, keep in mind that for warranty purposes, the proof of purchase (receipt or invoice) should be retained

9. Labels

	Reference number		Manufacturer
	LOT number		Manufacture date
	Serial number		Medical device
	Note		Please read the instruction

The manufacturer checked for compliance with essential medical devices requirements.

Our complete line of products is available from our distributors.
Learn more at: www.timago.com. Thank you for choosing Timago!

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