

INSTRUCTIONS FOR USE

Shower chair with backrest and armrest – tool free

Product code: TGR-R KP 355L



1. Characteristics


Shower chair consists of aluminium frame, backrest and wide seat which are made of durable PVC. It possesses height adjustable legs with anti-sliding rubber tips and side foamed handgrips which facilitate using of the chair. Specially designed rubber tips prevent the chair from sliding on wet surfaces effectively. The chair is additionally equipped with Tool-Free (TF) system facilitating the assembly.

The set includes:

- seat with the base of the chair – 1 pc
- side handgrips – 2 pcs
- backrest – 1 pc
- mounting tubes of the backrest – 2 pcs
- mounting screws (wing screws) – 4 pcs

Elements of the commode chair:

- a. backrest
- b. backrest tubes
- c. seat
- d. base tubes
- e. side handgrips
- f. pin locking the handgrip
- g. height adjustment – TF system
- h. anti-sliding tips

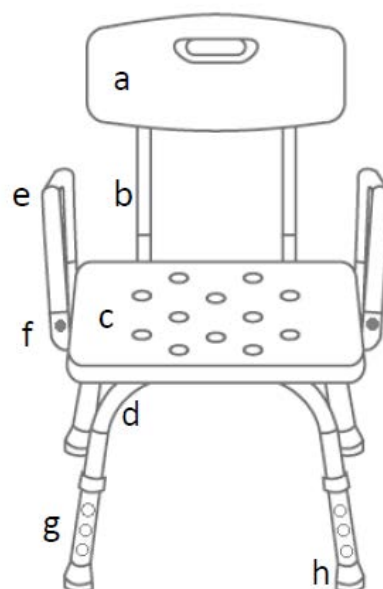

2. Application

The shower chair is ancillary equipment which helps the user to perform hygienic activities. The chair is recommended to people with

reduced mobility which prevents or hinders the unassisted performance of daily living activities. The chair can be used in the shower, in the bath, in bathrooms and other humid rooms at home and in hospitals.

3. Assembly

ASSEMBLY

Step 1. Assemble the backrest with the backrest tubes using four mounting screws (wing screws).

Step 2. Assemble the backrest tubes into the mounting openings located under the seat until the locking pins click.

Step 3. Place the assembled chair on the legs.

Step 4. Assemble the side handgrips into the mounting openings located on the outer side of the seat.

HEIGHT ADJUSTMENT

- To adjust the height of the chair, press the metal pin, select appropriate height of the leg, and then, lock the position of the legs relative to each other with the locking pin. The legs are adjustable in 2.5 cm increments.

Note! The pins must be set on the same level in each leg.

4. Technical data

Width	54 cm
Depth	48 cm
Seat height	42-55 cm
Overall height	72-85 cm
Seat dimensions	39 x 33 cm
Chair weight	3.2 kg
Maximum capacity	110 kg

5. Notes

Timago International Group does not bear responsibility of improper use of the shower chair, failure to comply safety regulations and misuse.

6. Maintenance

- Clean the shower chair regularly using only water, sponge and mild detergents for plastic materials such as soap. Dry thoroughly.
- **Do not use** for cleaning: strong detergents, abrasive agents, bleach, petrol or solvents (it may cause frame damage, visible defects and in consequence - loss of warranty rights).
- Systematically check:
 - tightening of the wing screws
 - the condition of the locking pins
 - the condition of the tubes (if cracks)

It is recommended to check the rubber tips regularly. In case of worn-out, replace the tips with the new ones.





7. Storage and transport conditions






Device should not be exposed to direct sunlight, water or moisture.

8. Warranty information

All products distributed by our company are covered by a warranty, the terms of which are described in the warranty card available on our website. Please contact the retailer from whom you purchased the product. Please, keep in mind that for warranty purposes, the proof of purchase (receipt or invoice) should be retained

9. Labels

	Reference number		Manufacturer
	LOT number		Manufacture date

	Serial number		Medical device
	Note		Please read the instruction
	The manufacturer checked for compliance with essential medical devices requirements.		

Our complete line of products is available from our distributors.
Learn more at: www.timago.com. Thank you for choosing Timago!



Timago International Group
Spółka z o.o. i Spółka – Spółka komandytowa
ul. Karpacka 24/12
43-316 Bielsko-Biała, Polska

T.: +48 33 499 50 00
F.: +48 33 499 50 11
E.: info@timago.com



Rev. II, date 20.05.2021