

INSTRUCTIONS FOR USE

Aluminium 4-wheeled rollator YOLA

Product code: TGR-R RA 883



1. Characteristics


Folding 4-wheeled rollator is made of aluminium frame which is coated with paint with metallic lustre. The rollator is equipped with brakes with locking mechanism, seat with backrest and shopping bag. It is supplied with wheels with narrow cross section 8'' in diameter and height adjustable handles with anatomic handgrips. It can also be folded into compact unit for transport. The axial pressure resistance was significantly increased owing to the crossbar frame and oval cross section of the tubes.

The set includes:

- Frame with seat and backrest
- Two push handles with brakes
- Shopping bag
- Bell
- Locking knobs – 4 pcs
- Front swivel wheels 8''
- Rear wheels 8''

Elements of the rollator:

- a. push handle
- b. brake
- c. bell
- d. backrest
- e. shopping bag
- f. brake cable
- g. locking pin
- h. locking knobs

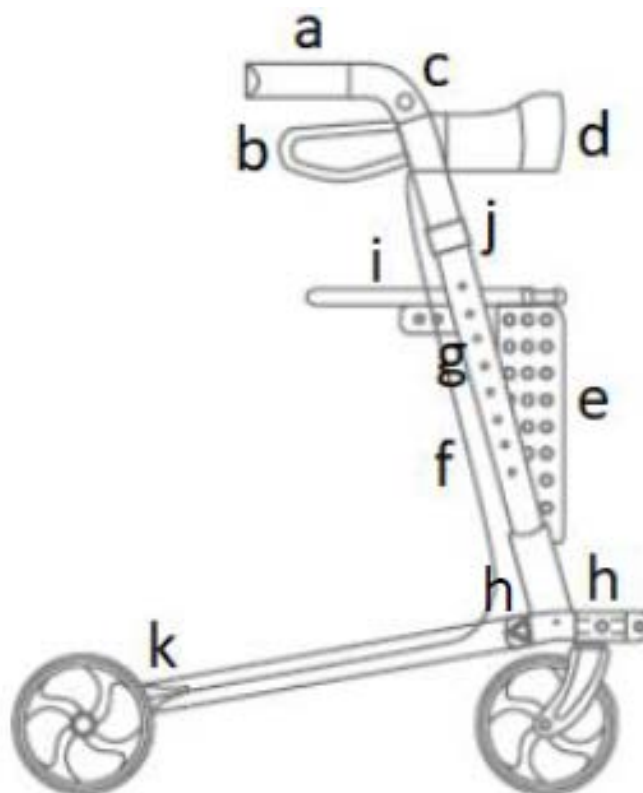

- i. strap for folding rollator
- j. cane holder
- k. lower adjustment of brake

2. Application

The rollator is designed for patients with reduced mobility who need stable support and help while walking. It is also recommended to the elderly as an aid in keeping balance, improving walking comfort and ensuring safety from falling. The use of the rollator affects positively on the return to activity of the people with locomotion and tilting to erect position problems. It guarantees better stabilization, support and security while walking.

3. Assembly

Unfolding and folding the rollator

- Firstly, unfold front wheels and secure them with locking knobs in the mounting openings of the frame in the part of wheels mounting. Then, unfold rear wheels and secure them with locking knobs at the top of the rear legs.
- To unfold the rollator, press the seat in the middle so that all the elements fit together and you hear “click”.
- Fix the shopping bag in the front part of the rollator.
- Before first use, check the tightening of mounting screws.
- Remove plastic casing from wheels.
- To fold the rollator, pull the strap located in the middle of the seat upwards.

Note! Be careful while folding or dismantling the rollator due to the risk of smashing fingers in hinges.

Brakes

- To stop the rollator, pull the brake handle upwards and hold.
- While stop or using the seat, lock the brake with locking mechanism by pushing down the brake handle.

- To release the brake, pull the brake handle upwards.

Height adjustment of handles

- To adjust the height of the handles, press metal pin which is located on the outer side of the frame, select appropriate height and lock the position of the tubes in relation to each other.

Note! The handles must be set on the same level.

Adjustment of brakes

Adjustment of lower brake

- To adjust the brake and increase the tension, unscrew the adjustment screw with the use of appropriate tools and tighten the locking screw by holding the adjustment screw.

4. Technical data

Overall width	64 cm
Overall depth	69 cm
Seat dimensions	46.5 cm x 26 cm
Seat height from the ground	53.5 cm
Handles height adjustment	79.5-92.5 cm
Weight	7.6 kg
Maximum capacity	130 kg
Maximum capacity of the shopping bag	5 kg
Frame colour	titanium, orange, green

5. Notes

Timago International Group does not bear responsibility of improper use of the rollator, failure to comply safety regulations and misuse.

6. Maintenance

- Clean the frame with damp cloth and dry thoroughly.

- Do not use strong polishing agents due to the damage of protecting coatings.
- Clean the dirt from moving parts of the rollator regularly (hinges, clamping screws, locks).










7. Storage and transport conditions

Device should not be exposed to direct sunlight, water or moisture.

8. Warranty information

All products distributed by our company are covered by a warranty, the terms of which are described in the warranty card available on our website. Please contact the retailer from whom you purchased the product. Please, keep in mind that for warranty purposes, the proof of purchase (receipt or invoice) should be retained.

9. Labels

	Reference number		Manufacturer
	LOT number		Manufacture date
	Serial number		Medical device
	Note		Please read the instruction
	The manufacturer checked for compliance with essential medical devices requirements.		

Our complete line of products is available from our distributors. Learn more at: www.timago.com. Thank you for choosing Timago!



Timago International Group
Spółka z o.o. i Spółka – Spółka komandytowa
ul. Karpacka 24/12
43-316 Bielsko-Biała, Polska

T.: +48 33 499 50 00
F.: +48 33 499 50 11
E.: info@timago.com



Rev. II, date 20.05.2021