INSTRUCTIONS FOR USE

Bath grab bars 30 cm / 40 cm / 45cm / 60 cm

Product code: HOLD-TIM-30 / HOLD-TIM-40

HOLD-TIM-45/ HOLD-TIM-60



1.Characteristic

The bath grab bar is made of durable water-resistant plastic. The surface of the grab bar is corrugated which guarantees a firm grip even in wet conditions. The grab bar can be mounted vertically or horizontally. Mounting hole covers are included.

The set includes:

- grab bar
- wall plug with screw - 6 sets



2. Application

Bath grab bar is ancillary equipment which facilitates tilting to erect position and keeping balance. It is recommended for people with reduced mobility that prevents or hinders the unassisted performance of daily living activities. The grab bar also increases the safety of elderly people using sanitary facilities. It is recommended for use in showers, baths, bathrooms and other humid rooms. For use at home and in hospital.

3. Assembly

Step 1. Before installation, determine the place where the grab bar is to be mounted. While pressing the bracket, mark 6 mounting holes on the wall.

Step 2. Using a impact drill, drill 6 mounting holes at the marked locations.

Step 3. Using 6 wall plugs with screws, install the grab bar.

NOTE: the type of wall plugs should be adapted to the type of wall (concrete, brick, hollow block, etc.). The included wall plugs are the universal type.

4. Technical data

Product code	Length / Width	Weight
HANDY-TIM-30	30 cm	0,46 kg
HANDY-TIM-40	40 cm	0,56 kg
HANDY-TIM-45	45 cm	0,61 kg
HANDY-TIM-60	60 cm	0,76 kg

5. Notes

Timago International Group does not bear responsibility of improper use of the bath grab bars, failure to comply safety regulations and misuse.

6. Maintenance

- Clean the grab bar regularly using water, a sponge and mild cleaners for plastics such as soap or mild detergents, wipe dry after each use.
- Do not use for cleaning: strong detergents, abrasives, bleach, gasoline or any solvents.
- Check systematically: tightening of screws, condition of grab bar (for cracks)

7. Storage and transport conditions

Device should not be exposed to direct sunlight, water or moisture.

8. Warranty information

All products distributed by our company are covered by a warranty, the terms of which are described in the warranty card available on our website. Please contact the retailer from whom you purchased the product. Please, keep in mind that for warranty purposes, the proof of purchase (receipt or invoice) should be retained

9. Labels

REF	Reference number		Manufacturer	
LOT	LOT number	\mathbb{Z}	Manufacture date	
SN	Serial number	MD	Medical device	
<u> </u>	Note	[]i	Please read the instruction	
The manufacturer checked for compliance with essential medical devices requirements.				

Our complete line of products is available from our distributors. Learn more at: www.timago.com. Thank you for choosing Timago!



Timago International Group Spółka z o.o. i Spółka – Spółka komandytowa ul. Karpacka 24/12 43-316 Bielsko-Biała, Polska

T.: +48 33 499 50 00 F.: +48 33 499 50 11 E.: info@timago.com



