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USER MANUAL: ELECTRIC STAIR CLIMBING SYSTEM FOR WHEELCHAIR

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INTENDED USE:

The product is used to transport people with limited ability to climb stairs safely due to physical limitations. These are generally people who are wheelchair dependent, for example due to spinal injuries or neurological conditions, or people who still have residual mobility but can no longer climb stairs due to age. A suitably trained operator is required to operate the device.

INDICATIONS:

For the transport of people, regardless of age & gender, with limited ability to climb stairs safely due to physical limitations. Must be used exclusively by qualified and trained personnel.

CONTRAINDICATIONS:

It is contraindicated for use by non-specialized and untrained personnel. Improper use of the device may cause injury to the patient. If the surface is damaged (cracks or breaks), do not use the product.

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CAUTION (!):

In case of damage or malfunction of your product DO NOT use the product and please contact the authorized and trained representative. Do not use the product for a purpose not indicated in this manual. MOBIAS S.A. declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration on the frame of the product. MOBIAS S.A. reserves the right to change the information contained in this document without previous notice. There is a $\pm 3\%$ Deviation for the Dimensions of the Products. It is always recommended to follow the procedures designed for emergency response. Improper use of the device can cause injury to the patient.

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Use only a wet cloth and mild soap on surfaces where they need cleaning. Then dry thoroughly before you use the product.
Dry away from direct sunlight.
DO NOT leave the product wet.
Do not use chemical cleaners to clean the frame. This could cause damage to the surface of the product.
Never use acids, alkalis or solvents such as acetone
If you need to disinfect the product, use a common, mild disinfectant.
Protect the item from scratches, cuts and punctures.

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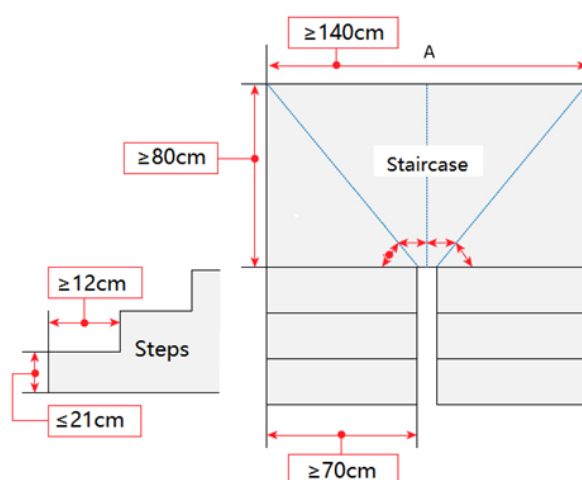
If one day you find that your product needs to be replaced or it is no longer working for you, consider protecting the environment:
1) Do not dispose your product along with the rest of the public waste (this is also the meaning of the shown recycling sign).
2) Contact your Public Authorities and they will instruct you of the Recycling centers to which your product must be disposed.
3) Correct disposal of your product helps the protection of the environment as well as the recycling of the product's components.

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The power switch is located on the battery, press red button to turn on the power supply.

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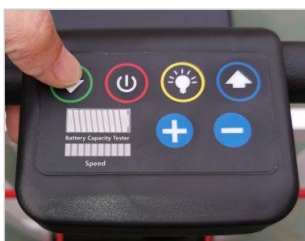
Press the round button on the battery indicator panel. The green lights will turn on if the battery is sufficiently charged. Make sure the unit is charged before use. Each green light corresponds to 25% charge level.

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You can confirm the battery level from the control panel located on the push handle. If the battery level is less than 25%, charging is required.

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Place the system close to the stairs but far enough away to allow the wheelchair user to safely get in. Lock the wheels.

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Extend the adjustable grab bar by pressing the red bar on the back of the frame. When you reach the desired position, release the red bar. You can choose between 3 height positions. Make sure the adjustable grab bar is locked in place by pulling the red bar.

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Adjust each locking handle so that the wheelchair fits onto the system based on the size of the wheelchair.

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Adjust the position of the wheelchair wheel mounting section by pulling the locking ring as shown in the picture to achieve the desired width.

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Place the wheelchair on the climbing system as shown below. The main wheels of the wheelchair must be firmly seated in the red mounting points.

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The locking handles should secure the back of the wheelchair. The headrest should be in a comfortable position for the user. Make any necessary adjustments before moving.

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Secure the wheelchair and user in place using the safety belts. Lock the main wheels of the stair climbing system.

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Place the system at the base of the stairs. Lock the wheels. Visually inspect the stairs to ensure there are no safety hazards such as breaks, cracks, ice or water. Release the tracks by pressing the button shown in the picture. Make sure the tracks are as close to the stairs as possible and locked in place.

Turn on the control unit by pressing the red POWER button. Tilt the climbing system backwards until the tracks contact the stairs.

Press the UP button on the control unit to start the movement. When you reach the top of the stairs, return the system to the upright position. Press the red POWER button to deactivate the track movement. Move the system far enough away from the stairs to allow the wheelchair user to safely exit. Lock the system wheels. Loosen the seat belts and release the wheelchair wheels and locking handles. Push the wheelchair forward until it is far away from the system.

GÅ NED AD TRAPPER / GOING DOWN THE STAIRS

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Place the system at the top of the stairs. Lock the wheels. Visually inspect the stairs to ensure there are no safety hazards such as cracks, breaks, ice or water. Release the tracks by pressing the button shown in the picture. Turn on the control unit by pressing the red POWER button. Tilt the system backwards and carefully push it towards the stairs until the tracks engage the stairs.

Press the DOWN button on the control unit to start the movement. When you reach the bottom of the stairs, bring the system to an upright position. Press the red POWER button to deactivate the track movement. Move the system far enough away from the stairs to allow the wheelchair user to safely remove themselves. Lock the system wheels. Loosen the seat belts and release the wheelchair wheels and locking handles. Push the wheelchair forward until it is removed from the system.

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Remove the red wheelchair wheel mounting parts by pulling the locking ring and sliding them off. Place the system on the ground. By pulling the red bar upwards and then pressing lightly, the system will fold.

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IMPORTANT (1):

The product has function on the motor that when the battery runs out while driving, it can be slide down as a manual chair climbing system.

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Turn off the main switch. The charging port is located next to the power switch. Unscrew the charging port cover. Connect the charging cable to the charging port. Plug the charging cable into a power outlet. When the battery is charged, remove the charging cable and screw the charging port cover back into place. The battery has two keys and can be used to lock the battery to the unit.

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The product is guaranteed for a period of two (2) years from the purchasing date.

The warranty apply on factory defects & DOES NOT apply on damages resulting from misuse, poor maintenance, modification, overuse, or non-compliance with the product's operating instructions.

Also DOES NOT apply on the parts, that can wear out during use or over time.

Damages or defects caused by natural disasters, unauthorized maintenance or repair, power supply problems (where applicable) & carriers are not covered by the warranty.

No service or spare part is applied unless it has been inspected by an authorized service department or the manufacturer of the product.

Shipping costs for goods of warranty are to be paid by the end user-customer or by dealer.

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Any serious incident that occurs in relation to use of the device must be reported to the manufacturer and a competent authority of the Member State in which the user and/or patient resides.

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We are solely responsible for declaring that the Medical Devices mentioned in this statement are of Low-Risk Class (Class I) and comply with the requirements of the European Regulation 2017/745 and where appropriate, the standards and legislation referred to.