REF HandyTim

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BRUGSANVISNING

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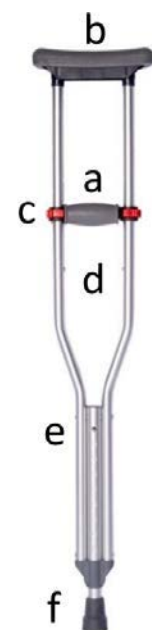
Product code: HandyTim

1. Characteristics

The underarm crutch consists of durable, aluminium leg with rubber tip, soft crutch pad, and adjustable handgrip. Owing to wide range of adjustment, the crutch can be used either by short or tall patients. Ergonomic shape and universal size make the crutch comfortable in use.

- a) handgrip
- b) crutch pad
- c) handgrip adjustment – locks
- d) height adjustment – upper locking pins
- e) height adjustment – lower locking pins
- f) rubber tip

2. Application

The underarm crutch is a support equipment which is used in case of lower limb injury with simultaneous injuries of wrist or elbow. It is also recommended to patients who need help and support while walking for gait re-education after lower limb injuries, and it can also be used as an aid in keeping balance in case of serious dysfunctions of lower limbs or trunk.

Contraindications: coordination and balance disorders, advanced rheumatic changes, injuries and dysfunctions of upper limbs that make it difficult for the user to support on the crutch.

3. Adjustment

- The crutch should be adjusted individually to the user so that the shoulder joints are not excessively raised and the level of grip provides a slight flexion at the elbow joint (at an angle of about 30°).
- To adjust the position of the handgrip, unlock both red locks and adjust the handgrip to the appropriate level. Then lock both locks.
- To adjust the length of the upper part of the crutch, first unlock both red handle locks. Then simultaneously push in both metal pins on the inner surface of the tubes and select the appropriate position of the adjustment holes. Once set, lock the two metal pins again (step adjustment in 2.5 cm increments).
- To adjust the length of the lower part of the crutch, push in the metal pins on both sides of the tube and select the appropriate position of the adjustment holes. Once set, lock both metal pins again (step adjustment in 2.5 cm increments).
- When using two crutches, all settings should be identical (handgrip, upper and lower pins). Handgrips and locking pins must be set at the same level. The leg of the crutch should always stand stably. The rubber tip should always have a firm, stable contact with the ground.

NOTE!

- Before use, ensure that the crutch is stable and the locking pins fully stick out through adjustment hole of the crutch.
- It is recommended to use two crutches simultaneously.
- Long-term use of underarm crutches may cause injury of axillary nerves - possible occurrence of: sensory and motor disorders, muscle weakening, numbness, and, as a result, muscle paralysis).

NOTE! In case of doubts concerning the technique of moving with the use of crutches, consult a person who has appropriate knowledge or qualifications.

4. Technical data

Type: triple adjustment

Colour: silver, black

Maximum capacity: 136 kg

Height: 97.5-147.5 cm

Crutch weight: 0.92 kg

5. Notes

Timago International Group does not bear responsibility of improper use of the product, failure to comply safety regulations and misuse.

6. Cleaning and maintenance

- Clean the crutch using only water and mild detergents, dry thoroughly.
- **Do not use:** strong detergents, abrasive agents, bleach, petrol nor solvents (it may cause frame damage, visible defects, and in consequence – loss of warranty rights).
- It is recommended to check rubber tip regularly. Replace worn tip immediately.

7. Storage and transport conditions

Device should not be exposed to direct sunlight, water or moisture.

8. Warranty information

All products distributed by our company are covered by a warranty, the terms of which are described in the warranty card available on our website. Please contact the retailer from whom you purchased the product. Please, keep in mind that for warranty purposes, the proof of purchase (receipt or invoice) should be retained

9. Labels

	Reference number		Manufacturer
	LOT number		Manufacture date
	Serial number		Medical device
	Note		Please read the instruction
	The manufacturer checked for compliance with essential medical devices requirements.		

Our complete line of products is available from our distributors.
Learn more at: www.timago.com. Thank you for choosing Timago!

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