INSTRUCTIONS FOR USE

Aluminium 4-wheeled rollator

Product code: RANGER



1. Characteristics

Four-wheeled, folding frame made of aluminum frame. Equipped with parking brakes, seat with backrest, crutch/cane holder, cup/bottle holder and handy bag. It has pneumatic 12" diameter wheels and height-adjustable handles finished with anatomical handgrips. The structure is based on a crossover frame and an oval cross-section of tubes, thanks to which the resistance to axial pressure is significantly increased.

The set includes: frame with seat, front wheels, legs with rear wheels, backrest, shopping bag.

Elementy podpórki:

- 1. Handgrip
- 2. Adjustable tubing handgrip
- 3. Parking brake
- 4. Adjustment clamp
- 5. Backrest
- **6.** Seat with handle
- 7. Bag hook
- 8. Front wheel
- 9. Brake adjustment
- 10. Rear wheel
- 11. Safety clamp
- 12. Kerb climber
- **13.** Crutch/cane holder
- **14.** Locking pins
- **15.** Cup holder



2. Application

The rollator is designed for patients with reduced mobility who need stable support and help while walking. It is also recommended to the elderly as an aid in keeping balance, improving walking comfort and ensuring safety from falling. The use of the rollator affects positively on the return to activity of the people with locomotion and tilting to erect position problems. It guarantees better stabilization, support and security while walking.

3. Assembling, unfolding, folding and adjustment

Assembling

- Unpack all components from the box and lay out on a flat surface (frame with 2 rear legs connected by brake lines, 2 front wheels, backrest, shopping bag).
- Pressing the locking pins, insert both rear legs with wheels into the holes in the frame of the rollator.
 (the right and left legs are connected to the correct side of the rollator's frame by brake cables).
- Pressing the locking pins, insert both front wheels into the holes in the front forks of the rollator's frame.
- Install the backrest into the clamps on the inside of the handles.
- Hang the bag on the hooks.







Strona 2 z 8

Unfolding

- To unfold the rollator, grab both handgrips and pull them outward.
- Press down on the seat rails until it is fully unfolded (click of the locking mechanism).

Folding

- To fold the rollator, lift the seat by the handle and pull it up.
- Grab both handgrips of the rollator and press them inward.

Handgrips height adjustment

To adjust the height of the handgrip:

- Press the brake down to set the parking mode and immobilize the rollator.
- Pull and hold the adjustment clamp.
- Adjust the height of the handles
- Release the adjustment clamp to lock the adjusted height of the handles. Pull the handle up and down several times to check the correct height locking.
- Adjust the height of the second handgrip in a similar way.
- Both handgrips should be set at the same height.



Brake function

- To stop the rollator, pull the brake handle up toward the handle, then hold.
- When stationary or using the seat, lock the brake with the parking clamp by pulling the brake handle down.
- To release the brake, pull the brake handle upward.





Brake adjustment

Unscrew the cable holder, pull it up to the desired distance and then lock it by tightening the clamping screw.



4. Technical data

Overall width	67 cm
Overall depth	75,5 cm

Seat width	21 cm	
Total height	89,5-102,5 cm	
Seat height from the	59 cm	
ground		
Weight	10,8 kg	
Maximum capacity	136 kg	
Maximum capacity of the	5 kg	
shopping bag		
Frame colour	White	

5. Notes

Timago International Group does not bear responsibility of improper use of the rollator, failure to comply safety regulations and misuse.

6. Maintenance

- Clean the frame with damp cloth and dry thoroughly.
- Do not use strong polishing agents due to the damage of protecting coatings.
- Clean the dirt from moving parts of the rollator regularly (hinges, clamping screws, locks).

7. Storage and transport conditions

Device should not be exposed to direct sunlight, water or moisture.

8. Warranty information

All products distributed by our company are covered by a warranty, the terms of which are described in the warranty card available on our website. Please contact the retailer from whom you purchased the product. Please, keep in mind that for warranty purposes, the proof of purchase (receipt or invoice) should be retained.

9. Labels

REF	Reference number	***	Manufacturer
LOT	LOT number	\mathbb{A}	Manufacture date
SN	Serial number	MD	Medical device
<u> </u>	Note	[]i	Please read the instruction
The manufacturer checked for compliance with essential medical devices requirements.			

Our complete line of products is available from our distributors. Learn more at: www.timago.com. Thank you for choosing Timago!



Timago International Group Spółka z o.o. i Spółka – Spółka komandytowa ul. Karpacka 24/12 43-316 Bielsko-Biała, Polska

T.: +48 33 499 50 00 F.: +48 33 499 50 11 E.: info@timago.com



