REF JMC-C 3223

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BRUGSANVISNING

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Product code: JMC-C 3223

1. Characteristics

4-wheeled walker with brakes is made of crosswise reinforced aluminium frame. The walker is equipped with 5" solid wheels on the front, hand brakes, including additional parking brakes, and interchangeable rear legs. Rubber handgrips provide proper stabilization and comfort of use. The height of the walker can be adjusted to the user's height. When folded, the walker takes up little space, making it easy to store and transport.

Elements of the walker:

- a. Handgrip
- b. Frame folding button
- c. Height adjustment
- d. Anti-sliding rubber tip
- e. Wheels 5" in diameter

2. Application

4-wheeled walker with brakes is an support equipment designed for people with reduced mobility who need stable support and assistance while walking. It is also recommended for the elderly as an aid in keeping balance, improving walking comfort and ensuring safety from falling. The use of the walker has a positive effect on the return to activity by people with lower limbs conditions and with locomotion or verticalization problems.

3. Assembly

Unfolding and folding

- To unfold the walker, pull the sides apart to the maximum position until the handgrips are locked.
- To fold the walker, press the rubber elements located at the top of the transverse frame and fold the sides inwards.

Height adjustment

- The height should be set on such a level that the user, who is leaning against the handgrips, stands upright, as far as he or she is able to, with his or her shoulders in rest position and with slightly bent elbows. It is inadvisable to hang on the walker with all the body weight.
- To adjust the height of legs, press the metal pin, select appropriate height and lock the position of all four legs. The legs are adjustable in 2.5 cm increments.

Note! The pins must be set on the same level in each leg.

Selection of functions

- Legs with tips or with wheels can be mounted on the back side of the walker.
- The brakes are equipped with a parking lock function, activated by pulling the handbrakes to a maximum position. To release the lock, grasp the brake handle, squeeze it and pull back the locking element (located under the brake handle).

4. Technical data

Overall width	59,5 cm
Inner width	46 cm
Overall depth	54 cm
Height adjustment	79-91 cm
Weight	3,8 kg
Maximum capacity	136 kg

5. Notes

Timago International Group does not bear responsibility of improper use of the walker, failure to comply safety regulations and misuse.

6. Maintenance

- Clean the frame of the walker with a damp cloth using water and mild detergent only. Dry thoroughly.
- **Do not use** for cleaning: strong polishing and abrasive agents, strong detergents, bleach, petrol or solvents (it may cause frame damage, visible defects and in consequence - loss of warranty rights).
- Remove dirt from moving components of the walker (hinges, clamping screws, locks) on a regular basis.

7. Storage and transport conditions

Device should not be exposed to direct sunlight, water or moisture.

8. Warranty information

All products distributed by our company are covered by a warranty, the terms of which are described in the warranty card available on our website. Please contact the retailer from whom you purchased the product. Please, keep in mind that for warranty purposes, the proof of purchase (receipt or invoice) should be retained.

9. Labels

	Reference number		Manufacturer
	LOT number		Manufacture date
	Serial number		Medical device
	Note		Please read the instruction
	The manufacturer checked for compliance with essential medical devices requirements.		

Our complete line of products is available from our distributors.

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