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***BRUGSANVISNING***

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Product code: BathTim

### 1. Characteristics

Bath bench with wide seat and backrest is made of durable PVC material. It has 10 levels of height adjustment and armrest that makes it easier to use. Special rubber tips with suction cups and anti-slip tips effectively reduce chances of sliding on wet surfaces, providing better stability.

**The set includes:** backrest made of PVC, seat legs with suction cups – 2 pcs., legs with anti-slip tips – 2 pcs., mounting tubes – 2 pcs., armrest.

- 1 universal backrest made of PVC
- 2 seat made of PVC with drain openings
- 3 armrest
- 4 height adjustment
- 5 suction cups
- 6 anti-slip tips

### 2. Application

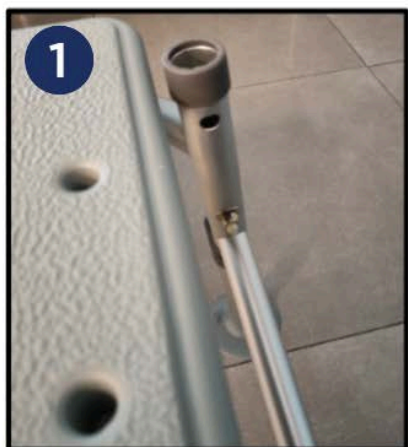
The bath benches are ancillary equipment which helps the user to perform hygienic activities. The bench is recommended to people with reduced mobility which hinders the unassisted performance of daily

living activities. Bench is recommended for use in shower, bathtub, bathroom and other damp rooms, at home and hospital.

### 3. Assembly

**Step 1.** Turn the seat and lay it flat on the ground. Fix the legs one after the other on both sides.

- Legs with suction cups should be mounted on the same side as armrest (picture 1).

**Step 2.** Fix tubes in seat openings (picture 2). Make sure that both locking pins are locked. Turn frame with seat on a side. Put backrest frame through openings located at the back of the seat until locking pin clicks (picture 3).

- Backrest has two level length adjustment (picture 3). Press locking pin and set appropriate length.

**Step 3.** Put bench on its legs and fix armrest into openings (picture 1). Secure handle with locking pins.

### HEIGHT ADJUSTMENT

To adjust bench height, press locking pin and set appropriate height for the user and lock it with a pin again. The legs are adjustable in 1 cm increments. All legs has to be set on the same height level.

### 4. Technical data

Seat dimensions	67 x 41 cm
Adjustable height	46-58 cm

Weight	4,70 kg
Maximum capacity	136 kg

## 5. Notes

**Timago International Group does not bear responsibility of improper use of the bench, failure to comply safety regulations and misuse.**

## 6. Maintenance

- Clean the bath bench regularly using only water, sponge and mild detergents for plastic materials such as soap. Dry thoroughly.
- Do not use for cleaning: strong detergents, abrasive agents, bleach, petrol or solvents (it may cause frame damage, visible defects and in consequence - loss of warranty rights).
- Regular inspection of the rubber cap, locking pins, mounting tubes, backrest and seat is recommended.

## 7. Storage and transport conditions

Device should not be exposed to direct sunlight, water or moisture.

## 8. Warranty information

All products distributed by our company are covered by a warranty, the terms of which are described in the warranty card available on our website. Please contact the retailer from whom you purchased the product. Please, keep in mind that for warranty purposes, the proof of purchase (receipt or invoice) should be retained

## 9. Labels

	Reference number		Manufacturer
	LOT number		Manufacture date
	Serial number		Medical device

	Note		Please read the instruction
	The manufacturer checked for compliance with essential medical devices requirements.		

Our complete line of products is available from our distributors.

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