

INSTRUCTIONS FOR USE

Transport commode chair

Product code: FS 692



1. Characteristics


Transport commode chair is made of powder coated steel frame. Flip-up armrests facilitate patients with considerably reduced mobility getting in and out the chair. The chair is equipped with soft seat which is upholstered with waterproof and easy to clean material, four swivel castors, the rear of which have locks, swing away footrests, and toilet bucket. Owing to the structure of the chair, it is possible to slide onto the toilet.

The set includes:

- Frame
- Seat
- Backrest
- Right footrest with footplate
- Left footrest with footplate
- Protective strap
- Key for height adjustment of footrests
- Toilet bucket with lid

Elements of the wheelchair:

- a. Locking clips of the frame
- b. Seat
- c. Armrest
- d. Locking pin
- e. Footrest
- f. Assembly pin of the footrest
- g. Castor lock
- h. Toilet bucket

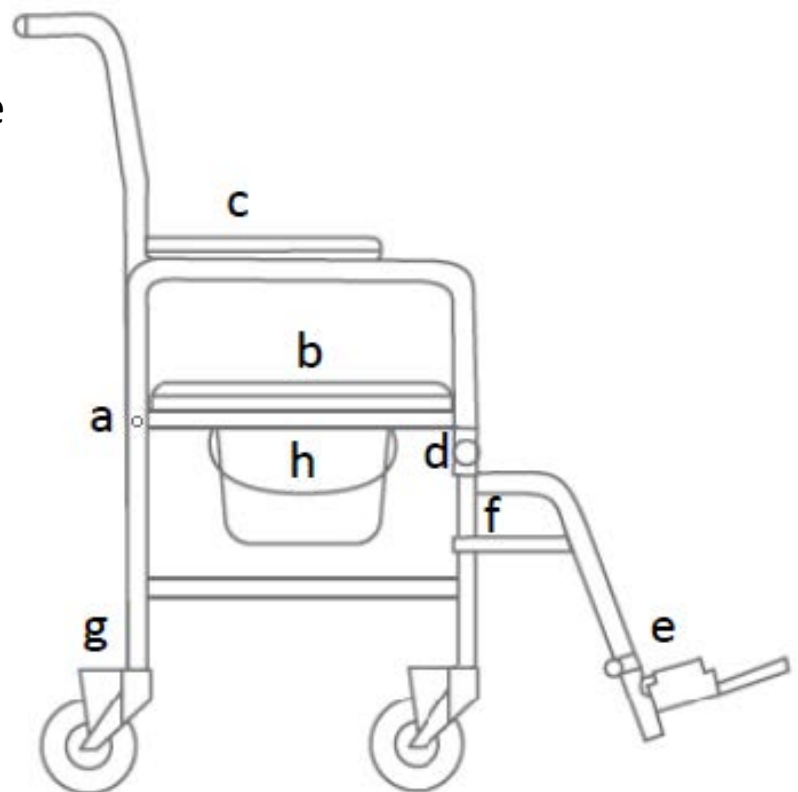

2. Application

Transport commode chair is designed for people who have problems with unassisted moving and for patients with dysfunctions and reduced mobility. The chair is suitable for indoor use only. The chair can be handled only with the help of an accompanying person.

The chair functions as a transport-toilet chair owing to the toilet bucket located under the seat and four swivel castors. It facilitates the user everyday hygiene. The chair is especially recommended to people who have problems with mobility which are caused by:

- paralysis
- overweight or abnormal underweight
- degenerations or deformity of limbs
- degenerations or joints injuries
- limbs injuries or amputations
- heart and blood vessels disease, circulatory disorders

3. Assembly

Step 1. Remove locking clips which are located on the frame of the backrest.

Step 2. Slide the backrest tubes into the two rear parts of the main frame and lock the backrest with locking clips.

Step 3. Assemble the armrest in the front part of the frame by pressing locking pins.

Step 4. Assemble the right and the left footrest in the front part of the frame by placing them on two pins and turn them inwards until they are locked.

Step 5. Assemble and adjust the protective strap on the footrest.

Step 6. From the back slide the toilet bucket without the lid into the rails which are located under the seat. (Check the correctness of the assembled toilet bucket).

The wheelchair is ready to use.

WHEELCHAIR HANDLING

- Check the condition of the castor lock, construction and tyres. Check also the correctness of the assembled components and their adjustment.
- Before using, make sure that the chair width (45 cm) matches to the user, as the chair is not equipped with width adjustment.
- The chair is designed to be used only on short distances inside the rooms. The chair does not have shock absorber so that long usage is not recommended.

Note! Do not use the chair if any defects or deficiency are noticed.

FLIP-UP FUNCTION OF ARMRESTS

- Press the locking pin, remove the front part of the armrest from the lock, and move it downwards.
- To close the armrest, move it upwards until the locking button clicks.
- **Do not** use the armrest as a support while standing up. If required ask the third person for help.

BRAKES

- To lock the castors, press the lever of the right and left lock with foot. To release the castor, pull the lever upwards with foot.

FOOTRESTS

- The footplates should be in vertical position when getting in or out the wheelchair. **Do not use the footplates as a support while getting in or out the wheelchair due to the risk of wheelchair tipping over and falling.**

GETTING IN THE WHEELCHAIR

- While getting in or out the wheelchair, changing the user's position or stop, the castors must be absolutely.

- The wheelchair should be located at the place of changing the seat as close as possible. The wheelchair should touch the bed or other seat being used, rear castor and front element of the frame. If the user cannot change his/her seat independently, transfer the user to the wheelchair.
- The user should sit as deep as possible in the middle of the seat in order to avoid chair instability.

4. Technical data

Overall width	55 cm
Inner width	45 cm
Overall depth	87 cm
Seat height from the ground	53 cm
Overall height	95 cm
Seat dimensions	44 x 44 cm
Weight	10.60 kg
Maximum capacity	100 kg
Folded width	55 cm
Frame color	red

5. Notes

Timago International Group does not bear responsibility of improper moving in the commode chair, failure to comply safety regulations and misuse.

6. Cleaning and maintenance

Carry out the maintenance of the wheelchair components regularly:

- The upholstery and the cushion can be cleaned with damp (not wet) cloth and cold or warm water, without the use of detergents. Difficult stains can be cleaned with a sponge. Do not soak the upholstery too much. Dry away from heat sources.

- Plastic elements should be cleaned with damp cloth possibly with the use of appropriate detergents for plastic elements. In such case follow the instructions given by the producer and safety measures.
- Empty and disinfect the toilet bucket and toilet seat after each use. Use cleaning agents according to producer's recommendations.










7. Storage and transport conditions

Device should not be exposed to direct sunlight, water or moisture.

8. Warranty information

All products distributed by our company are covered by a warranty, the terms of which are described in the warranty card available on our website. Please contact the retailer from whom you purchased the product. Please, keep in mind that for warranty purposes, the proof of purchase (receipt or invoice) should be retained

9. Labels

	Reference number		Manufacturer
	LOT number		Manufacture date
	Serial number		Medical device
	Note		Please read the instruction
	The manufacturer checked for compliance with essential medical devices requirements.		

Our complete line of products is available from our distributors.
Learn more at: www.timago.com. Thank you for choosing Timago!



Timago International Group
Spółka z o.o. i Spółka – Spółka komandytowa
ul. Karpacka 24/12
43-316 Bielsko-Biała, Polska

T.: +48 33 499 50 00
F.: +48 33 499 50 11
E.: info@timago.com



Rev. II, date 20.05.2021