REF

DRIFT-TIM

PL INSTRUKCJA OBSŁUGI
CS NÁVOD K OBSLUZE
DA BRUGSANVISNING
DE BENUTZERHANDBUCH
EL ΟΔΗΓΙΕΣ ΧΡΗΣΗΣ
EN USER MANUAL
ET KASUTUSJUHEND
HR KORSNIČKE UPUTE
HU FELHASZNÁLÓI ÚTMUTATÓ
IT MANUALE D'USO
LT NAUDOJIMO INSTRUKCIJA
LV LIETOTĀJA INSTRUKCIJA
PT MANUAL DO UTILIZADOR
RO MANUAL DE INSTRUCTIUNI
SK NÁVOD NA POUŽITIE
SL NAVODILA ZA UPORABO
SV BRUKSANVISNING

PL INSTRUKCJA OBSŁUGI	5
CS NÁVOD K OBSLUZE	11
DA BRUGSANVISNING	16
DE BENUTZERHANDBUCH	21
EL ΟΔΗΓΙΕΣ ΧΡΗΣΗΣ.....	27
EN USER MANUAL.....	33
ET KASUTUSJUHEND	38
HR KORSNIČKE UPUTE	43
HU FELHASZNÁLÓI ÚTMUTATÓ	48
IT MANUALE D'USO.....	54
LT NAUDOJIMO INSTRUKCIJA	59
LV LIETOTĀJA INSTRUKCIJA.....	64
PT MANUAL DO UTILIZADOR	69
RO MANUAL DE INSTRUCTIUNI.....	75
SK NÁVOD NA POUŽITIE	80
SL NAVODILA ZA UPORABO	85
SV BRUKSANVISNING	90

ELEMENTY WÓZKA

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ELEMENTS OF THE WHEELCHAIR

- | | |
|-------------------------------------|-------------------------------|
| 1. Handgrip | 10. Calves belt |
| 2. Attendant brake | 11. Armrest adjustment button |
| 3. Backrest folding mechanism | 12. Backrest |
| 4. Armrest | 13. Seat with security belt |
| 5. Parking brake (hand brake) | 14. Footrest lever |
| 6. Handrim | 15. Solid, rear 24" wheel |
| 7. Quick release mechanism | 16. Solid, front 8" wheel |
| 8. Footplate | 17. Anti tip wheel |
| 9. Center of the gravity adjustment | |

1. Characteristics

The wheelchair is made of a lightweight aluminum frame. It features height-adjustable profiled armrests and an optional detachable upholstery (for backrest and seat) made of a breathable mesh material. Thanks to the quick release function on rear wheels and centre of gravity adjustment on front and rear wheels, wheelchair can be adjusted to the user's individual needs. It is also equipped with attendant brakes and a foldable backrest. Available in version on rear, solid 24" wheels and front, solid 8" wheels.

The set includes: wheelchair frame with wheels, seat and backrest, user belt, assembly keys.

Optional accessories: anti-tip wheels, footrests with safety tape, mesh up upholstery.

2. Application

The wheelchair is designed for people who have problems with unassisted moving and for patients with dysfunctions and reduced mobility. This self-propelled wheelchair can also be pushed by an accompanying person. Thanks to the ability to adjust the settings to the individual needs of the user, the wheelchair provides better control, greater stability, and a more dynamic, active ride on everyday bases. The wheelchair is suitable for indoor and outdoor use.

3. Assembly

Step 1. Unfolding the wheelchair

Place the wheelchair on the ground. Unfold the wheelchair using the armrests. Then, press the front and rear edge of the seat so that all the elements fit together.

Step 2. Backrest and seat

Lift the backrest upwards and lock it with backrest locks which are located below the hand grips. Fix the upholstery on the wheelchair and fasten the lower part of the upholstery with the use of velcro under the seat.

Step 3. Footrests

Fix the plastic grip of the footrest on the outer side of wheelchair frame. Then, put the assembly element of the footrest into the opening in the front of the frame turning the footrest inwards until it is locked.

Step 4. Armrests

The armrests are adjustable in height with the use of a pin on the outer side of the armrest.

Step 5. Height adjustment of footrests

Adjust the right and the left footrest by tightening the screw which is placed on the footrest tube with included key. Proper space should be left between the footplates and the ground as footplates set too low may cause difficulties in overcoming such obstacles as curbs. Adjust the footplates so that the angle between trunk and thighs is 90° (optimum sitting position).

Step 6. Assembly of protective strap

Put the protective strap on the upper part of the footrest to prevent user's legs from sliding down the footplates.

FOLDING

The wheelchair can be re-folded in order to be transported and stored more easily.

- Swing away the footplates and set them in vertical position.
- Detach the upholstery from the seat.
- Use the handle to lift the seat upwards until the frame of the wheelchair is folded.
- To fold the backrest, simultaneously press locking elements which are located at the back of the backrest and pull the backrest downwards.

PARKING BRAKE

To lock the wheels, push the parking brake lever forward until the wheel is fully blocked (separately for left and right wheel). Please, note that the wheel lock only serves to lock the wheels when the wheelchair is at a complete stop.

ATTENDANT BRAKES

The levers of the brakes are located behind the backrest, under the hand grips on the right and left side, and are connected with wheel lock by cable.

- In order to brake, squeeze the lever of the brake and in order to release the brake, drop the levers. Perform this activity smoothly and with appropriate force.
- The brakes are equipped with a parking lock. To apply it, grasp the brake lever, squeeze it and pull out the locking element.

PARKING BRAKE ADJUSTMENT

The correct lock of the parking brake depends on its adjustment and position. The handle of the wheel lock should be positioned so that the brake shoe blocks the wheel firmly and the lever does not face excessive resistance.

- In case of excessive resistance, pull the handle slightly forward by twisting off the locking screw which is located on the other side of the parking brake (from the inside of the wheelchair). Set the brake shoe in such position that it blocks the wheel easily without excessive resistance.
- When the parking brake shoe does not touch or lock the wheel sufficiently, follow the above instruction by moving the handle in the opposite direction.

4. Before use

Step 1. Checking the condition of the wheelchair

Check the condition of the wheel locks, construction and tires. Also, check also the correctness of the assembled components and their adjustment. **Do not** use the wheelchair if any defects or deficiencies are noticed.

Step 2. Engaging the wheel locks

Engage the wheel locks when getting in or out of the wheelchair.

Step 3. Setting the footplates

The footplates should be in vertical position when getting in or out the wheelchair.
Note! Do not stand on the footplates due to the risk of damage. Do not use the footplates as a support while getting in or out the wheelchair due to the risk of tipping over and falling.

Step 4. Setting the wheels

Front wheels should be positioned in the direction of riding.

Step 5. Getting in the wheelchair

The wheelchair should be located at the place of changing the seat as close as possible. The wheelchair should touch the bed or other seat being used, rear wheel and front element of the frame. If the user cannot change his/her seat independently, transfer the user to the wheelchair.

Step 6. Unfolding the footplates

When the user is sitting in the wheelchair, unfold the footplates parallel to the ground so that his/her feet are on the footplates while using the wheelchair.

Step 7. Correct position of the user

The user should sit in the upright position and his/her knees should be bent at the angle of 90°.

The wheelchair is ready to use.

5. Technical data

Seat dimensions: 43 / 46 x 41 cm	Overall height: 91 cm
Overall width: 64 / 67 cm	Weight: 11,73 / 11,78 kg
Overall depth: 106 cm	Maximum capacity: 136 kg
Seat height from the ground: 50 cm	Folded width: 34 cm

Frame color: red, green

6. Notes

Timago International Group does not bear responsibility of improper use of the product, failure to comply safety regulations and misuse.

7. Cleaning and maintenance

Carry out the maintenance of the wheelchair components regularly:

- The upholstery and the cushion can be cleaned with damp (not wet) cloth and cold or warm water, without the use of detergents. Difficult stains can be cleaned with a sponge. Do not soak the upholstery too much. Dry away from heat sources.
- Plastic elements should be cleaned with damp cloth possibly with the use of appropriate detergents for plastic elements. In such case follow the instructions given by the producer and safety measures.

8. Storage and transport conditions

The product should not be exposed to direct sunlight, water or moisture.

9. Warranty information

All products distributed by our company are covered by a warranty, the terms of which are described in the warranty card available on our website. Please contact the retailer from whom you purchased the product. Please, keep in mind that for warranty purposes, the proof of purchase (receipt or invoice) should be retained.

10. Medical device incidents

Any serious incident involving a medical device must be reported to the manufacturer and to the competent authority of the European Union member state where the user or patient resides.

11. Labels

	Reference number		Manufacturer
	LOT number		UDI code
	Serial number		Medical device
	Note		Please read the instruction
	The product meets the legal requirements of the European Union for medical devices (MDR 2017/745)		

Our complete line of products is available from our distributors. Learn more at: **www.timago.com**. Thank you for choosing Timago!

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