USER MANUAL

Electric wheelchair

Product code: SMART-TIM

1. Security information

- Read the manual before first use.
- Do not use the wheelchair until fully assembled.
- Use of the wheelchair by people who are unable to operate it independently is not recommended.
- Maximum load capacity must not exceed 115 kg.
- The wheelchair can only be used by one person at a time.
- It is recommended to keep your feet on the footrest and your forearms on the armrests when driving.
- Always check the battery level on the control panel before use.
- This electric wheelchair should only be used on a flat surface or at low elevations. It should not be ridden on a surface inclined at an angle of more than 6 degrees.
- Do not drive the wheelchair at maximum speed when going uphill or downhill.
- Do not use the wheelchair on soft and slippery surfaces such as mud, sand etc.
- Do not ride the wheelchair sideways on highly elevated areas.
- Do not put any additional load such as a trailer etc. on the wheelchair.
- Do not repair or modify the construction of the wheelchair by yourself. Use only original spare parts provided by the manufacturer.
- Do not stand up or sit down on the wheelchair when the control module is switched on.
- Do not use the wheelchair without anti-tip wheels.
- When the freewheel (manual) mode is on, do not get on or off the wheelchair without assistance.
- Do not lean forward to grab items laying in front, on the side or behind the wheelchair. Do not stand on the footrest. It might result in the wheelchair falling over.
- Avoid rapid changes of the direction while driving on elevations.
- Always slow down while driving downhill or making turns.
- Before use check if the wheels are correctly mounted.
- Do not make sudden moves while using the joystick.

- When storing, protect the wheelchair from rain and adverse weather conditions.
- Before turning on the wheelchair, make sure that the levers are set to electric mode.
- Avoid areas with horizontal gaps.
- If the wheelchair is not going to be used for a long period of time – remove the battery.
- Do not use the wheelchair on the road. It might result in a car accident.
- Do not store the wheelchair in places with high temperatures and humidity.

2. Product description and application

Electric wheelchair is designed for people who have problems with mobility, helping disabled patients with their everyday tasks. It can replace a standard wheelchair.

Electric wheelchair is suitable for patients with:

- disability, who have problems with unassisted movement,
- dysfunctions and reduced movement,
- heart and cardiovascular system diseases,
- weakening body due to advanced age.

A. ELEMENTS OF THE WHEELCHAIR

1. Handle
2. Backrest
3. Seat
4. Armrest
5. Control module
6. Footrest
7. Rear wheel 8"
8. Front wheel 6"
9. Mounting for control module
10. Safety belt
11. Armrest folding button
12. Battery holding rails

- 13. Pocket for small goods
- 14. Engines and operation mode levers
- 15. Shock absorbers (front and back)
- 16. Controller box
- 17. Anti-tip wheels
- 18. Folding clamp

Wheelchair after folding

B. PRODUCT CHARACTERISTICS

- Aluminium, folding frame
- Solid rear 8" and front swivel 6" wheels
- Control module with LCD display
- Safe and reliable electromagnetic brakes
- Fully closed, maintenance-free battery with handle
- Brushless engines
- Security belt with lock system and anti-tip wheels
- Flip-up armrests and footrest
- Front and back shock absorbers
- Pocket for small goods located at the back
- Reflective stickers included
- Two modes of operation: electric and manual

C. TECHNICAL PARAMETERS

1. Product type: for indoor and outdoor use
2. Maximum speed: ≤ 6 [km/h]
3. Maximum capacity: 115 [kg]

4. Braking distance on horizontal surface: ≤ 1 [m]
5. Max. range: 18 [km]
6. Battery: lithium, 24 [V] x 12 [Ah]
7. Max. angle of elevation: $\leq 6^\circ$
8. Max. turning radius: 0,90 [m]
9. Wheelchair weight: 14,70 kg
10. Battery weight: 2,30 kg

Note! Some parameters might vary depending on user weight, road conditions and battery usage.

- Safety Classification: type B
- Ingress protection level for engine: IP54
- Engine power level: 2x150[W]
- Wheelchair cannot be used in areas with flammable gas mixed with air, oxygen or nitrous oxide.

3. Icon description


Symbol	Meaning
	Safety Classification type B
	Important! Read the user manual
IP54	Protection against limited dust ingress and water spray from any direction.
	Fragile
	This side up
	Protect from humidity and moisture

4. Assembly and adjustment

- **Unfolding:** Take the wheelchair out of the box. Unfold it, till you hear a “click” sound. Mount battery on the rails underneath the seat from the front of the wheelchair. Push the battery further to make sure it’s connected to the power socket. Mount the control module and connect to the module power socket. Power on by pressing the on/off button located on the battery.
- **Folding:** To fold the wheelchair, unlock the clamp located at the back, grab the handle and push it down.

Note! Remove the battery before folding the wheelchair.

- **Control module adjustment:** Adjustment should be done while seated. Untighten safety buckle and loosen the mounting screw, adjusting the position or length of the module. Tighten the screw and buckle afterwards.
- **Armrest tilting:** Press the buttons located on the outside of the wheelchair to lift the armrests. To lower the armrest, pull them down until you hear a “click” sound.
- **Anti-tip wheels assembly:** Put the wheels into the mounting openings located at the back of wheelchair. Pull the blocking screw and push wheels till you hear click sound.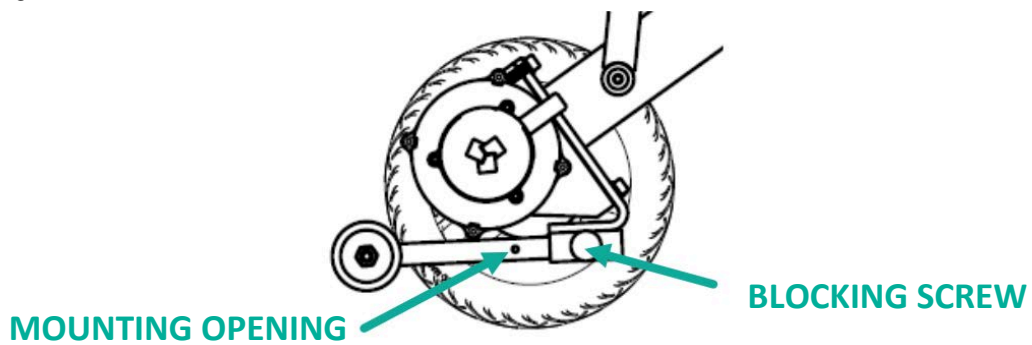


- **Removing the battery:** To remove the battery, slowly pull the battery out from the front by the handle.

Note! Before removing battery, make sure power button on battery is turn off.

5. Operation

A. CONTROL MODULE FUNCTIONS

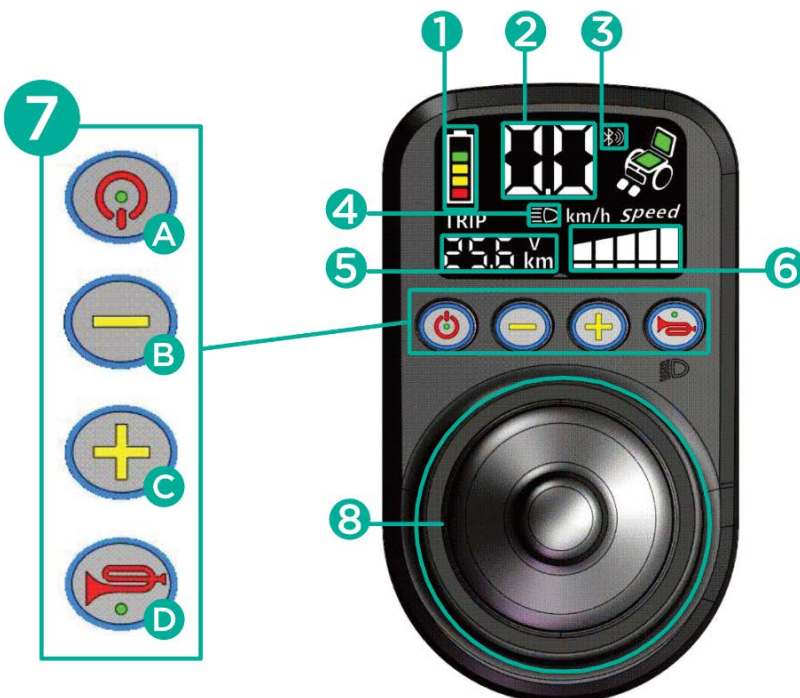1. Battery level
2. Current speed display
3. Bluetooth connection
4. Flashlight on/off
5. Drive distance
6. Speed levels
7. Operation buttons
 - A. Power on/off
 - B. Speed level down (press)
Warning function (hold)
 - C. Speed level up (press)
Voice assistant on/off (hold)
 - D. Light on/off (press)
horn (hold)
8. Joystick

- **Joystick (8)** – used to control the speed and direction. The further the joystick is pushed away from the middle, the faster the wheelchair will move. It will lose speed automatically after the joystick is released.
- **Turn on/off button (7 A)**
- **Speed adjustment buttons (7 B/C, press)** - press +/- to increase or decrease the speed.
- **Speed levels indicator (6):** 1 – slowest, 5 – fastest.
- **Horn button (7 D, hold)** – hold button to use the horn.
- **Flashlight (7 D, press)** – press button to turn on/off the flashlight located under the control module panel.
- **Battery level indicator (1)**
- **Voice assistant off/on (7 C, hold)** – hold button for more than 5 seconds to turn on/off the voice assistant.
- **Warning function (7 B, hold)** – hold button to activate warning function. To turn it off, press any other button.

B. MANUAL AND ELECTRIC MODE SWITCHING

To use the wheelchair in electric mode, move down the left and right engine levers located at the back. To use it in manual mode, move the levers up.

C. CHARGER

- Set includes a charger dedicated to the battery.
- Battery charging time is around 6-8 hours.
- Plug the charger into the charging port located in the control module (remove flashlight module before charging) or on the battery.
- Connect the charger to power supply.
- Check the connection.

- To avoid the risk of fire, do not modify or disconnect the charging circuit while charging the battery.

D. DIRECTIONS FOR USE

- Select the desired driving mode.

Note! Do not switch the mode while driving on elevation.

- Push the power button to check the proper functioning of brakes. Brakes are working correctly (electric mode) if you cannot push the wheelchair. Otherwise, contact the Seller.

- Tilt the footrest and sit comfortably on the seat. Lower the footrest back to horizontal position.

Note! Do not use the footrest as a support when sitting down or standing up from the wheelchair. This may cause the wheelchair to tilt and fall.

- Sit on the wheelchair and turn the control panel on. The LCD panel will light up. Joystick should be in neutral position.
- Take caution when using for the first time. Control the speed and direction carefully. The electromagnetic brake will get released only after pushing the joystick (electric mode). To speed up, push the joystick further away from its neutral position. To slow down – do the opposite.
- To stop the wheelchair, release the joystick. Pressing the power button will also stop the wheelchair.
- Using dedicated buttons on the control panel, adjust the wheelchair’s speed according to user’s physical state and road conditions.
- Wheelchair should only be used on flat surfaces. Driving on muddy or uneven surfaces can cause control system damage.

6. Technical data

Product code	SMART-TIM
Overall height	88 cm
Overall width	54 cm
Overall length	82 cm
Seat height from the ground	44 cm
Backrest height	46 cm
Seat cushion dimensions	41 x 42 cm

Battery weight	2,30 kg
Maximum capacity	115 kg
Wheelchair weight	14,70 kg
Wheels	front: solid, swivel 6'' back: solid, 8''

7. Cleaning and maintenance

- It is recommended to regularly inspect the construction and its elements (wheels, frames, cables, control module, brakes etc.).
- Any used or damaged wheelchair parts need to be replaced (in such case, please contact the Seller).

8. Battery maintenance and cleaning:

- 1) Regularly check the battery level displayed. If the level is very low, charge it immediately to avoid a voltage drop (which may affect battery life).
- 2) If wheelchair won't be used for longer period of time, charge battery at least once a month.
- 3) Battery has a certain lifespan. Exchange your battery if it wears out.
- 4) Battery should be kept clean and away from kids. Avoid causing any shock or damage to the battery.
- 5) Charge it regularly to expand its lifespan.
- 6) Used batteries should be utilized according to local environmental regulations.

9. Storage and transport conditions

A. TRANSPORTATION

- Take caution during transportation. Do not throw, turn over or put pressure on the product and its packaging.
- For wheelchair folding, check „Assembly and adjustment”.

B. STORAGE

Wheelchair should be kept in dry and well-ventilated space. Avoid: high temperature and its rapid changes, acids, alkalis and corrosive chemicals.

C. STORING CONDITIONS

Temperature: -40[°C] ~+55[°C]

Humidity: ≤80%

Air pressure: 86 ~ 106 kPA

10. Troubleshooting

- In the event of a malfunction, the control module will indicate it with voice notification, sound and warning appearing on the screen. The number displayed on the screen will help to identify the problem.

Issue indicator	Cause	Possible solution
<p>Module doesn't turn on after pressing power button</p>	<p>A. Battery: isn't connected to power cabel / isn't charged / power button is off / is damaged. B. The power button is damaged / control module is damaged.</p>	<p>A. Check if battery: is charged / power button is on / power cabel is correctly mounted / isn't damaged. B. Contact the seller to replace damaged parts.</p>
<p>After turning on, the red warning appears on screen, with ring alarm and voice notification.</p> 	<p>The battery level is low.</p>	<p>Charge the battery for minimum 30 minutes before turning it on.</p>
<p>After turning on, the red warning appears on screen, with ring alarm and voice notification: left brake fault.</p> 	<p>Left brake is in manual mode / is damaged.</p>	<p>A. Switch left brake to electric mode. B. Check if engines or power cords are damaged. If yes – contact seller. C. Pull off left brake connecting cabel and connect to right brake connector. Do the same for right brake connector. If problem still appears, then it's related ti the controller. Contact seller for more information.</p>

<p>After turning on, the red warning appears on screen, with ring alarm and voice notification: right brake fault.</p>  <p>E5</p>	<p>Right brake is on manual mode / is damaged.</p>	<p>A. Switch right brake to electric mode. B. Check if engines or power cords are damaged. If yes – contact seller. C. Pull off right brake connecting cable and connect to right brake connector. Do the same for right brake connector. If problem still appears then it's related to the controller. Contact seller for more information.</p>
<p>After turning on, the red warning appears on screen, with ring alarm and voice notification.</p>  <p>EA/E9</p>	<p>Right/Left-side engine cable connector fell off or is damaged.</p>	<p>A. Check if the right/left connector cable is loose. B. If the error still occurs despite of both cables being properly mounted, change the right side cable connection with left side. If the sound notification still appears 9/10 times, it's a problem with control panel. Turn on the controller. If the notification still appears, the source of problem is in left (10 times – EA)/right (9 times – E9) engine. Contact seller for more information.</p>
<p>After turning on, the red warning appears on screen, with ring alarm and voice notification.</p>  <p>E6</p>	<p>Problem with signal transmission between control module and control box.</p>	<p>Contact the seller to exchange control module and controller box.</p>
<p>After turning on, the red warning appears on screen, with ring alarm and voice notification.</p>  <p>E8</p>	<p>Control module issue</p>	<p>Contact seller to exchange control module (or both control module and controller box in case of more severe issue).</p>

11. Warranty information

- All products distributed by our company are covered by a warranty, the terms of which are described in the warranty card available on our website. Please contact the retailer from whom you purchased the product. Please, keep in mind that for warranty purposes, the proof of purchase (receipt or invoice) should be retained. As a user-friendly company, we provide only tested products – in terms of material, quality and functionality. If you have any questions regarding service procedures, please contact us.
- Warranty does not cover:
 - wearable parts: upholstery, tires, armrests, battery
 - damages resulting from: self-repairs, changes in product construction, dropping, transportation, inappropriate usage, failure to follow instructions, natural disasters (fire, earthquake, flood).

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Xuzhou Heabens Medical Technology Co., Ltd.
SUNGO Europe B.V.