INSTRUCTIONS FOR USE Aluminium 4-wheeled rollator Product code: TGR-R RA 882

1. Characteristics


Folding 4-wheeled rollator is made of aluminium frame which is coated with paint with metallic lustre. The rollator possesses wide upholstered seat, soft backrest and removable basket. The product is equipped with height adjustable handles with anatomic handgrips and brakes with locking mechanism. The rollator is supplied with wheels with narrow cross section 8" in diameter.

The set includes:

- Two handles with brakes
- Frame with seat
- Backrest
- Basket
- Locking knobs 2 pcs

Elements of the rollator:

- a. Push handle
- **b.** Brake
- c. Locking knob
- d. Brake cables
- e. Backrest
- f. Locking pin
- g. Side crossbar
- h. Hinge of side crossbar
- i. Strap for folding rollator
- j. Basket
- k. Upper adjustment of brake
- I. Lower adjustment of brake
- m. Cane holder

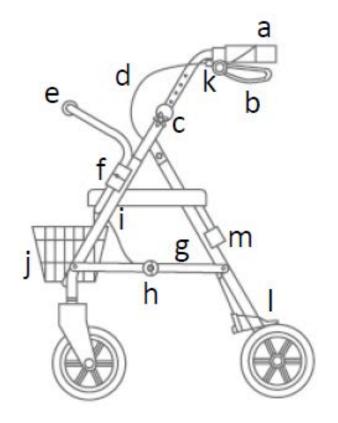

Timago

2. Application

The rollator is designed for patients with reduced mobility who need stable support and help while walking. It is also recommended to the elderly as an aid in keeping balance, improving walking comfort and ensuring safety from falling. The use of the rollator affects positively on the return to activity of the people with locomotion and tilting to erect position problems. It guarantees better stabilization, support and security while walking.

3. Assembly

Unfolding and folding the rollator

- To unfold the rollator, press the side crossbar, which is placed under the seat, until the hinges are completely unfolded. Then, fix the handles into the mounting openings and select appropriate height by tightening the locking knobs. Brake cables should be placed at the front. Lean the seat against unfolded frame.
- Fix the backrest into the mounting openings at the front of the rollator until the pins are locked.
- The basket can be hanged on the frame or fixed under the seat.
- Remove plastic casing from wheels.
- To fold the rollator for transport, lower the seat and pull upwards the strap which joins the side crossbar with frame until the frame is completely folded.

NOTE! Be careful while folding or dismantling the rollator due to the risk of smashing fingers in hinges.

Brakes

- To stop the rollator, pull the brake handle upwards and hold.
- While stop or using the seat, lock the brake with locking mechanism by pushing down the brake handle.
- To release the brake, pull the brake handle upwards.

Height adjustment of handles

• To adjust the height of the handles, unscrew the locking knobs which are located on the outer side of the frame, select appropriate height and tighten the locking knobs.

Note! The handles must be set on the same level.

Adjustment of brakes Adjustment of upper brake

Unscrew the nut of the cable, pull it up to the appropriate length and lock it by tightening the locking screw.

Adjustment of lower brake

Loosen the mounting screw with the use of open-end wrench, pull the black brake cable to adjust the tension of the cable, and then, tighten the mounting screw.

Overall width	60 cm
Overall depth	66 cm
Seat dimensions	35 cm x 32 cm
Seat height from the ground	53 cm
Handles height adjustment	77-88 cm
Weight	8.2 kg
Maximum capacity	110 kg
Maximum capacity of the basket	5 kg
Frame colours	red, blue, titanium

4. Technical data

5. Notes

Timago International Group does not bear responsibility of improper use of the rollator, failure to comply safety regulations and misuse.

6. Maintenance

Cleaning

- Clean the frame with damp cloth and dry thoroughly.
- Do not use strong polishing agents due to the damage of protecting coatings.
- Clean the dirt from moving parts of the rollator regularly (hinges, clamping screws, locks).

7. Storage and transport conditions

Device should not be exposed to direct sunlight, water or moisture.

8. Warranty information

All products distributed by our company are covered by a warranty, the terms of which are described in the warranty card available on our website. Please contact the retailer from whom you purchased the product. Please, keep in mind that for warranty purposes, the proof of purchase (receipt or invoice) should be retained

9. Labels

REF	Reference number	•••	Manufacturer
LOT	LOT number	$\sim \sim$	Manufacture date
SN	Serial number	MD	Medical device
\triangle	Note	i	Please read the instruction
CE	The manufacturer checked for compliance with essential medical devices requirements.		

Our complete line of products is available from our distributors. Learn more at: www.timago.com. Thank you for choosing Timago!



Timago International Group Spółka z o.o. i Spółka – Spółka komandytowa ul. Karpacka 24/12 43-316 Bielsko-Biała, Polska

T.: +48 33 499 50 00 F.: +48 33 499 50 11 E.: info@timago.com



Rev. II, date 20.05.2021