REF TGR-R KP 340L

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BRUGSANVISNING

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Product code: TGR-R KP 340L

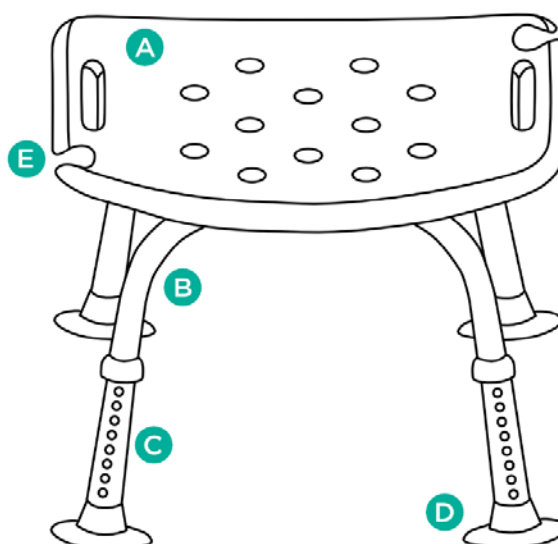
1. Characteristics

Shower chair consists of aluminium frame and wide seat made of durable PVC. The chair has a height adjustable legs, handles for the shower head and side handles which facilitate using the product. Specially designed anti-slip rubber tips prevent the chair from sliding on wet surfaces. The chair is additionally supplied with Tool-Free (TF) system facilitating the assembly.

The set includes: seat, chair base (2 pcs), height adjustable legs (4 pcs), mounting screws (5 pcs).

Elements of the chair:

- A. seat
- B. base tubes
- C. height adjustment
- D. anti-sliding tip
- E. shower head handle

2. Application

The shower chair is ancillary equipment which helps the user to perform hygienic activities. The chair is recommended for people with reduced mobility which prevents or hinders the unassisted performance of daily living activities. The chair can be used in the shower, in the bath, in bathrooms and other humid rooms at home and in hospitals.

3. Assembly

Step 1. Place seat upside down on a flat surface.

Step 2. Put tubes that creates base of the shower chair in designated places on seat.

Note! Ensure that base tubes hollows are facing each other (picture 1).

Step 3. Fix crossed base tubes with seat with mounting screw in the middle.

Step 4. Fix rest of the mounting screws in mounting openings. To assure safe and stable connection between base and seat, use specially profiled metal pads.

Step 5. Check if all mounting screws are correctly fixed.

Note! Regularly check tightness of the screws.

HEIGHT ADJUSTMENT

- To adjust the height of the shower chair, press metal pin, select appropriate height of leg, and lock the position of the tubes relative to each other with the use of metal pin. The legs are adjustable in 2.5 cm increments.

Note! The pins must be set on the same level in all legs.

4. Technical data

Width	48 cm
Depth	41 cm
Height	38-57 cm
Seat dimensions	51 x 31 cm
Weight	1,80 kg
Maximum capacity	136 kg

5. Notes

Timago International Group does not bear responsibility of improper use of the shower chair, failure to comply safety regulations and misuse.

6. Maintenance

- Clean the shower chair regularly using only water, sponge and mild detergents for plastic materials such as soap. Dry thoroughly.

- **Do not use** for cleaning: strong detergents, abrasive agents, bleach, petrol or solvents (it may cause frame damage, visible defects and in consequence - loss of warranty rights).
- **At least once every three months check:** tightening of mounting screws, condition of locking pins, condition of tubes (if cracks).

It is recommended to check the rubber tips regularly. In case of worn-out, replace the tips with the new ones.

7. Storage and transport conditions

Device should not be exposed to direct sunlight, water or moisture.

8. Warranty information

All products distributed by our company are covered by a warranty, the terms of which are described in the warranty card available on our website. Please contact the retailer from whom you purchased the product. Please, keep in mind that for warranty purposes, the proof of purchase (receipt or invoice) should be retained

9. Labels

	Reference number		Manufacturer
	LOT number		Manufacture date
	Serial number		Medical device
	Note		Please read the instruction
	The manufacturer checked for compliance with essential medical devices requirements.		

Our complete line of products is available from our distributors.

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