EDITION 2024/11

MADE IN ITALY

# KING SHOWER SEATS

WITH/WITHOUT BACKREST & ARMRESTS

EN USER MANUAL  
 DE BENUTZERHANDBUCH  
 FR MANUEL DE L'UTILISATEUR  
 ES MANUAL DE USUARIO  
 IT MANUALE D'USO  
 PT MANUAL DO USUÁRIO  
 SL NAVODILA ZA UPORABO  
 HR KORISNIČKI PRIRUČNIK  
 HU HASZNÁLATI UTASÍTÁS  
 RO MANUAL DE UTILIZARE

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## INTENDED USAGE:

The products of the *Shower Seats KING* line allow the user an increase in independence during the shower stage of daily personal hygiene, reducing muscle fatigue due to standing position as well as slip and injury risks. The non-slip rubber feet ensure grip with the floor, and the height adjustable legs allow users to change the total height according to their needs easily. In addition, the shower seat with customised height can be used with different length settings between the front and rear legs. In particular, the front legs can be lower than the rear ones: this allows the user to keep a more correct posture while taking a shower, as well as to facilitate standing and sitting procedures. Due to its flexibility, the shower seat can be positioned even on a bumpy and irregular floor, like those typical of many shower trays. The versions provided with backrest and/or armrests give additional safety and comfort. The shower seat can be easily moved due to its lightweight. The seat can be completed with a soft cushion and a soft cushion for backrest to make the use more comfortable. The soft cushions, armrests, and backrest are available separately as accessories.

## INDICATIONS AND CONTRAINDICATIONS:

The Shower Seats KING line products are aimed at elderly people, wheelchair users, people with reduced mobility of the lumbar, knee, or hip, and all those who have difficulty maintaining the standing position during the shower stages of daily personal hygiene. The user is required to have autonomy of balance during sitting and standing procedures as well as to have adequate ability to keep sitting. The shower seat must not be used if the user has problems related to balance or perception disorders.

## SAFETY WARNING AND INDICATIONS:

- The maximum user weight indicated in the fitting instructions must not be exceeded.
- Before every use, be sure the shower seat is correctly assembled in a stable position.
- The shower seat can be used only on a surface free from obstacles.
- During assembly, strictly adhere to what is shown in the User Manual.
- The risk of finger injuries may increase during the mounting process.
- Do not swing on the shower seat; this may lead to slipping, overturning, or subsidence of the legs. Always keep the shower seat on its 4 legs.
- Do not modify the shower seat and its components in any way.
- Do not use the shower seat if damaged or altered in any way.
- After use, the shower seat should be removed from the tray to avoid obstruction.
- Do not use the shower seat for different purposes from the intended one.
- The shower seat must not be stored outdoors.
- Avoid direct contact with open wound or broken skin to prevent infection.

## MAINTENANCE INFO:

Product maintenance consists of cleaning and sanitation procedures. In order to guarantee safe use and an adequate standard of hygiene, the user should perform these procedures before every use. The user has to make certain of the structural integrity of the device and its components, and the tightness of the screws must be checked. For cleaning and sanitation procedures, follow the steps below:

1. Gloves should be worn.
2. Remove the evident dirt before proceeding with the most accurate disinfection.
3. Use water or non-aggressive detergent for the cleaning procedure.
4. Wipe with a dry and clean cloth.

Alternatively, the product can be cleaned in domestic dishwashers at the maximum temperature. The soft cushion must be cleaned strictly by hand.

## REUSE REQUIREMENTS:

The product is suitable for reuse by other users. Before reusing by others, the product must be correctly cleaned and sanitized. Before the transfer of the product to others, the product must be checked by qualified dealers.

## CORRECT DISPOSAL PROCEDURE:

Do not dispose of the product in the regular household waste. Ask your local authority beforehand for the correct and environmentally friendly way to dispose of it.

## INCIDENT REPORTING:

If there is a serious incident involving this device, contact the manufacturer of this product at [info@kingbathrooms.com](mailto:info@kingbathrooms.com) and the responsible authority in your country.

ITEM CODE	DESCRIPTION	PAGE
<b>KING-SS1-KN</b>	Shower seat King	<b>44-45</b>
<b>KING-SS1A-KN</b>	Shower seat with armrests King	<b>44-45</b>
<b>KING-SS1B-KN</b>	Shower seat with plastic backrest King	<b>46-47</b>
<b>KING-SS1BA-KN</b>	Shower seat with plastic backrest and armrests King	<b>46-47</b>
<b>KING-BR1-KN</b>	Backrest for seats, chairs, and commodes King	<b>48</b>
<b>KING-A-KN</b>	Armrests for shower seat and chair King	<b>49</b>
<b>KING-BSS1-KN</b>	Black shower seat King	<b>44-45</b>
<b>KING-BSS1A-KN</b>	Black shower seat with armrests King	<b>44-45</b>
<b>KING-BSS1B-KN</b>	Black shower seat with plastic backrest King	<b>46-47</b>
<b>KING-BSS1BA-KN</b>	Black shower seat with plastic backrest and armrests King	<b>46-47</b>
<b>KING-BBR1-KN</b>	Black backrest for seats, chairs, and commodes King	<b>48</b>
<b>KING-BA-KN</b>	Black armrests for shower seat and chair King	<b>49</b>
<b>KING-CSS2/4-KN</b>	Soft cushion for shower seat light blue/grey King	<b>50</b>
<b>KING-CBR2/4-KN</b>	Soft cushion for backrest light blue/grey King	<b>51</b>

ICON	DESCRIPTION
	Made in Italy
	Tool Free Assembly
	Dishwasher Friendly
	Max User Weight
	5 Years Warranty
	Scan the QR CODE to watch the assembly video

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LM Global Design Ltd.