INSTRUCTIONS FOR USE

WHEELCHAIR CUSHION

Product code: **SitWELL**

WHEELCHAIR CUSHION

Product code: **ErgoTIM**

Before using the product, read these instructions. The instruction contains information necessary for the safe and proper use of the product, how to maintain and clean it. Use of the product not in accordance with the rules of the instructions for use can risk serious complications. The instruction should be kept in an easily accessible place

1. Characteristics

The cushion is made of polyurethane foam and comes with a zippered waterproof, vapor-permeable cover.

2. Application

The cushion is recommended for use in a wheelchair, armchair, chair, or car. It provides comfort, even weight distribution, and free airflow, helping to prevent pressure sores, hemorrhoids, and varicose veins. Ideal for everyday use and travel.

3. Contraindications for use:

- Do not use the device if it is not compatible with the wheelchair.
- Do not use the product other than its intended use.
- Do not use the product near an open flame or on a hot surface. It may seriously injure the user and lead to serious health damage.

4. Notes

Manufacturer does not bear responsibility of improper maintenance of the product, failure to comply safety regulations and misuse.

- The skin at the site of connection should be clean.
- It is necessary to regularly check the condition of the patient's skin.

- The position of users should be changed regularly to avoid the formation of sores/wounds.
- Do not use sharp objects and equipment that can damage the cover.
- The cushion should be matched to the seat size. Maintain high hygiene of the cushion cover: clean and/or wash.

If the product is no longer suitable for use, it should be disposed of. Before disposal, the product should be cleaned to eliminate the risk of contamination.

If there are any adverse symptoms resulting from the use of the device, report it to the manufacturer and to the competent authority of the Member State where the user or patient resides.

5. Unpacking and fitting

- When you receive the shipment with the cushion, open it carefully so as not to damage the cushion.
- Lay the cushion on a flat surface and wait for all the deformations caused during transportation to return to its original shape.
- The cushion should be fitted to the patient's seat to fully perform its functions.

6. Technical data

PARAMETERS	MODEL	
	SitWELL	ErgoTIM
Length	40 cm	40 cm
Width	43/46 cm	46 cm
Height	5 cm	7 cm
Max. weight capacity	120 kg	120 kg
Construction	perforated surface	“hedgehog”- type surface

7. Cleaning and maintenance

- Do not wash the foam insert.
- Wash the cover after removing the foam insert in a mild detergent at temperatures up to 90°C.

- The cover can be sterilized at temperatures up to 120°C.
- The cover can be disinfected with up to 60% alcohol solution.
- Do not use bleach or chemicals.
- Do not chemically clean.
- Do not use chemicals containing chlorine.
- Drying in an unfolded state is recommended.
- Do not allow heavy soiling.

8. Storage and transport conditions.

Product should not be exposed to direct sunlight, water or moisture.

9. Warranty information

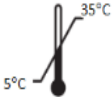
All products distributed by our company are covered by a warranty, the terms of which are described in the warranty card available on our website. Please contact the retailer from whom you purchased the product. Please, keep in mind that for warranty purposes, the proof of purchase (receipt or invoice) should be retained.

COMPLAINTS ARE NOT SUBJECT TO:

- Mechanical damage (abrasions, scuffs).
- Damage resulting from natural use.
- Defects resulting from improper sizing of the cushion for the seat
- Defects resulting from failure to observe the conditions of use or the method of cleaning and maintenance.
- Discoloration of the cover.
- Comfort of use.

10. Labels

	Reference number		Manufacturer
	LOT number		Unique device identifications
	Serial number		Medical device
	Note		Please read the instruction

	Use by date		Permissible storage temperature
	Store in a dry place		Protect from sunlight
	Packaging is recyclable		The packaging should be disposed of in the trash
	The manufacturer checked for compliance with essential medical devices requirements.		

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