REF

SUPPORT TIM COOLING TIM

Timago
International Group

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BRUGSANVISNING

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Anti-decubitus ring cushion

Product code: SUPPORT TIM

Orthopedic seat cushion with cooling gel

Product code: COOLING TIM

1. Characteristics

SUPPORT TIM

Thanks to its rounded design and the use of memory foam, the cushion provides comfort while sitting. It supports and relieves pressure on the spine and pelvis, reducing the risk of bedsores. The pillow is equipped with a high-quality soft cover with a zipper.

COOLING TIM

Thanks to its contoured design and the use of memory foam, the cushion provides comfort and additional support for the spine while sitting. The cooling gel insert helps dissipate heat effectively and regulate the temperature of the body. The pillow is equipped with a high-quality, breathable cover with a zipper.

2. Application and recommendations

SUPPORT TIM

The anti-decubitus ring cushion is a product for patients exposed to the risk of bedsores. It is used by people remaining in the sitting position for a prolonged period of time, with circulatory disorders in systemic cardiovascular system. Before use, consult with your doctor or physical therapist.

COOLING TIM

The orthopedic seat cushion with cooling gel is product for patients suffering from problems with osteoarticular system and spine. It is used by people remaining in the sitting position for a prolonged period of time, with circulatory disorders and pain in the sacrum of the spine. Additional cooling gel insert helps dissipate heat and regulate the temperature. Avoid direct contact with bare skin.

3. Contraindications

Users with serious medical conditions or changes of the cervical spine should consult a physician or physiotherapist prior to first use of the pillow.

4. Notes

Timago International Group does not bear responsibility of improper use of the product, failure to comply safety regulations and misuse.

5. Technical data

Model	SUPPORT TIM	COOLING TIM
Dimensions	39 x 39 x 8 cm	47 x 37 x 8 cm
Composition parts:	Cover: Polyester 95% Elastane 5% Filling: Thermoelastic foam	Cover: Polyester 100% Filling: Thermoelastic foam Cooling gel insert
Cover	zipper	breathable, zipper
Colour	grey	black and blue

6. Maintenance

- Wash the pillowcase by hand in lukewarm water, do not spin, do not chlorinate, do not iron, do not bleach, do not tumble dry, do not dry clean.
- Dry at room temperature away from intense heat sources.

NOTE! Maintenance information can be found on the fabric label.

7. Storage and transport conditions

Device should not be exposed to direct sunlight, water or moisture.

8. Warranty information

All products distributed by our company are under a warranty, the terms of which are described in the warranty card received at the time of purchase.

Please, keep in mind that for warranty purposes, the proof of purchase (receipt or invoice) should be retained.

9. Labels

	Reference number		Manufacturer
---	------------------	---	--------------

	LOT number		Manufacture date
	Serial number		Medical device
	Note		Please read the instruction
	The manufacturer checked for compliance with essential medical devices requirements.		

Our complete line of products is available from our distributors.

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