REF **COMPLEX-TIM**

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BRUGSANVISNING

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USER MANUAL

Shower and commode aluminium wheelchair

Product code: COMPLEX-TIM

1. Characteristics

Shower and commode wheelchair is made of powder coated aluminum frame. It features flip-up armrests for easy entry and exit for people with reduced mobility. It is equipped with a seat with wedge made of PU, front swivel 5" wheels with brakes, back 24" solid wheels and quick-release, tiltable and removable footrests, safety straps, handle for the accompanying person and a sanitary container. The construction of the wheelchair makes it possible to slide over the toilet seat.

The set includes: wheelchair frame with wheels, seat with wedge, two mounting knobs, right and left footrest with safety straps, sanitary container with lid.

Elements of the chair:

1. Handle
2. Backrest
3. Footrest latch
4. Seat with wedge
5. Sanitary container
6. Rear wheels 24" with quick-release
7. Armrests
8. Footrest with safety straps
9. Front wheels 5" with brakes
10. Parking brakes

2. Application


Shower and commode wheelchairs are designed for people who have problems with unassisted movement and for patients with dysfunctions and reduced mobility. The wheelchair is suitable for indoor use only. The wheelchair functions as a shower and commode chair owing to the sanitary container located under the seat and moisture-resistant aluminium frame. It facilitates everyday hygiene activities. The wheelchair is especially recommended to people who have problems with mobility which are caused by: paralysis, obesity or abnormal underweight, degenerations or deformity of limbs, degenerations or joints injuries, limbs injuries or amputations, heart and blood vessel disorders and circulatory disorders.

3. Assembly

9. To prevent wheelchair frame from moving during assembly, lock the brakes by pressing red brake levers down (picture 1).

10. Mount rear wheels by inserting them into holes on the back of the frame.

11. Slide the parking brake onto the tube at the front of the frame, then tighten with a hex key. A correctly assembled brake should be positioned 4 cm from the end of the tube (picture 2). Repeat it on the other side. Check the correct functioning by pulling the parking brake lever.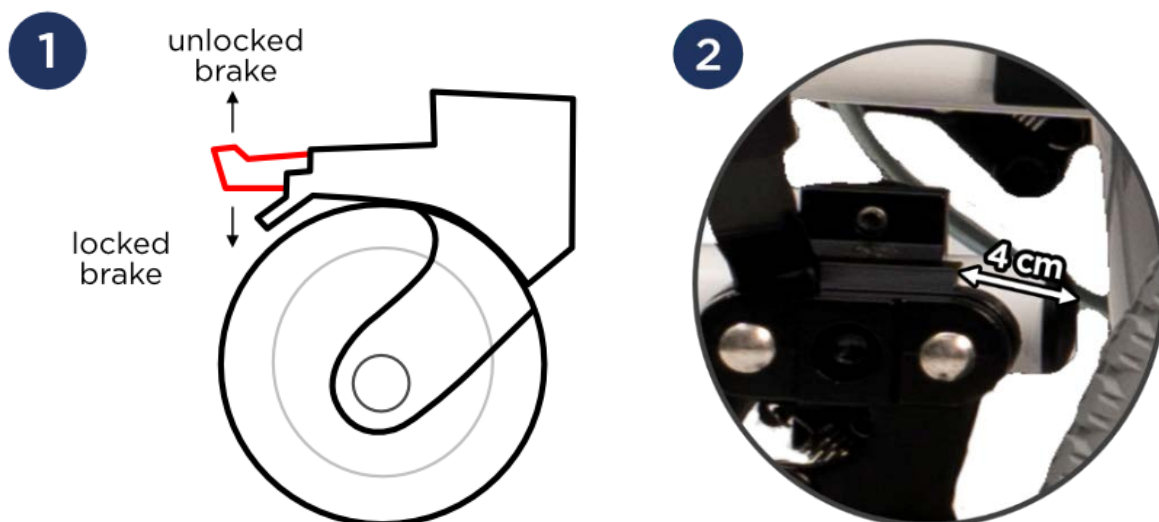

12. Unscrew mounting knobs from the back frame and put backrest tubes into wheelchair frame. Tighten it with knobs and lower the armrests.

13. Fix the seat on the frame of the chair by pressing it until it locks into place (the front of the seat is indicated by the wedge).

14. Fix the sanitary container by sliding it from the back of the seat.
15. Insert the footrests into the main frame from the side, and then rotate them to the front until the footrest latches lock in place. To remove the footrests pull up the latch lever and rotate the footrests outwards.
16. Fix the calves safety straps around the footrests.

WHEELCHAIR HANDLING

- Check the condition of the brakes, construction and castors. Check the proper assembly of the components and their adjustment.
- Before using, make sure that the chair's width matches the user, as the chair is not equipped with width adjustment.
- The chair is designed to be used for short distances and inside only. The chair doesn't have a shock absorber, so long-term use is not recommended.

NOTE! Do not use the chair if any defects or deficiencies are noticed.


FLIP-UP FUNCTION OF ARMRESTS

- Lift the armrest upwards with a gentle movement.
- **Do not** use the armrest as a support while standing up. If help is required – ask for assistance.

BRAKES

- To lock the front castors, press the red pedals with your foot. To unlock the wheel, press the black levers inside the red pedals with your foot.
- To lock the rear wheel, pull the parking brake lever backwards. To unlock the wheel, pull the brake lever in the opposite direction.

FOOTRESTS

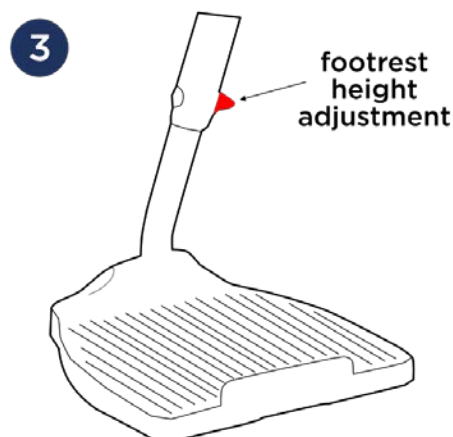- The footplates should be in vertical position when getting in or out of the chair. **Do not use the footplates as a support while getting in or out of the chair due to the risk of falling over.**
- Footrests have height adjustment. Untighten the mounting screw on back of the footrests (picture 3) to adjust height. Tighten it after.

GETTING ONTO THE CHAIR

- It is essential to lock the brakes while getting in or out of the chair, changing the user's position or getting off.
- While changing place, the chair should be located as close as possible to the place of transfer. The chair should touch the bed or other seat being used. If the user cannot change his/her seat independently, transfer the user to the chair.
- The user should sit as deep as possible in the middle of the seat in order to avoid chair instability.

4. Technical data

Overall width	63,5 cm
Inner width	45 cm
Overall depth	102 cm
Seat height from the ground	53 cm
Overall height	93 cm
Seat dimensions	44 x 42 cm
Weight	17 kg
Maximum capacity	150 kg
Frame color	white

5. Notes

Timago International Group does not bear responsibility for improper use of the chair and failure to comply with safety regulations.

6. Cleaning and maintenance

Carry out the maintenance of the chair components regularly:

- The seat can be cleaned with a damp cloth and cold or warm water, without the use of detergents. Difficult stains can be cleaned with a sponge.
- Plastic elements should be cleaned with damp cloth with the use of appropriate detergents for plastic elements. In such case follow the instructions given by the producer and safety measures.
- Empty and disinfect the toilet bucket and toilet seat after each use. Use cleaning products according to producer's recommendations.

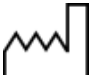
7. Storage and transport conditions

The product should not be exposed to direct sunlight.

8. Warranty information

All products distributed by our company are covered by a warranty, the terms of which are described in the warranty card available on our website. Please contact the retailer from whom you purchased the product. Please, keep in mind that, for warranty purposes, the proof of purchase (receipt or invoice) should be retained.

9. Labels

	Reference number		Manufacturer
	LOT number		Manufacture date
	Serial number		Medical device
	Note		Please read the instruction
	The manufacturer checked for compliance with essential medical devices requirements.		

Our complete line of products is available from our distributors.

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