INSTRUCTIONS FOR USE

Back rest with armrests

Product code: BackTim

1. Characteristics

The back rest is made of powder-coated steel frame. It consists of a base with stepped backrest angle adjustment, a seat and folding armrests. The backrest is made of durable and breathable fabric with an additional head cushion for added comfort. Adjustable angle allows for individual adjustment to the patient's needs while lying or sitting.

Elements of the back rest:

- a. backrest angle adjustment
- b. backrest
- c. head pillow
- d. armrests
- e. seat

2. Application

Back rest is ancillary equipment designed for patients with reduced mobility and who are in lying position permanently which prevents or hinders the unassisted performance of daily living activities. Back rest helps to maintain half-sitting position without having to be hold by the other person. It is also used in order to tilt the patient to erect position by gradually accustoming the cardiovascular system to sitting position. The back rest is recommended for use at home and in hospitals.

3. Adjustment

The backrest angle of the arm rest is adjustable in steps from 47° - 70°. The armrests have a folding / unfolding function.

4. Technical data

Overall width	55 cm
Inner width	50 cm
Overall depth	50 cm
Inclination angle adjustment	47° - 70°
Weight	3,78 kg
Maximum capacity	136 kg
Frame colour	white

5. Notes

Timago International Group does not bear responsibility of improper use of the back rest, failure to comply safety regulations and misuse.

6. Cleaning and maintenance

Reviews

Make visual inspection of all the joining elements (clamps, hinges) of the back rest at least once every three months including:

- condition of ratchet mechanism
- stability and look of the frame

Cleaning

Control the technical condition of the back rest regularly:

- Clean the frame using only water and mild detergents. Dry thoroughly.
- **Do not use** for cleaning: strong polishing agents and detergents, abrasive agents, bleach, petrol or solvents (it may cause frame damage, visible defects and in consequence - loss of warranty rights).

7. Storage and transport conditions

Device should not be exposed to direct sunlight, water or moisture.

8. Warranty information

All products distributed by our company are covered by a warranty, the terms of which are described in the warranty card available on our website. Please contact the retailer from whom you purchased the product. Please, keep in mind that for warranty purposes, the proof of purchase (receipt or invoice) should be retained

9. Labels

	Reference number		Manufacturer
	LOT number		Manufacture date
	Serial number		Medical device
	Note		Please read the instruction
	The manufacturer checked for compliance with essential medical devices requirements.		

Our complete line of products is available from our distributors. Learn more at: www.timago.com. Thank you for choosing Timago!

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