REF PEDRO

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BRUGSANVISNING

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USER MANUAL

Indoor aluminium 4-wheeled rollator

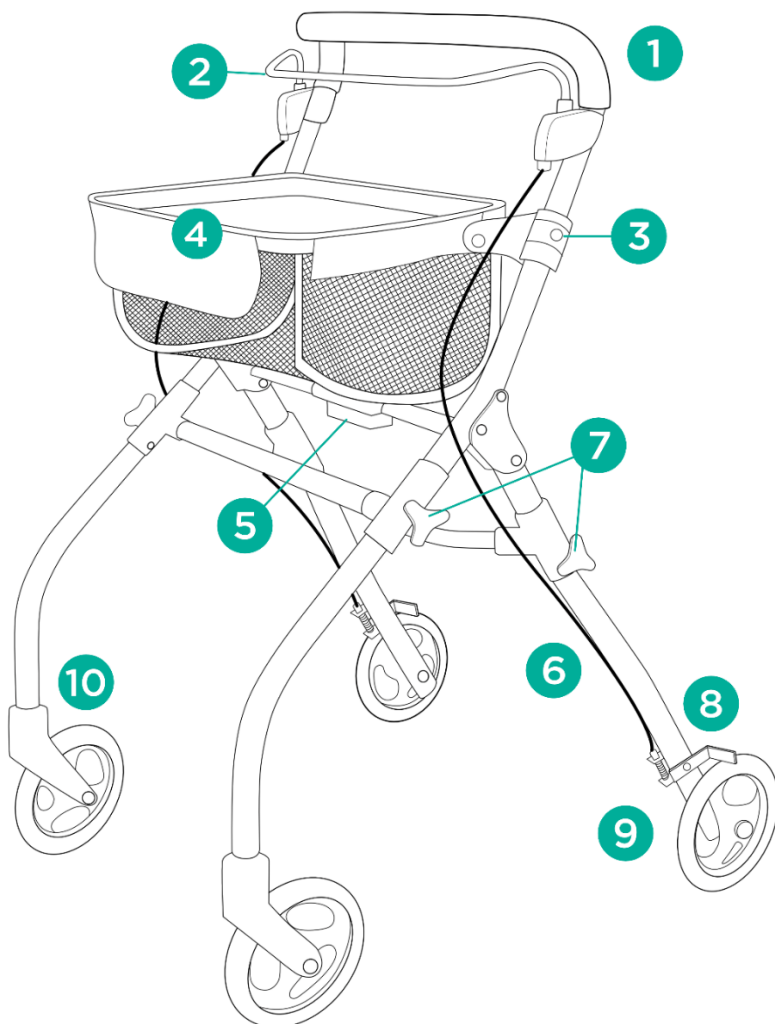
Product code: PEDRO

1. Characteristics

Indoor 4-wheeled rollator is made of anodized aluminium frame. The rollator is equipped with brakes with parking function and tray with bag. The product has solid 8" rear wheels with brakes, solid swivel 8" front wheels and height adjustable handle with a soft grip.

The set includes: frame with bag, front and back legs with wheels, tray, locking knobs (4 pcs).

Elements of the rollator:

1. handle
2. brake handle with parking function
3. adjustment clip
4. bag with tray
5. folding mechanism
6. brake cable
7. locking knobs
8. brake
9. rear 5" wheel
10. front 5" swivel wheel

2. Application

The rollator is designed for patients with reduced mobility who need stable support and help while walking. It is also recommended to the elderly as an aid in keeping balance, improving walking comfort and ensuring safety from falling. The use of the rollator has a positive effect on the return to activity of people with locomotion and tilting to erect position problems. It guarantees better

stabilization, support and security while walking. Rollator is designed for indoor use.

3. Assembly

Unfolding the rollator

- Put front and back legs into openings on the frame until you hear a click sound and secure them with locking knobs on the sides.
- Unfold the frame and bag. Put the tray on top of the bag.

Folding the rollator

- Remove the tray, the bag slightly to the back and then fold downwards.
- Press the folding mechanism under the bag to fold the rollator.

Brakes function

- To stop the rollator, pull the brake handle upwards and hold.
- Lock the brake by pushing down the brake handle. To release the brake, pull the brake handle upwards.

Height adjustment

- To adjust the height of the handle, open the adjustment clip and select appropriate height. Lock the clip afterwards to secure the height.

Note! The handle must be set on the same level on both sides of the frame.

4. Technical data

Overall width	60 cm
Overall depth	66 cm
Tray dimensions	38 x 29 cm
Handle height adjustment	89-94 cm
Weight	5,6 kg
Max. user weight	115 kg
Max. load of the bag and tray	5 kg
Frame colour	green

5. Notes

Timago International Group does not bear responsibility of improper use of the rollator, failure to comply safety regulations and misuse.

6. Maintenance

Cleaning

- Clean the frame with damp cloth and dry thoroughly.
- Do not use strong polishing agents due to the risk of damage of protecting coatings.
- Clean the dirt from moving parts of the rollator regularly (hinges, clamping screws, locks).

7. Storage and transport conditions

Device should not be exposed to direct sunlight, water or moisture.

8. Warranty information

All products distributed by our company are covered by a warranty, the terms of which are described in the warranty card available on our website. Please contact the retailer from whom you purchased the product. Please, keep in mind that for warranty purposes, the proof of purchase (receipt or invoice) should be retained.

9. Labels

	Reference number		Manufacturer
	LOT number		Manufacture date
	Serial number		Medical device
	Note		Please read the instruction
	The manufacturer checked for compliance with essential medical devices requirements.		

Our complete line of products is available from our distributors.

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