

INSTRUCTIONS FOR USE

Upholstered commode chair

Product code: BLUE-TIM




1. Characteristics

The upholstered commode chair is made of powder-coated steel frame. It has height-adjustable legs finished with non-slip rubber tips. Adjustability allows the product to be tailored to the individual patient's needs, side armrests support stability and safety in use. The chair is equipped with a toilet seat and an easily extendable plastic sanitary container. Soft, upholstered seat with backrest enhances comfort.

Set includes: chair frame with backrest, legs (4 pcs), leg pin locks (4 pcs), soft seat, sanitary container.

The set includes:

- a. chair frame
- b. soft seat
- c. legs
- d. sanitary container
- e. anti-sliding tips

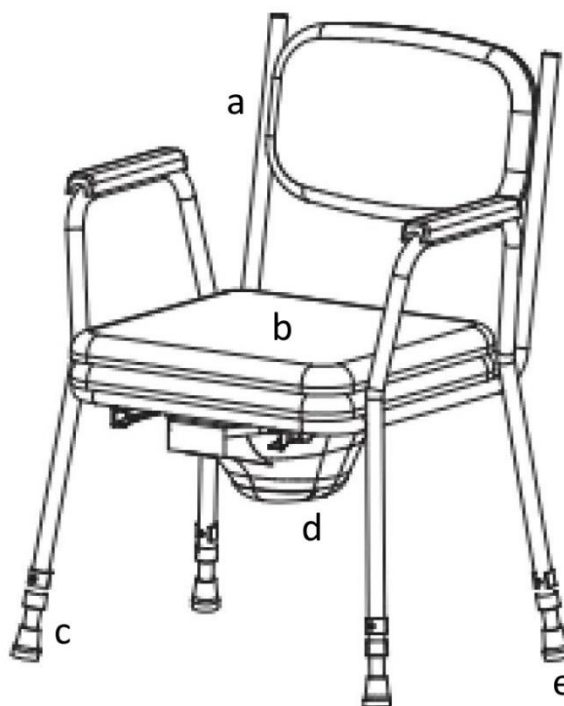

2. Application

The commode chair is ancillary equipment which helps the user to perform hygienic activities. The chair is recommended to people with dysfunctions and reduced mobility who have problems with unassisted walking and for those who lost the ability of unassisted sitting down

and standing up. The chair functions as a toilet owing to the toilet bucket located under the seat. The chair facilitates performing physiological activities and taking care of the user.

3. Assembly, adjustment and usage

Unfolding

Step 1. Turn the chair frame over and insert the four legs into the mounting holes. Lock the legs using pin locks.

Step 2. Turn the chair over and place on the ground.

Note: all legs must be set at the same level. All legs should stand firmly on the ground.

Step 3. Slide the sanitary bin using the guides under the seat.

Step 4. Place the soft seat on the toilet seat.

Height adjustment

To adjust the height of the chair, pull out the pin lock, select the desired leg height, and then lock the set height with the pin lock.

The legs are adjustable in increments of 2.5 cm.

Note: all legs must be set at the same level. All legs should stand firmly on the ground

Usage

Lift the soft seat, and remove the lid of the sanitary container. After use, cover the container with the lid, and place the soft seat on the toilet seat

Note! Remember to empty the toilet sanitary container after each use.

4. Technical data

Overall width	59 cm
Inner width	55 cm
Overall depth	47 cm
Backrest height	42 cm

Seat height from the ground	43-53 cm
Overall height from the ground	81-91 cm
Seat dimensions	43 cm x 46 cm
Chair weight	7.8 kg
Maximum capacity	136 kg
Frame colour	blue

5. Notes

Timago International Group does not bear responsibility of improper use of the commode chair, failure to comply safety regulations and misuse.

6. Maintenance

Carry out regular maintenance of the technical condition of the chair:

- Clean the commode chair regularly using only water and mild detergents. Dry thoroughly.
- **Do not use** for cleaning: strong polishing agents and detergents, abrasive agents bleach, petrol or solvents (it may cause frame damage, visible defects and in consequence - loss of warranty rights).
- Empty and disinfect the sanitary container and the toilet seat after each use.

7. Storage and transport conditions










Device should not be exposed to direct sunlight, water or moisture.

8. Warranty information

All products distributed by our company are covered by a warranty, the terms of which are described in the warranty card available on our website. Please contact the retailer from whom you purchased the

product. Please, keep in mind that for warranty purposes, the proof of purchase (receipt or invoice) should be retained

9. Labels

	Reference number		Manufacturer
	LOT number		Manufacture date
	Serial number		Medical device
	Note		Please read the instruction
	The manufacturer checked for compliance with essential medical devices requirements.		

Our complete line of products is available from our distributors. Learn more at: www.timago.com. Thank you for choosing Timago!



Timago International Group
 Spółka z o.o. i Spółka – Spółka komandytowa
 ul. Karpacka 24/12
 43-316 Bielsko-Biała, Polska

T.: +48 33 499 50 00
 F.: +48 33 499 50 11
 E.: info@timago.com



03/2024(I)