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USER MANUAL

Anti-bedsore cushion

Models: BUBBLE BASIC, BUBBLE
EVERYDAY, BUBBLE EXPERT

1. Introduction

Before using the product check whether the equipment has no transport damage. If any damage is identified, please contact the retail outlet.

Then it is important to refer to the content of this manual that provides relevant information for the user.

If you have any questions concerning the use of the equipment, please contact us. Our personnel will provide you with the required information on an ongoing basis.

We would like to point out at the same time that if you have any questions not related to the product use, please contact your doctor, nurse or physical therapist.

2. Design

Applies to BUBBLE EVERYDAY, BUBBLE EXPERT

The mattress is made of high quality materials. Individual pockets connected into one air system ensures adequate body balance. Pressure exerted to one pocket causes an increase of pressure in adjacent pockets, which allows particular body parts to be relieved. Multi-pocket design and the possibility to adjust hardness by inflating or releasing air provide high comfort of use. In addition BUBBLE EXPERT is equipped with

an extra valve allowing for the cushion's hardness adjustment at any time, depending on individual preferences, and BUBBLE EVERYDAY has an efficient manual pump.

The cushion is manufactured using heat sealing method to ensure its durability and tightness.

It has a protective cover with a zip fastener, made of a mesh material with an anti-slip bottom. A handy holder facilitates manual carrying. The protective cover has additional self-engaging fasteners that allow the cushion to be attached if required.

Applies to BUBBLE BASIC

A comfortable shape and the possibility to adjust hardness by inflating or releasing air provide high comfort of use. The kit includes an efficient pump. High workmanship quality is provided by waterproof and washable medical nylon. The cushion is manufactured using heat sealing method to ensure its durability and tightness. A hidden air valve prevents from accidental air release and improves comfort of use.

The kit contains:

- cushion;
- protective cover (BUBBLE EVERYDAY and BUBBLE EXPERT);
- self-engaging fasteners (BUBBLE EVERYDAY and BUBBLE EXPERT);
- manual pump (BUBBLE BASIC and BUBBLE EVERYDAY).

3. Application and recommendations

The pneumatic anti-bedsore cushion is an auxiliary device

intended for persons exposed to the risk of bedsores. It is used by persons remaining in sitting position for a prolonged period of time with circulatory disorders in systemic cardiovascular system. It is also used in bed sore preventive care and in treatment of bedsores up to the first degree (acc. to NPUAP/EPUAP).

Avoid contact with bare skin. Before use consult with your doctor, rehabilitator or physical therapist.

4. Method of use

1. Take out the product from the packaging.
2. Unfold the cushion to obtain a proper shape.
3. Open the air valve and insert the pump (in BUBBLE BASIC and BUBBLE EVERYDAY) or turn the valve anticlockwise – the cushion will be inflated automatically (BUBBLE EXPERT).
4. Pump to the required hardness.
5. Fit the protective cover (BUBBLE EVERYDAY and BUBBLE EXPERT);

5. End of use

To release air:

- in BUBBLE BASIC and BUBBLE EVERYDAY – open the air valve and squeeze it with your index finger and thumb;
- in BUBBLE EXPERT – turn the valve anticlockwise, sit on the cushion and leave the valve open.

6. Technical data

- Cushion dimensions:
BUBBLE BASIC – 43 x 43 x 6 cm

BUBBLE EVERYDAY – 42 x 42 x 4 cm

BUBBLE EXPERT – 46 x 38 x 4 cm

(+/-10 mm when inflated)

- Maximum load: up to 120 kg
- Weight:
 - BUBBLE BASIC – 0,13 kg
 - BUBBLE EVERYDAY – 0,65 kg
 - BUBBLE EXPERT – 0,65 kg

7. Application and recommendations

- Pump the cushion using the supplied pump only (for BUBBLE BASIC and BUBBLE EVERYDAY). Make sure to not damage the air valve during pumping.
- Do not pump with a mechanical compressor. Excessive pressure may tear the cushion apart.
- The BUBBLE EXPERT is provided with an automatic valve, so that no pumps should be used (the cushion is inflated automatically).
- Do not exceed the maximum cushion load.
- Do not jump onto the cushion.
- Do not stand on the cushion – risk of injury.
- Do not pump the cushion additionally during use.
- Use only in accordance with the intended use.
- Before using the cushion on a light upholstery fabric carry out a test as there is a probability of stamping dots from the anti-slip bottom of the cover on the fabric.
- Store in a dry room, away from fire and sharp objects.
- Do not expose to high temperatures and sunlight as the

increase of temperature inside the cushion may lead to its bursting.

8. Maintenance

Cushion cleaning:

- Clean the cushion with a damp cloth moistened with mild cleaning agents with NIH (National Institute of Hygiene) certificate. Do not wet the cushion. Wipe it dry after every maintenance. Strong cleaning agents or detergents may damage the product. Do not use abrasives.
- **Attention!** Before cleaning close the pumping valve to protect it from wetting.
- **Attention!** Do not use heat sources for drying.
- **Attention!** Do not use whitening agents, fabric softeners and dry cleaning agents for cleaning!

Protective cover cleaning:

- The protective cover can be washed manually or in the washing machine at 40°C using mild detergents. Before cleaning fasten the zip. The protective cover can be dried with a stream of warm air or in a drier at a low temperature.
- **Attention!** The cushion can be fitted with the protective cover only when the latter has completely dried.
- **Attention!** Do not iron the protective cover!

Disinfection:

- Disinfect using a cloth soaked with a mild detergent.

9. Guarantee

All products distributed by our company are covered by a guarantee with terms and conditions set out in the guarantee certificate that you received when you purchased the product. We would like to point out that for the guarantee purpose you should retain the proof of purchase (receipt or invoice).

As a user-friendly company, we deliver only the products that have been verified in terms of materials, quality and proper functioning.

If you have any questions regarding the servicing procedure, do not hesitate to contact us.

THANK YOU FOR CHOOSING TIMAGO!

10. Markings

	Reference number		Manufacturer
	Lot number		Date of manufacture
	Serial number		Medical device
	Attention		Refer to the manual
	The manufacturer assessed the product's conformity with the essential requirements for medical devices.		

KASUTUSJUHEND

Lamatistevastane padi

Mudelid: BUBBLE BASIC, BUBBLE
EVERYDAY, BUBBLE EXPERT

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2. Tooteinfo

Mudelid BUBBLE EVERYDAY, BUBBLE EXPERT

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