



EL EN DE FR

MULTILINGUAL
MANUALS



MOBIAK®

MEDICAL • GASES • FIREFIGHTING est. 1977

Οδηγίες Χρήσης:

Στρώματος Κατακλίσεων με Αεραντλία 'PREMIUM OCEAN'

User Manual:

Anti Decubitus Air Mattress with Pump

'PREMIUM OCEAN'

Benutzerhandbuch:

Anti-Dekubitus-Luftmatratze mit Pumpe „PREMIUM OCEAN“

Manuel utilisateur:

Matelas pneumatique anti-escarres avec pompe «PREMIUM OCEAN»

0806276 (0806277 & 0806278)



Διαβάστε το εγχειρίδιο οδηγιών πριν από τη χρήση και φυλάξτε το για μελλοντική αναφορά.

Σημαντικές οδηγίες ασφαλείας

Read the instruction manual before use and keep it for future reference.

Important safety instructions

12/3/2024

PRODUCT DESCRIPTION

Anti-decubitus mattresses are specifically designed to treat and help bedsores in bedridden patients. They can be used for hospital or home use. Evenly distribute the pressure on the body to prevent edema.

The pump switches the air pressure in the mattress, creating a sense of light massage and activate the immobilized due sprawling cells.

This product is indicated and suggested for patients who are likely to spend more than 15 hours a day in bed and for patients with low to medium risk of developing ulcers & bedsores

OPERATION USE

For the bubble mattress (0806277)

- Place the air mattress over the bed mattress, having first stretch the bottom sheet, making sure the air inlet is at the side of the legs.
- Secure it with fixing straps below the mattress
- Place over the air mattress a thin breathable fabric, if not accompanied by its own cover, so it does not come in direct contact with the patient's skin.

For the pump (0806278)

- Attach the mattress to the pump using appropriate hoses that are in the package. Connect the pump to the power supply, press the "ON/OFF" switch and turn the knob to the maximum in order to inflate the mattress, BEFORE been used by the patient
- When the mattress is fully inflated you can adjust the pressure range, depending of the user's requirements, using the switch.
- ATTENTION: the STATIC mode is for 20 minutes only!!!

OPERATION USE

For the pump (0806278) :

- Press the “orange” switch to select Static Pressure. This switch activates or deactivates the alternating air flow from the pump to the mattress and gives continuous flow.

EC Declaration of Conformity

We are solely responsible for declaring that the Medical Devices mentioned in this statement are of Low-Risk Class (Class I) and comply with the requirements of the European Regulation 745/2017 and where appropriate, the standards and legislation referred to.

SPECIFICATIONS

PUMP	Voltage (V) - Frequency (Hz)	AC220V/ 50HZ
	Air Output	≥ 5-6 lit/min
	Pressure Range	50mmHg – 110mmHg
	Cycle Time	12 min
	Power Consumption	≤ 7 W ± 10%
	Noise	≤ 25 dB ± 10%
	Dimensions (LxWxH)	26 x 12,5 x 8,5 cm
	Pump Weight	1,3 Kg
BUBBLE MATTRESS	Max Load	130 Kg
	Color	Blue
	Material	Medical Grade PVC
	PVC Thickness	0,3 mm
	Dimensions (LxWxH)	200 x 90 x 7.0 cm
	Hose Length .	120cm
	Weight	2 Kg

MAINTENANCE

- To clean the mattress Use a soft cloth or sponge & water with mild soap. ATTENTION: Do not use acidic & caustic dilutions.
- Do not smoke & do not use any flame near the mattress.
- Do not have the mattress in contact with direct heat sources.
- If you need to save the mattress, empty of air, folding it
- Keep or store the pump in a clean, dry and well ventilated environment.
- Before cleaning the pump, disconnect from the electrical supply.
- Clean the pump with a dry cloth. ATTENTION: Do not bring the pump into contact with liquids.

IMPORTANT (!)

CAUTIONS

- Do not disconnect the hoses while the pump is running.
- Do not use sharp objects near the mattress.
- Do not exceed the maximum load (kg) recommended by the manufacturer.
- Protect the product by oils, lotions and creams, by covering it with the cotton wool fabric, or a bath towel.
- Do not use abrasive alcohol-based cleaning when the mattress. This could damage the material of the layer.
- Clean the upholstery after each daily use.
We recommend that you clean the upholstery with a mild detergent to protect it from oil stains and dust. A common practice is to use a spray bottle filled with a mixture of anti-bacterial soap and water. Spray the surface and then wipe with a clean towel. The best solution is to be cleaned with warm water mixed with a mild soap and then thoroughly dried.
- Protect the layer from scratches, cuts and punctures.
- Do not leave the product run on, If not using it

IMPORTANT (!)

In case of damage or malfunction of the pump, please contact your authorized and trained dealer.

Do not use the product for a purpose not indicated in this manual

Only qualified personnel or trained for the treatment and prevention of bedsores, can operate with the device

MOBIAK S.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product.

MOBIAK S.A reserves the right to change the information contained in this document without previous notice.



Environmental Protection

If you find one day that your appliance needs replacement, or no longer serves you, consider the environment:

- 1) Do not dispose of your device along with the municipal waste (this is also the importance of the stated recycling symbol).
 - 2) Contact your municipal authority to indicate the point of disposal of your device for recycling.
 - 3) With the most useless your device in the right place
- Recycling helps to protect the environment and the operation again your device materials.

SPARE PARTS

REF. NUMBER	DESCRIPTION
0806277	AIR MATTRESS "OCEAN"
0806278	PUMP "PREMIUM"

WARRANTY



The product has warranty for two (2) years in pump and six (6) months on the mattress and the other non-mechanical parts. The warranty covers factory defects and not damages resulting from improper use / maintenance of the product.

The warranty apply factory defects & DOES NOT apply damages resulting from improper use , poor maintenance, modification, abuse or non-compliance with the product's operating instructions.

Damage or defects caused by: natural disasters, unauthorized maintenance or repair, power supply problems (where applicable) and transportation companies are not covered by the warranty.

Warranty Form

Buyers Data

Name:
Date of purchase: S/N.....
LOT:.....Phone:

Resellers Data

Name :
.....
.....
Address:.....
.....
Phone:

Reseller Stamp

INCIDENT REPORTING:

Any serious incident that occurs in relation to use of the device must be reported to the manufacturer and a competent authority of the Member State in which the user and / or patient resides.


MOBIAK S.A
KAOIANA AKPQTHPIOY, XANIA - KPHTH
KATHIANA AKROTIRIOU CHANIA - CRETE
T: +30 28210 63222 | WEB: www.mobiaicare.com



Ερμηνεία Συμβόλων που Βρίσκονται στην Ετικέτα, στο Χαρτοκιβώτιο ή στις Οδηγίες χρήσης του Προϊόντος

Description of symbols that have been printed on the label, Master Carton and or User Manual



Ιατρική Συσκευή
Medical Device



Σήμα συμμόρφωσης CE
CE Mark



Κατασκευαστής
Manufacturer



Ημερομηνία Παραγωγής
Production Date

REF

Κωδικός Προϊόντος
Product Number



Διαβάστε τις Οδηγίες Χρήσης
Read the instruction for use

SN

Σειριακός Αριθμός
Serial Number



Μην χρησιμοποιείται εάν το
κουτί είναι κατεστραμμένο
Do not use if Package is Damage

LOT

Αριθμός Παρτίδας
Batch Number



Τοποθετήστε με αυτή τη Φορά
Right Way Up

UDI

Μοναδικός αναγνωριστικός
κωδικός προϊόντος
Unique Device Identification



Κρατήστε μακριά από
ηλιοφάνεια
Keep away from sunlight



Διατηρήστε Στεγνό
Keep Dry



Υψος στοίβαξης 3 κιβώτια
Stacking Height



Χειριστείτε με προσοχή
Handle With Care



Εύθραστο
Fragile



Χρήση Μόνο για
Εσωτερικό Χώρο In Door
Use Only



Απόρριψη Ηλεκτρικών
Εξαρτημάτων
Waste Electrical and Electronic
Equipment Directive