## OVERBED TABLE

Art.: 272010

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## 1. Introduction

Thank you for buying a high-quality Mobilex product. These instructions for use contain a description and important information about the use of the medical device. It is intended to help you to use the product correctly and safely. It is very important to read the instructions carefully before using the device. Pay particular attention to the safety guidelines and always follow them. Mobilex is constantly trying to improve and upgrade its products. Therefore we reserve the right to change specifications and technical data of the product without further notice.

### Intended use of the device

Designed for use when sitting in either a chair or bed these tables provide a height adjustable flat surface for reading, writing, eating etc. The table top can be tilted for increased comfort during use. The weight of these items must not exceed the carrying capacity of the tray (see specifications below).

### Indications

Due to the wide range of possible applications, the indications cannot be tied to a specific clinical area, since tables are suitable for multiple applications involving people with limited mobility.

### Contraindications

Contraindications cannot be tied to a specific clinical area. Please read the safety information.

### Areas of use

The product can be used in home care, health and long-term care facilities, and hospitals.

### Medical incidents and unwanted events

In case of any unwanted events related to the device, please contact the dealer or manufacturer:

Mobilex A/S	Phone:	+45 87 93 22 20
Grønlandsvej 5	Telefax:	+45 87 93 17 77
DK-8660 Skanderborg	Email:	info@mobilex.dk
Denmark		

## 2. Safety requirements

- Make sure that this manual has been read by all persons using the device.
- The manufacturer assumes no responsibility for damage and/or injury resulting from failure to follow the operating instructions.
- Only use the product in flawless condition.
- If defects or errors are detected you must immediately contact the dealer.
- Follow all manufacturer's recommendations and warnings.
- Be aware of the information on the product label.
- Only use the product for the described purpose.
- Don't climb on to the device
- Avoid making constructive changes on the device, unless you have the manufacturer's written acceptance for such modifications.
- During different adjustments which are possible on the device the user must take care not to squeeze the fingers or other body parts between two moving elements of the device.
- Do not overload the table top, 15kg Max Load.
- Check knobs are tightened at regular intervals.
- Do not attempt to use the table as a walking aid.
- Do not attempt to sit or stand on the table.
- The device must only be used on flat, smooth & stable surfaces.
- All wheels should be in contact with the floor at all times during use. This will ensure the product is properly balanced and should avoid accidents.
- The device is not flame retardant. Don't leave it near to sources of fire, ignition or extreme heat.

### Parts Supplied

The device is supplied in a carton box, disassembled for easier transportation.

The contents of a package are:

- ✓ Table Leg (only with castors) X 2
- ✓ Cross Bar (with castors) X 1
- ✓ Table Top X 1
- ✓ Telescopic Tube X 2
- ✓ Knob - Long X 2
- ✓ Umbraco screw - Short X2
- ✓ Assembly parts and tools (3 x 2 x 1) (1-9)

### Preliminary check

Check the contents of the box with the list above. If at this stage you discover that a part is missing or damaged, contact your supplier immediately.

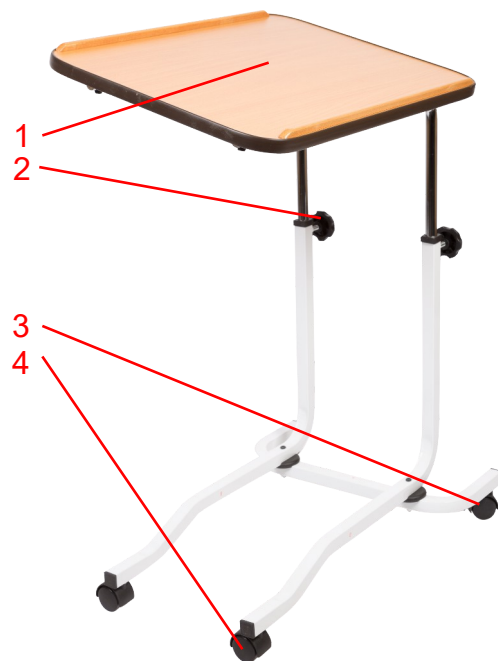
### 3. Description

#### Product description

Overbed table with height and tilt adjustable wooden chipboard tray and a metal frame. It comes with 2 lockable wheels.

#### The table consists of following elements:

1. Chipboard tray - Standard table tray.
2. Height adjustment knobs - They enable adjusting the height and angle of the tray.
3. Wheel locks - They prevent movement of the table.
4. Four turning wheels - Thanks to these wheels the table can be easily re-positioned.

#### Function description

##### Height and tilt adjustment

To adjust the height of the tray, loosen the two knobs (2), slide the tubes to the desired position, and re-tighten the knobs.

To change the angle of the tray, simply slide one of the tubes higher or lower than the other while adjusting it.

##### Lockable turning wheels

The two wheels (3) can be locked in place by pushing down the plastic pedal.

**WARNING!** The wheels must be fully locked and the table secured in place before usage.

#### Assembly

The assembly instructions are on the last page of this manual.

#### Accessories

There are no accessories provided for this device

### 4. Technical data

Overbed Table	Item no. 272010
Size of the tray (width x length)	41 x 55 cm
Height (adjustable)	61 - 87 cm
Weight	7,5 kg
Carrying capacity	15 kg

#### Product label

A product label is placed on the front of the frame.

Example of product label

	QR-code link to homepage		Manufacturer
	CE marked product		Item Number
	Observe the user manual		Max load
	Medical Device		Manufacturing date
	To be used indoor		Serial no

Meaning of the icons

## 5. Care & Maintenance

Normal dirt can be removed with standard cleaning agents and a sponge or soft rag. Check the specific product information and use only commercial cleaning products which are suitable for cleaning and disinfection (do not use alcohol-containing products and no solvents or abrasives).

The device basically does not need maintenance. Despite the solid construction and the use of resistant materials the product is subject to wear. It is therefore recommended to check the product at regular intervals by a professional service. We recommend the following to be checked:

- ✓ Cleaning and disinfection of the product
- ✓ Frame/product should be checked if it has been bent, damaged, worn or corroded
- ✓ Wear parts are checked and replaced if necessary
- ✓ General re-tightening of the product
- ✓ Cleaning the wheels in hot water with a mild cleaning agent. Lubricate the bearings oil periodically or when the wheels start squeaking.

**Expected service life:** Normally 10 years depending on use.

**Storage:** We recommend storage in a dry place in temperatures above 0 °C.

### Transportation

The device should be transported in a stabilized and secured way with no risk of sudden or dangerous movements during transport.

### Reuse

The product is suitable for reuse. Before reusing, where a new user is to take over the product, the product must be checked according to point 5.

## 6. Most common problems and solutions

<i>Problems</i>	<i>Possible causes</i>	<i>Solutions</i>
The wheels don't turn smooth and easily	There is a dirt in the wheels	Clean the space between wheels and forks. Lubricate the axles of the wheels

For realization of basic service actions the set of socket wrenches, or adjustable spanner, flat and cross screwdriver will be sufficient.

## 7. Disposal of the product

The product can not be disposed of with household waste but must be brought to the local recycling center.

## 8. Guarantee terms

Mobilex A/S offers 2 years of warranty for damages caused by errors in production or material errors. The warranty is not valid by not-intended repair or usage of the product. Parts subject to normal wear are not covered by the warranty, unless wear is caused by a manufacturing fault. If product changes are made without our written acceptance the CE-marking and product warranty will not be valid. In the event of product damages covered by this warranty, please notify your dealer or Mobilex A/S directly. The warranty does not cover transport costs and does not include compensation for personal injuries or non-use of the product during repair. The warranty does not cover damages due to the user manual not being followed.

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**Mobilex A/S, Grønlandsvej 5, DK-8660 Skanderborg, Denmark, SRN: DK-MF-000021885**