
RUBBER RAMPS

Art. 306040 - 30672M

Gebrauchsanleitung
Brugervejledning
User Manual
Manuale
Bruksanvisning
Instrukcja Obsługi
Manuel de l'utilisateur

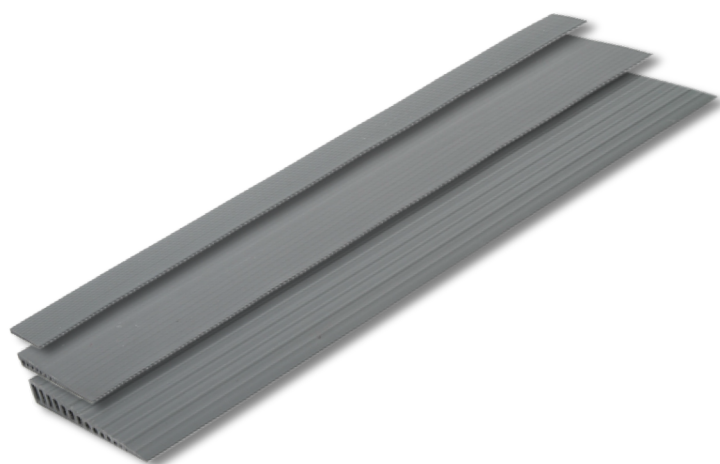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mobilex.dk/medias/2021-09/CE_rubber-ramps_30610L-306602_8601_0x0px.pdf

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1. Introduction

Thank you for buying a high-quality Mobilex product. These instructions of use contain a description and important information about the use of the medical device. It is intended to help you to use the product correctly and safely. It is very important to read the instructions carefully before using the device. Pay particular attention to the safety guidelines and always follow them. Mobilex is constantly trying to improve and upgrade its products. Therefore we reserve the right to change specifications and technical data of the product without further notice.

Intended use of the device

To be used to assist people in wheelchairs or scooters by enabling passage over elevated points.

Indications

Ramps are indicated for people with mobility limitations who due to problems with walking, standing or sitting are forced to use mobility devices such as wheelchairs, scooters, rollators etc. Ramps are indicated for elimination of architectural obstacles by smooth connection of the level differences that a wheelchair user has to overcome and thus they allow the wheelchair user to reach places normally inaccessible to him/her. The ramp is intended for use independently by the user or with the help of an attendant. Using the ramp for purposes other than those indicated above, is prohibited.

Contraindications

Contraindications cannot be tied to a specific clinical picture. The choice of a suitable ramp type and length depends on multiple factors and must always be made with the help of a specialist doctor or physiotherapist. Observe the safety information in chapter 2.

Areas of use

The product can be used in home care, health and long-term care facilities, and hospitals.

Medical incidents and unwanted events

In case of any unwanted events related to the device, please contact the dealer or manufacturer:

Mobilex A/S Grønlandsvej 5 DK-8660 Skanderborg Denmark	Phone: Telefax: Email:	+45 87 93 22 20 +45 87 93 17 77 info@mobilex.dk
---	------------------------------	---

2. Safety requirements

- Make sure that this manual has been read by all persons using the device.
- The manufacturer assumes no responsibility for damage and/or injury resulting from failure to follow the operating instructions.
- Only use the product in a flawless condition.
- If defects or errors are detected you must immediately stop using the device and contact the dealer.
- Follow all manufacturer's recommendations and warnings.
- Be aware of the information on the product label.
- Only use the product for the described purpose.
- Only structural modifications allowed are the ones stated in this user manual unless you have the manufacturer's written acceptance for other modifications.
- The device must not be loaded with more weight than indicated in Technical data.
- During width adjustments the user must take care not to cut the fingers or other body parts.
- The device must only be used on a stable surface.

Parts supplied

The device is supplied in a carton box. The contents of the package are:

- ✓ 1 or more rubber ramps;
- ✓ 1 User manual;

Preliminary check

Check the contents of the box with the list above. If at this stage you discover that a part is missing or damaged, contact your supplier immediately.

3. Description


Product description

Rubber ramps are made of TPE and don't contain toxic substances (PVC free). They are used for door steps and obstacles of small sizes. The product has a grooved surface which makes it slip-resistant. The ramps can be used both indoors and outdoors.


Assembly description

Width adjustment

Width adjustment is possible by cutting the ramp to a desired width using a box cutter or a saw blade tool (accessory). Cutting a tall ramp might require repeated cutting, which will be made easier by bending the ramp backwards and widening the cut groove.

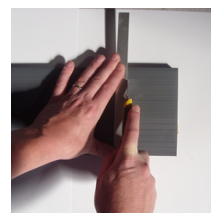Low ramp cutting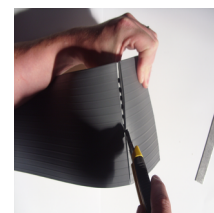

High ramp cutting


Installation with double sided adhesive tape (accessory)

This is recommended for residential adaptations on robust floor coverings such as concrete, tiles, marble floors and varnished wooden floors. The tape is tested for temperature resistance from -30 to 120 degrees.


Clean the bottom surface of the ramp and apply the tape onto it (Pic. 1). In most cases, it is not necessary to use the entire tape to attach the rubber ramp. Remove a few strips of the foil at 3-4 points per ramp (Pic. 2, 3, 4). This makes it easier to detach the ramps when necessary and increases the recycling value. Clean the surface that the ramp will be attached to and place the ramp down so that there will be no gaps between the obstacle and the ramp.


Pic. 1 Application of adhesive tape

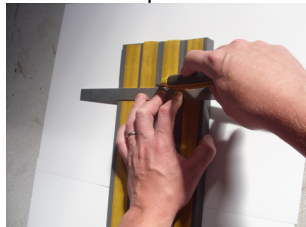Pic. 2 Cutting the foil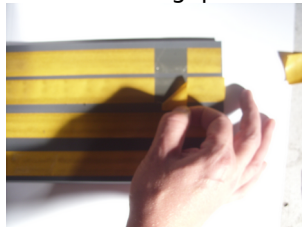

Pic. 3 Removing the foil

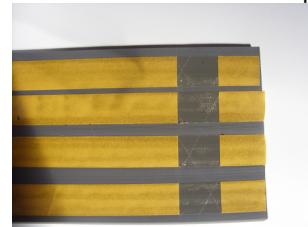
Pic. 4 Ready for installation

Installation with mounting brackets

This is recommended for delicate floor coverings such as carpets, untreated or delicate wooden floors and temporary home adaptations. Ramps with height equal to or above 28mm can be installed with mounting brackets (Pic. 1).


To install the ramp with the mounting brackets, first put the rubber ramp to the edge of the step or an obstacle. Fit the mounting brackets and mark the spots where they will be attached (Pic. 2). The ramp can be moved away for the bracket installation process. Make sure the brackets are mounted with the slots for pins upwards (Pic. 2). Once the mounting brackets are screwed to the obstacle, put the square pins into the sides of the ramp. Slide the screws into the mounting brackets from the top and push the ramp towards the obstacle until the ramp touches it. Tighten the screws in the mounting brackets.

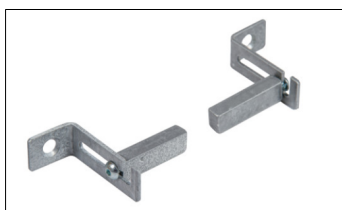Pic. 1 Mounting brackets (306910)

Pic. 2 Using the mounting brackets.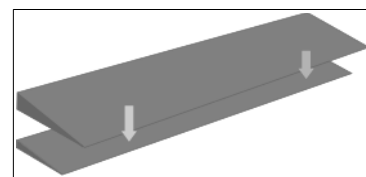

Pic. 3 Combining ramps

Using a set of ramps

It is possible to stack ramps on top of each other to achieve a greater height of the ramp (Pic. 3), as well as combining a few smaller ramps on stair steps to make a larger ramp (Pic. 4).


Function description

Intended use of the device is described in chapter **Introduction**.

Accessories

The device can be equipped with the following accessories (not a part of the standard equipment):


- ✓ Mounting set for rubber ramp zinc plated, 28-60 mm height
- ✓ Jigsaw blade for rubber ramp
- ✓ Double adhesive tape, 20 mm x 50 m
- ✓ Double adhesive tape, 35 mm x 50 m
- ✓ Degreaser for removal of glue 1/2 L

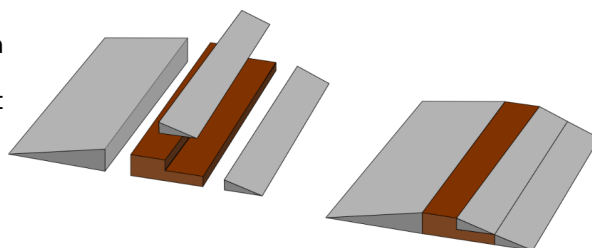Pic. 4 Adaptation of ramps to doorsteps of different heights

4. Technical data

Height	Depth	Width	Weight	Height	Depth	Width	Weight
4 mm	40 mm	900 mm	130 g	64 mm	350 mm	700 mm	4620 g
4 mm	40 mm	1000 mm	150 g	64 mm	350 mm	900 mm	5940 g
6 mm	60 mm	900 mm	310 g	64 mm	350 mm	1000 mm	6600 g
6 mm	60 mm	1000 mm	350 g	68 mm	350 mm	700 mm	4830 g
8 mm	80 mm	900 mm	490 g	68 mm	350 mm	900 mm	6210 g
8 mm	80 mm	1000 mm	540 g	68 mm	350 mm	1000 mm	6900g
10 mm	100 mm	900 mm	770 g	72 mm	350 mm	700 mm	4970 g
10 mm	100 mm	1000 mm	860 g	72 mm	350 mm	900 mm	6390 g
12 mm	120 mm	900 mm	1070 g	72 mm	350 mm	1000 mm	7100 g
12 mm	120 mm	1000 mm	1190 g				
16 mm	150 mm	900 mm	1230 g				
16 mm	150 mm	1000 mm	1370 g				
20 mm	150 mm	900 mm	1850 g				
20 mm	150 mm	1000 mm	2050 g				
24 mm	150 mm	900 mm	1950 g				
24 mm	150 mm	1000 mm	2170 g				
28 mm	200 mm	900 mm	2180 g				
32 mm	200 mm	900 mm	2960 g				
36 mm	200 mm	900 mm	3990 g				
40 mm	250 mm	900 mm	3550 g				
44 mm	250 mm	900 mm	4250 g				
48 mm	250 mm	900 mm	5190 g				
52 mm	300 mm	900 mm	5800 g				
56 mm	300 mm	900 mm	6500 g				
60 mm	300 mm	900 mm	7700 g				
60 mm	300 mm	900 mm	4770 g				
60 mm	300 mm	1000 mm	5300 g				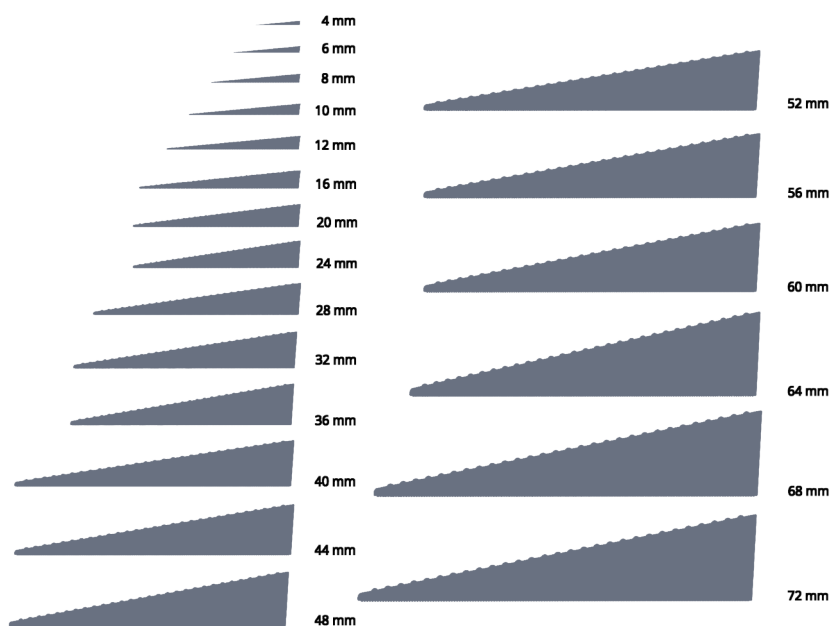

Rubber ramps are also available in length 700 mm in sizes 10-48 mm (all models with self-adhesive tape).

Product label

Example of the product label

	QR-code link to homepage		Manufacturing date
	Manufacturer		Max load
	Observe the user manual		Batch no/serial no
	CE-marked product		Item Number
	To be used indoor and outdoor		Medical Device
	Max Elevation / height		

Meaning of the icons

5. Care and maintenance

Regular dirt can be removed with standard cleaning products and a sponge or soft rag. Check the specific product information and use only commercial cleaning products which are suitable for cleaning. For disinfection, use only disinfectants certified for use with medical devices available locally.

Despite the solid construction and the use of resistant materials the product is subject to wear. It is therefore recommended to check the product at regular intervals by a professional service. We recommend the following:

- ✓ Cleaning and disinfection of the product
- ✓ Checking the product for damage or wear and replacing it if necessary

Recommended service life: Normally 10 years, depending on use.

Storage: We recommend storage in a dry place in temperatures above 0 °C.

Transportation

The device is not designed for transportation as it is fixed to the desired surface.

Reuse

The product is suitable for reuse. Before a new user takes over the product, the device must be checked on a technical level and disinfected according to point 5. When ramps with applied adhesive are reused, the old adhesive must be removed and new tape applied.

Cleaning the attachment point: When removing the rubber ramps installed with adhesive tape, adhesive residue may remain on the surface. These are easily removed with common tools (e.g. a spatula) and the remaining adhesive residue can be removed with the degreaser accessory (art. no. 306905) or a similar product.

6. Most common problems and solutions

Problems	Possible causes	Solutions
The rubber ramp is unstable or does not stay in the desired position	The surface is not leveled	Use the mounting kit to attach the ramp to the doorstep or seek alternative installation methods*

* On delicate surfaces such as carpets and wooden floors, it is not recommended to attach rubber ramps with adhesive tape.

7. Disposal of the product

The product can not be disposed of with household waste and must be brought to the local recycling center.

8. Guarantee terms

Mobilex A/S offers 2 years of warranty for damages caused by errors in production or material errors. The warranty is not valid by not-intended repair or usage of the product. Parts subject to normal wear are not covered by the warranty, unless wear is caused by a manufacturing fault. If product changes are made without our written acceptance the CE-marking and product warranty will not be valid. In the event of product damages covered by this warranty, please notify your dealer or Mobilex A/S directly. The warranty does not cover transport costs and does not include compensation for personal injuries or non-use of the product during repair. The warranty does not cover damages due to the user manual not being followed.

9. Declaration of conformity

The device has passed the conformity assessment in accordance with EU Regulation: MDR 2017/745 regarding medical devices and is CE marked. A declaration of conformity has been issued for the device and is available at:

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