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INTENDED USE:

Intended for the prevention of bed sores of people who maintain a sedentary lifestyle. The first step in preventing bed sores is to change the pressure points on the body. When placing the anti-decubitus cushion, it is important for the patient to stand up or to be lifted at regular intervals to limit prolonged pelvic compression in the area of the protruding bones.

CAUTION (!):

- In case of damage or malfunction of your product DOT NOT use the product and please contact the authorized and trained representative.
- Do not use the product for a purpose not indicated in this manual
- MOBIAS S.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product.
- MOBIAS S.A reserves the right to change the information contained in this document without previous notice
- There is a $\pm 3\%$. Deviation for the Dimensions of the Products.

GENERAL SAFETY WARNINGS:

- If you don't read this user manual, it is preferable not to use this product or another available part. If you don't understand the usage or the precautions, please contact with the dealer or the proper technical person before you use the product because may be caused damage.
- Keep away the product from heat sources.
- Save this document for future reference.
- Be careful when children are nearby and do not allow the children to play with the product.
- Do not use the product for a different purpose from the intended one.
- Do not exceed the weight limit of the safe working load.

BEFORE EVERY USE:

- Check if the frame is damaged in order to guarantee a safe use of the product.
- In case of damage do not use the product and contact your dealer for further instructions.
- Always check the condition of wear for the parts to ensure the absolute safe use of the product.

ENVIRONMENTAL PROTECTION:

If one day you find that your product needs to be replaced or it is no longer working for you, consider protecting the environment:

- 1) Do not dispose your product along with the rest of the public waste (this is also the meaning of the shown recycling sign).
- 2) Contact your Public Authorities and they will instruct you of the Recycling centers to which your product must be disposed.
- 3) Correct disposal of your product helps the protection of the environment as well as the recycling of the product's components.

CLEANING & DISINFECTION INFORMATION:

To guarantee a safe use and adequate standard of hygiene, the user should perform these procedures before every use.

The user must make certain of the structural integrity of the device and its components. For cleaning and sanitation procedures, follow the steps below:

1. Gloves should be worn.
2. Remove the evident dirt before proceeding to accurate cleaning.
3. Use water or mild detergent for the cleaning procedure.
4. Wipe with a dry and clean cloth.
5. DO NOT leave the product wet.
6. Do not use chemical cleaners to clean the frame. This could cause damage to the surface of the product
7. If you need to disinfect the product, use a common, mild disinfectant
8. Protect the item from scratches, cuts and punctures.
9. Do not wash it in the washing machine and do not use a dryer to dry it. Do not use a steam cleaner to clean it.

ASSEMBLING:

The Product are delivered assembled & ready to use.

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Description of symbols that have been printed on the label, master carton and or user manual

**FURTHER INSTRUCTIONS:**

1. Inflate the pillow with the air piston.
2. Place the pillow on the seat.
3. The user can now sit on the Cushion.
4. We recommend checking and refill the cushion pressure regularly.

DECLARATION OF CONFORMITY:

We are solely responsible for declaring that the Medical Devices mentioned in this statement are of Low-Risk Class (Class I) and comply with the requirements of the European Regulation 745/2017 and where appropriate, the standards and legislation referred to.

INCIDENT REPORTING: Any serious incident that occurs in relation to use of the device must be reported to the manufacturer and a competent authority of the Member State in which the user and / or patient resides.

WARRANTY:

The product is guaranteed for a period of one (1) year from the purchasing date. The warranty apply on factory defects & DOES NOT apply on damages resulting from misuse, poor maintenance, modification, overuse, or non-compliance with the product's operating instructions. Also DOES NOT apply on the parts, that can wear out during use or over time. Damages or defects caused by natural disasters, unauthorized maintenance or repair, power supply problems (where applicable) & carriers are not covered by the warranty. No service or spare part is applied unless it has been inspected by an authorized service department or the manufacturer of the product. Shipping costs for goods of warranty are to be paid by the end user-customer or by dealer.

BRUGSANVISNING: ANTI-TRYKSÅRSPUDE MED HUL 45 CM PVC GRÅ
USER MANUAL: ANTI-DECUBITUS CUSHION WITH HOLE 45CM PVC GREY

REF: 0803308

MOBIAK S.A
KAGIANA AKROTIRIOY, XANIA - KPTH
KATHIANA AKROTIRIOY CHANIA - CRETE
T: +30 28210 63222 | WEB: www.mobiaicare.com