**DA      EN**

**BRUGSANVISNING: [TOILETSTOL](#)**  
**USER MANUAL: [COMMODE CHAIR](#)**

**REF:0803390**

**CE**

*REVISION 06/2023*

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## INTENDED USE

The commode chair is used as a toilet aid by people with limited mobility and is designed for indoor use only.

It is intended for people with special needs, patients, and the elderly as well as for people with mobility difficulties.

**Suitable for use in shower.**

## SPECIFICATIONS

FRAME	ALUMINUM, PP (POLYPROPYLENE), HEIGHT ADJUSTABLE IN 6 LEVELS
COLOR	GREY
SEAT MATERIAL	PE & PVC
BACKREST MATERIAL	PE & PVC
<b>WITH DETACHABLE ARMRESTS, DETACHABLE BACKREST AND LEGS</b>	

## DIMENSIONS

46cm	43cm	54cm	42-54.7cm	36cm	5.1Kg	120Kg

## **SYMBOLS**

Please refer on **ANNEX A**

## **ASSEMBLY**

Please refer on **ANNEX B**

## **MAINTENANCE**

The purpose of periodic check of the commode chair, is to confirm its proper operation as an aid and must be done at least once a month.

- Check the commode status for any breaks, cracks, or leaks.
- Check plastic parts for looseness, cracks or breaks.
- Check the stability of the commode chair.

In case of repair, use only authentic spare parts.

## **CLEANING**

Clean the commode immediately after use. Initially, rinse with warm water. Then use a soft cloth impregnated in a mild detergent or antiseptic solution. Wipe with a clean, soft cloth.

**CAUTION:** Do not use acidic or caustic solutions.

We recommend that you clean the frame and the seat with a mild detergent or antiseptic solution. For your own convenience, you can use a spray bottle after adding the solution. Spray the product and then clean with sponge. With a clean cloth, pass the surfaces to dry.

## **IMPORTANT(!)**

- When adjusting the height, make sure the height of the 4 legs is the same and the pins are fixed firmly.
- Must be placed in a dry, ventilated room free of corrosive materials.
- When transporting it must be handled with care.

Before using the commode chair, make sure:

- It is in full extent & secured in this position.
- The seat is properly applied & secured in this position.
- The commode is correctly positioned in the appropriate slot.
- There are no cracks or breaks on the frame.
- Do not smoke & do not use flame near the product.
- Protect the product from abrasions, cuts and jabs.
- When adjusting the height, make sure the height of the 4 legs is the same and the pins are fixed firmly.
- Must be placed in a dry, ventilated room free of corrosive materials.
- When transporting it must be handled with care.

## **OTHER INSTRUCTIONS**

- When sitting or raised, you must press the handles down so that the commode chair stays still and not slipping.
- It must not be used on slippery surfaces.
- When you are going to get up you need to rely on both handles. Support only in one hand (and without companion) can cause accident with damage or injury.
- Regularly clean the commode chair.
- Do not leave the commode chair wet.
- Do not let children near the commode chair without supervision.

## **WARNING**

In case of problem, do not use the commode chair until the problem is solved.

- Do not use the product for a purpose not indicated in this manual.
- The manufacturer declines all responsibilities for any consequences resulting from an incorrect use of this product.
- The manufacturer reserves the right to change the information contained in this document without notice.
- Do not allow children to play on the commode chair.
- Do not attempt to lift the commode chair by any removable parts.
- Keep the packed commode chair away from heat sources.
- Getting in and out of the commode chair may be performed independently, provided you have an adequate mobility and upper body strength.

## **ENVIRONMENTAL PROTECTION**

If you find one day that your product needs to be replaced or if it is of no further use to you, think of the protection of the environment:

- 1) Do not dispose your product along with the rest of the public waste (this is also the meaning of the shown recycling sign).
- 2) Contact your Public Authorities and they will instruct you of the Recycling centers to which your appliances must be disposed.
- 3) Correct disposal of your product helps the protection of the environment as well as the recycling of the product components.

## **INCIDENT REPORTING:**

Any serious incident that occurs in relation to use of the device must be reported to the manufacturer and a competent authority of the Member State in which the user and / or patient resides.

## WARRANTY

Your commode chair is guaranteed for 2 years from the purchase date.

The warranty covers factory defects & does NOT cover defects resulting from misuse, poor maintenance, modification, overuse, or non-compliance with the product instructions for use. It also does NOT cover worn parts during use. Damage or defects caused by: natural disasters, unauthorized maintenance or repair, power supply problems (where provided), transport companies are also not covered by the warranty. Shipping costs for goods of warranty are to be paid by the customer or by dealer.

## WARRANTY CLAIM FORM

BUYERS DETAILS			
FULL NAME:			
PURCHASING DATE:		PHONE:	
SELLER DETAILS			
FULL NAME		ADDRESS	
PHONE		<b>SIGN AND SIGNATURE</b>	

### E.C. Declaration of Conformity

We are solely responsible for declaring that the Medical Devices mentioned in this statement are of Low-Risk Class (Class I) and comply with the requirements of the European Regulation 745/2017 and where appropriate, the standards and legislation referred to.

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Description of symbols that have been printed on the label, Master Carton and/or User Manual

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