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INTENDED USAGE:

The shower chair allows the user to increase independence during showering, in daily personal hygiene, reducing muscle fatigue due to standing as well as the risks of slipping and injuries. The non-slip rubber feet ensure grip with the floor & the bathtub/ shower surface and the height-adjustable feet allow the user to easily change the overall height according to their requirements. The backrest provides additional safety and comfort. The bath seat can be easily moved due to its low weight and rotated 360° through the use of an easy-to-use lever.

INDICATIONS AND CONTRAINDICATIONS:

The shower chair is aimed to elderly people, wheelchair users, people with reduced mobility of lumbar, knee or hip, and to all those who have difficulty to maintain the standing position during the shower stages of daily personal hygiene. The user is required to have autonomy of balance during the sitting and standing procedures as well as to have adequate ability to keep sitting. The shower seat must not be used if the user has problems related to balance or perception disorders.

CAUTION (!):

- In case of damage or malfunction of your product DOT NOT use the product and please contact the authorized and trained representative.
- If you don't read this user manual, it is preferable not to use this product or another available part.
- Do not use the product for a purpose not indicated in this manual
- MOBIAK S.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product.
- MOBIAK S.A reserves the right to change the information contained in this document without previous notice
- There is a ± 3%. Deviation for the Dimensions of the Products.
- Keep away the product from heat sources.
- Save this document for future reference.
- Be careful when children are nearby and do not allow the children to play with the product.
- Special care is always required where there are moving parts that can cause limb entrapment and injury.
- Do not attempt to lift the product by any removable parts
- Protect the product from scratches, cuts and punctures.

SAFETY WARNINGS AND INDICATIONS:

- The maximum user weight indicated in the instructions must not be exceed.
- Before every use, be sure the seat is correctly assembled in a stable position.
- The seat can be used only on surface that are free from obstacles.
- During assembly, strictly adhere to what is shown in the User Manual.
- The risk of finger injuries may increase during the mounting or assembling process.
- Do not swing on the seat, this may lead to slipping, overturning or subsidence of the legs. If you are using the shower chair, always keep the seat on its 4 legs.
- Do not modify the seat and its components in any way.
- Do not use the shower seat if damaged or altered in any way. Cracks and deformations may strongly affect the performances and safety of the device.
- After use, the shower seat should be removed from the shower tray to avoid any obstruction.
- Do not use the shower seat for a different purpose from the intended one.
- The shower seat must not be stored outdoors.
- Check if the shower chair's legs are firmly on the floor, the armrests are secured, and if the chair rotates smoothly.

MAINTENANCE INFO:

In order to guarantee a safe use and adequate standard of hygiene, the user should perform these procedures before every use.

The user must be certain of the structural integrity of the device and its components. For cleaning and sanitation procedures, follow the steps below:

- Gloves should be worn.
- Remove the evident dirt before to proceed with the most accurate disinfection.
- Use water or mild detergent for the cleaning procedure.
- Wipe with a dry and clean cloth.
- If you need to disinfect the product, use a common, mild disinfectant

ENVIRONMENTAL PROTECTION:

If one day you find that your product needs to be replaced or it is no longer working for you, consider protecting the environment:

- Do not dispose your product along with the rest of the public waste (this is also the meaning of the shown recycling sign).
- Contact your Public Authorities and they will instruct you of the Recycling centers to which your product must be disposed.
- Correct disposal of your product helps the protection of the environment as well as the recycling of the products' components.

INCIDENT REPORTING:

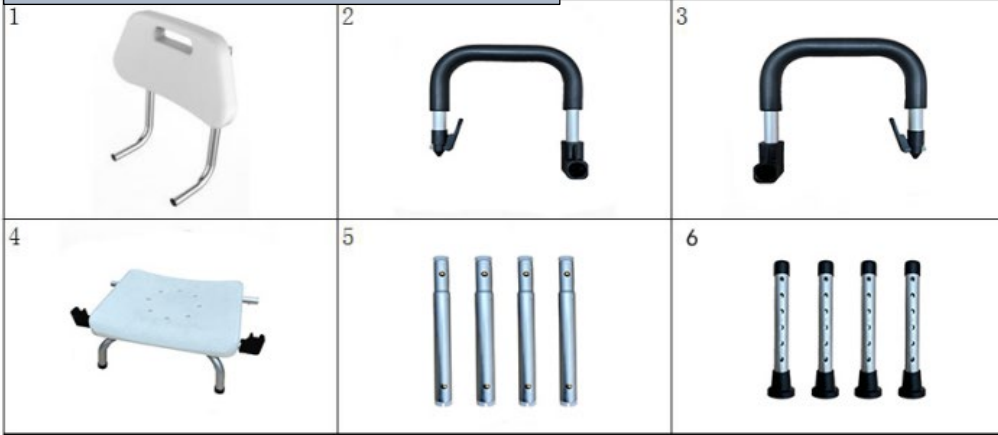
Any serious incident that occurs in relation to use of the device must be reported to the manufacturer and a competent authority of the Member State in which the user and / or patient resides.

DECLARATION OF CONFORMITY:

We are solely responsible for declaring that the Medical Devices mentioned in this statement are of Low-Risk Class (Class I) and comply with the requirements of the European Regulation 2017/745 and where appropriate, the referred standards and legislation.

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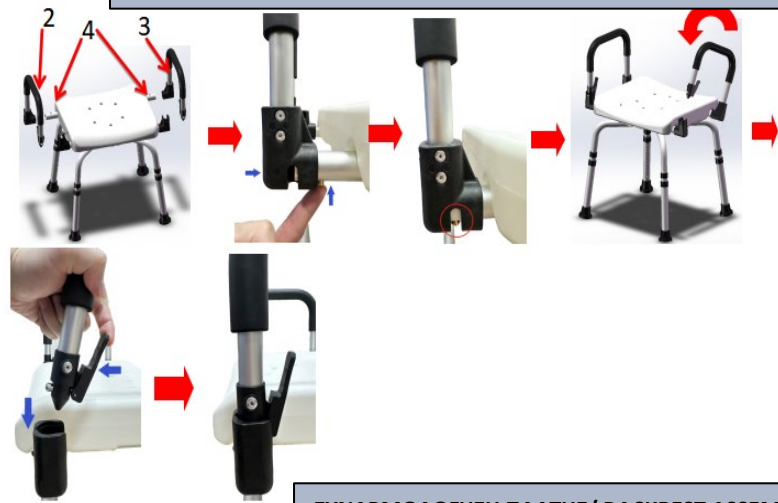
WARRANTY:

The product is guaranteed for a period of two (2) years from the purchasing date. The warranty apply on factory defects & DOES NOT cover damages resulting from misuse, poor maintenance, modification, overuse, or non-compliance with the product's operating instructions.

Also DOES NOT cover the parts, that can wear out during use or over time. Damages or defects caused by natural disasters, unauthorized maintenance or repair, power supply problems (where applicable), carriers are not covered by the warranty.




No service or spare part is applied unless it has been inspected by an authorized service department or the manufacturer of the product. Shipping costs for goods of warranty are to be paid by the end user- customer or by dealer.

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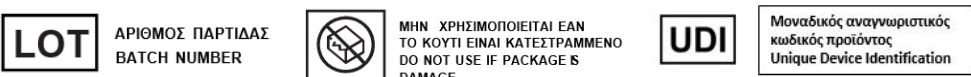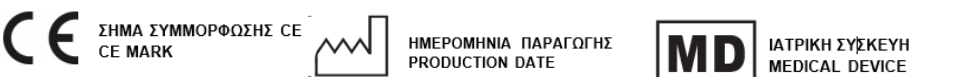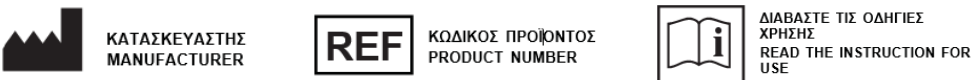Description of symbols that have been printed on the label, master carton and or user manual

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When using the swivel bath chair, if you need to rotate it, first pull the lever on the side of it and then rotate it. You do not need to pull the lever for a long time while rotating the chair. There are four directions of rotation (360°), from one direction to the next, the lever will make a "click", indicating that the swivel chair has locked once and is not rotating.

REF								
0805660	50cm	50cm	40cm	34cm	34cm	43-55 cm	136kg	4.9Kg

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MOBIAK®
MEDICAL • GASES • FIREFIGHTING est. 1977

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