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Introduction

We would like to thank you for choosing the MOBIAK S.A. electric wheelchair. The product User Manual is an important document that provides you information on the technical characteristics, operating modes, and after-sales support of the electric wheelchair.

Please read the user manual carefully before using the product. Read the "Safety" section carefully and use the product correctly according to the instructions in this manual.

- ✧ This manual contains operations, assembly methods, and simple faults solutions.
- ✧ This manual applies to our models: **0805721**
- ✧ This manual contains wheelchair maintenance and self-checking methods, please put it in proper place.
- ✧ Please provide this manual for reference when other people are going to use this wheelchair.
- ✧ The annotations and illustrations in this manual might be slightly different with the real parts due to quality improvement or changing design. Please check it carefully.
- ✧ The wheelchair that you have in your hands may differ from the one in the following photos. The assembly process is similar.
- ✧ Contact with your dealer if there is any ambiguity or question.
- ✧ Improper use of any vehicle may lead to injury. Unsafe driving could harm yourselves and others.
- ✧ The electric wheelchair is intended to comfortably transport persons with walking difficulties or no walking abilities.
- ✧ This electric wheelchair is designed to transport 1 person only.
- ✧ In case of damage or malfunction of your product DOT NOT use the product and please contact the authorized and trained representative.
- ✧ Do not use the product for a purpose not indicated in this manual.
- ✧ MOBIAK S.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product.
- ✧ MOBIAK S.A reserves the right to change the information contained in this document without previous notice.
- ✧ There is a $\pm 3\%$. Deviation for the Dimensions of the Products.
- ✧ It is always recommended that you follow the procedures designed to deal with emergency situations.
- ✧ Always follow your doctor's instructions.

Warning sign.

Follow the instructions next to this symbol carefully.

Failure to pay attention and follow the instructions may result in physical injury or damage to the wheelchair or to the environment.

Product Performance

This series of electric wheelchairs are powered by battery, driven by motor. Users control direction and adjust speed by joystick controller. It is suitable for application of low speed, good road condition and small slope.

Main Structure

This series of electric wheelchairs are mainly composed of two parts:

- 1) Frame Part: frame, wheel, footrest, armrest, backrest etc.
- 2) Electrical Part: controller, motor, battery.

Intended Use

Our electric wheelchair is for disabled and elderly people who use it as a mobility vehicle, as well as for frail people who have difficulty on walking. It is suitable for outdoor use, for medium distances and is not suitable for use on motorways.

1. Safety instructions

Warning

Contraindications: The patients with mental problems, including the users who temporarily or permanently lose attention or judgment, are not allowed to use this product.

Attention

1.1 Electrical safety.

This Electric wheelchair has passed the test in accordance with ISO 7176-9 Climatic Test. This test can ensure that the users of the Electric wheelchair or attendants have sufficient time to move to safety place in a rainy day.

- DO NOT operate the Electric wheelchair under thunderstorm.

- DO NOT place this Electric wheelchair under rainstorm.
- DO NOT place this Electric wheelchair in a wet place for a long time.
- DO NOT spray, wash or wash the Electric wheelchair in an automatic car washer.
- Direct exposure to rain, smoke, or humid air may cause electrical and mechanical failure of the Electric wheelchair and may also cause rust.

Attention!

1.2 Conditions of use.

Ambient temperature: -10°C~50°C.

Relative humidity: 20%~95%.

Atmospheric pressure: 70kPa-110 kPa; With no strong electromagnetic interference.

1.3 Driving on slopes.

- This Electric wheelchair has passed the test requirements of climbing on a slope of not exceeding 15° with a load of 150kg. DO NOT try to climb the slopes exceeding 15°
- During reversing on a slope, please be sure to slow down the Electric wheelchair. DO NOT reverse on a slope exceeding 15°, during reversing, please be extremely careful.
- The ability of climbing slope and driving distance are affected by battery and the current the conditions of use. For example: The weight of the users, the land morphology (such as grass or gravel), Gradient of hill, Battery life and service life, extreme temperature, Use and weight of the attached accessories, etc.
- Avoid driving on steep slopes. The slope range should be up to 15 degrees.
- Keep moving forward when going uphill or downhill, maintain a low speed when going downhill.
- DO NOT drive on steps and avoid pit or potholes.

1.4 Braking

When the Electric wheelchair brakes, please sit properly, steadily hold the handrail and lose controller joystick, the Electric wheelchair will stop in several seconds.

Note: Electromagnetic brake will not take effect immediately, it will take effect after the wheel rotates for 1/2 cycle.

1.5 Emergency Braking.

In case of critical situation or unexpected movement, press the power switch key to make the Electric wheelchair stop (Although this emergency brake is effective, it must not be used under normal conditions).

Note: Frequent use of emergency brake may lead to the motor fault, which further causes that the Electric wheelchair cannot be operate normally.

1.6 Use of mobile phone

During operating the Electric wheelchair, DO NOT use mobile phones or wireless devices. Please pay attention on the road condition when you driving.

Use of mobile phones or wireless devices will generate strong electromagnetic fields, the electronic system of the Electric wheelchair will be interfered. If the mobile phones or wireless devices have to be used, the Electric wheelchair has to be stopped and powered off in advance. Follow the Highway Code.

1.7 Protection from Electromagnetic Interference

The wheelchair should away from strong magnetic fields and large inductive electrical equipment. Pay attention to that if there are sources of electromagnetic interference nearby, as far as away from those sources to avoid electromagnetic interference.

Warning!

1.8 General safety warnings (read carefully)

- Please practice in parks or other safe open places until you can use the wheelchair skillfully.
- Fully practice driving in a safe place, to master principle of moving forward, stopping and turning circles.
- Please strictly follow traffic rules of the pedestrian, do not regard yourself as vehicle driver.
- Please drive on the sidewalk and the zebra crossing. Do not drive on vehicle lane.
- Steadily drive, avoid driving in 'Z' line or sharp turn.
- Please keep pressure of pneumatic tire properly. The abnormal tire pressure may cause unsteady driving or excessive loss of battery energy.
- DO NOT try to pick up objects by bending down when sitting on the Electric wheelchair, the Electric wheelchair or user may roll over or be damaged.
- DO NOT use the Electric wheelchair on escalators, which may increase the risk of being injured.
- DO NOT drive in a vehicle lane or anywhere other than on pavements.
- DO NOT drive on the surface of water, oil, ice or other slippery objects. In case of violation, the Electric wheelchair may be out of control.
- DO NOT attempt to cross over obstacles and ditches exceeding **40mm**, violation of this operation may cause personal injury.
- DO NOT turn sharply and do not reverse at high speed.
- DO NOT carry passengers.
- DO NOT try to drag another Electric wheelchair.
- DO NOT operate the Electric wheelchair without checking the fasteners, connectors or detachable parts for correct connection and fastening. Please always pay attention to the surrounding environment.
- DO NOT repair the parts, accessories, or adapters of the Electric wheelchair without authorization.
- DO NOT connect other medical equipment (such as life support equipment and respirator, etc.) to the electronic system of the Electric wheelchair.
- DO NOT operate the Electric wheelchair when your judgement may be affected (such as drink, take drugs or take medicine).
- DO NOT operate the Electric wheelchair if you suddenly DO NOT feel well.
- DO NOT operate the Electric wheelchair when your vision is severely weakened.

- For the injury accident caused by violation of local or international laws or regulations, we undertake no responsibility.
- All the functions can only be operated after you correctly sit on the seat.
- DO NOT place your feet in any part out of the footrest.
- DO NOT stand on the seat of the Electric wheelchair.
- DO NOT try to drive on a slope exceeding **15°**.
- DO NOT reverse on a slope exceeding **15°**.
- When driving on a slope, if you release the joystick of the controller, the Electric wheelchair may move back. When moving forward or backward, the Electric wheelchair will roll back about 30 cm before the brake function takes effect.
- Before use of the Electric wheelchair, please check the connection of all electric parts for safety and stability.
- DO NOT disconnect, snip or modify the wiring harness ports which have been installed on the Electric wheelchair or connected to the Electric wheelchair under any condition.
- DO NOT use other unqualified batteries, for example, the lead acid batteries which cannot be recycled, etc.
- Before installation, please read the information related to batteries and battery charging.
- If you don't read this user manual, it is preferable not to use this product or another available part.
- If you don't understand the usage or the precautions, please contact with the dealer or the proper technical person before you use the product because may be caused damage.
- Be careful when children are nearby and do not allow the children to play with the product.
- Do not exceed the maximum payload.
- Special care is always required where there are moving parts that can cause limb entrapment and injury.
- Do not attempt to lift the wheelchair by any removable parts.




In case of non-Compliance of the above requirements, there may be unexpected injuries or damages.

2. Product Introduction

2.1 Product features and functions

- The DC brushless motor which is the core part to drive the Electric wheelchair. The motor is featured for its large torque, high efficiency and power saving.
- Using lithium battery has high density energy & long life.
- The intelligent control of motor can be done by using the controller with the joystick. Can be done the following actions: forward movement, differential turning and low speed reversing, which makes the control of the product easier.
- The frame structure mechanism achieves quick and efficient folding and unfolding actions. The Electric wheelchair is equipped with the function of manual folding, and it has the advantages of being safe, reliable and simple to operate.
- The foldable frame structure has the advantages of portability. It is compact and easy to carry (trunk and air transportation) since it does not occupy too much space.
- In case of no operation (with a key, the joystick or the APP) within 30 or 60 minutes, the Electric wheelchair will power off automatically. There is an option to select If it will power off at 30 or 60 minutes, or not at all.
- To enter the cruise control mode, press for 3 seconds the "speed ▲ " button. Then push the joystick forward so the Electric wheelchair start moving and to adjust the speed.
- There is a mode for novice users.: Press Fn to enter the novice mode. The voice prompts "Enter the novice mode" sounds and the screen displays the "novice mode" icon. The first and second gear are available only. Speed gear limit 2.

2.2 Component introduction & Structure of the product (unit: mm)

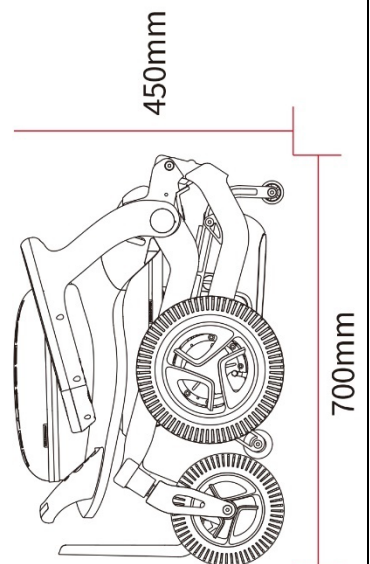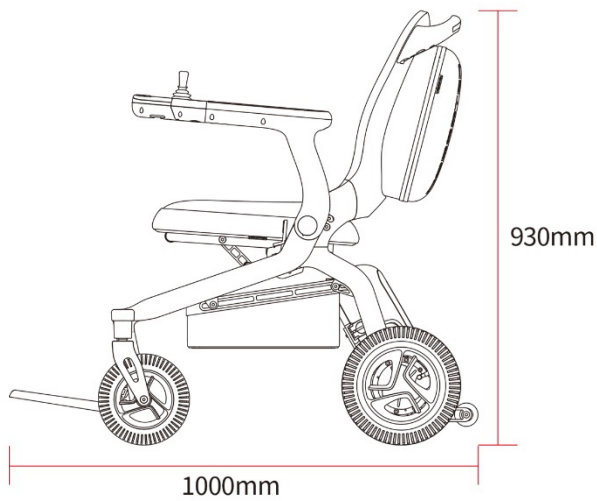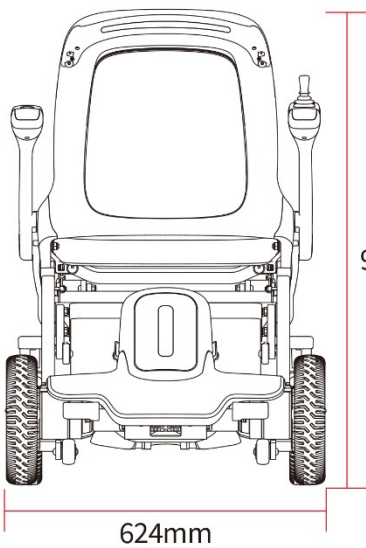2.3 Technical Characteristics of the product 0805721

Folding mode	Manual
Motor parameters	DC motor, rated voltage 24V, Rated power 200W*2
Lithium battery parameter	1 pc X 24 V DC, 20 AH
Battery weight	1 pc X 1,94 ±0,15 kg
Total weight	25 ±2 kg
Maximum Speed	8 km/h
Approximate Autonomy	23 km
Max Slope	15°
Front and rear wheel specifications	The front wheels are 8" solid, and the rear wheels are 10" pneumatic with PU tire.
Brake	Electromagnetic brake
Foot plate	Fixed on the frame & foldable
Suspensions	No
Max Load	150 kg
Bearing capacity of the storage basket	5 kg
Maximum barrier height	40 mm
Ditch width	100 mm
Minimum radius of gyration	850 mm
Seat	Fabric
Seat Wide	460 mm
Armrestes	Foldable
Overall dimensions	L:1.000 mm, W:624 mm, H:930 mm
Dimensions for storage	L:700 mm, W:624 mm, H:450 mm

2.4 Package Contents

The package contains, the Main unit x1, the Battery x1, User's manual the x1 and the Charger with the power Cable x1.

2.5 Storage conditions

- The Electric wheelchair should be stored in its packaged, at temperature of $-20^{\circ}\text{C} \sim 60^{\circ}\text{C}$.
- The humidity has to be no more than 95%
- The Atmospheric pressure shall be between 56~106 kPa.
- The Electric wheelchair shall be stored in a room without the harmful gas which may cause corrosion and with good ventilation. It is also not allowed to be stacked and pressurized.
- When packed, the maximum stack height is five layers.
- DO not store the product for a long term into places which get close to high temperature source or direct sunlight.
- If you need to store it for a long time, fist fully charge the battery and then turn off the power switch.
- If the product has been stored for more than a month, fully charge the battery and then use it or continue storage. That is to check the level and health of the battery every month.
- After long-term storage, all batteries in the first charge cycle provide less capacity than rated capacity. Full capacity can be reached after several charge/discharge cycles, usually at least 3 cycles.

3. Use of Functions

Attention!

3.1 Driving Notice

- When operating the Electric wheelchair for the first time, try to choose a flat and hard ground and ensure that there are no obstacles and pedestrians around you.
- The maximum load is 150 kg. DO not drive overload. Limited use for 1 person.
- Ensure that the Electric wheelchair is completed unfolded.
- Ensure that you have fastened the safety belt before you start moving.
- Sit up properly on the Electric wheelchair and gently push the joystick in the direction you want. With this way it moves forwards or backwards and turns left or right. After release the joystick it will stop. Practice these basic functions until you are familiar with the handling of the wheelchair.
- When driving, make sure there are no obstacles around you.

- When driving. DO NOT steer on pedestrians.
- When operating the electric wheelchair in relatively difficult conditions (such as entering the door or steering), make delicate handling. In difficult conditions, it is recommended to select the lower speed.
- Check that the condition of the electric wheelchair is good and that it works properly.
- The electric wheelchair will switch off automatically after 30 or 60 minutes of inactivity.

3.2 Folding and Unfolding

Unfolding procedure: When manually expanding the electric wheelchair, please press the folding switch with one hand (position 1 in Figure 1) and pull the armrest up with the other hand (position 2 in Figure 1) until the electric wheelchair is fully expanded.

Folding procedure: When manually folding the electric wheelchair, please press the folding switch with one hand (position 1 in Figure 2) and pull the armrest up with the other hand (position 2 in Figure 2) until the electric wheelchair is fully folded.

Remarks: Before folding the electric wheelchair, empty the items in the storage basket to prevent damage.

Attention! During the folding/Unfolding procedure, do not contact the points indicated in the Figure 3.

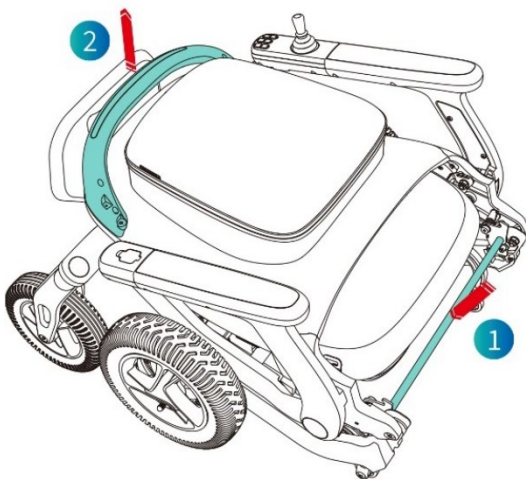
Figure 1

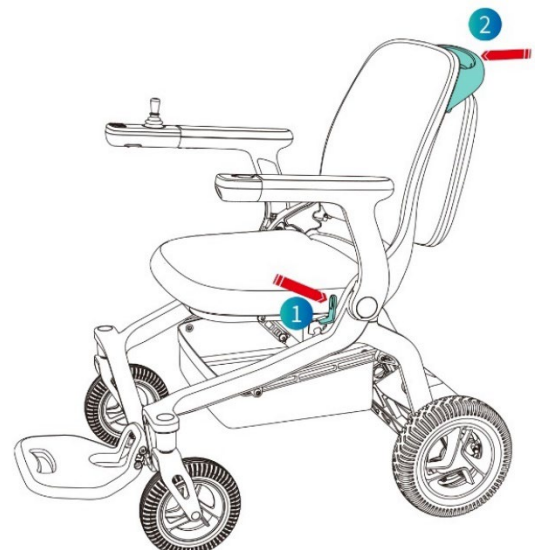
Figure 2

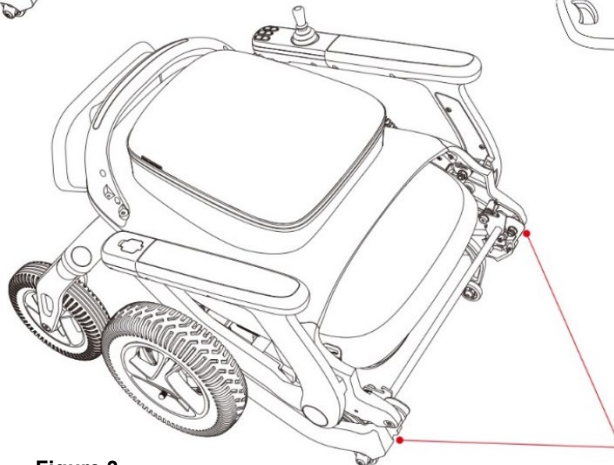
Figure 3

3.3 Controller with Joystick

Controller Introduction:

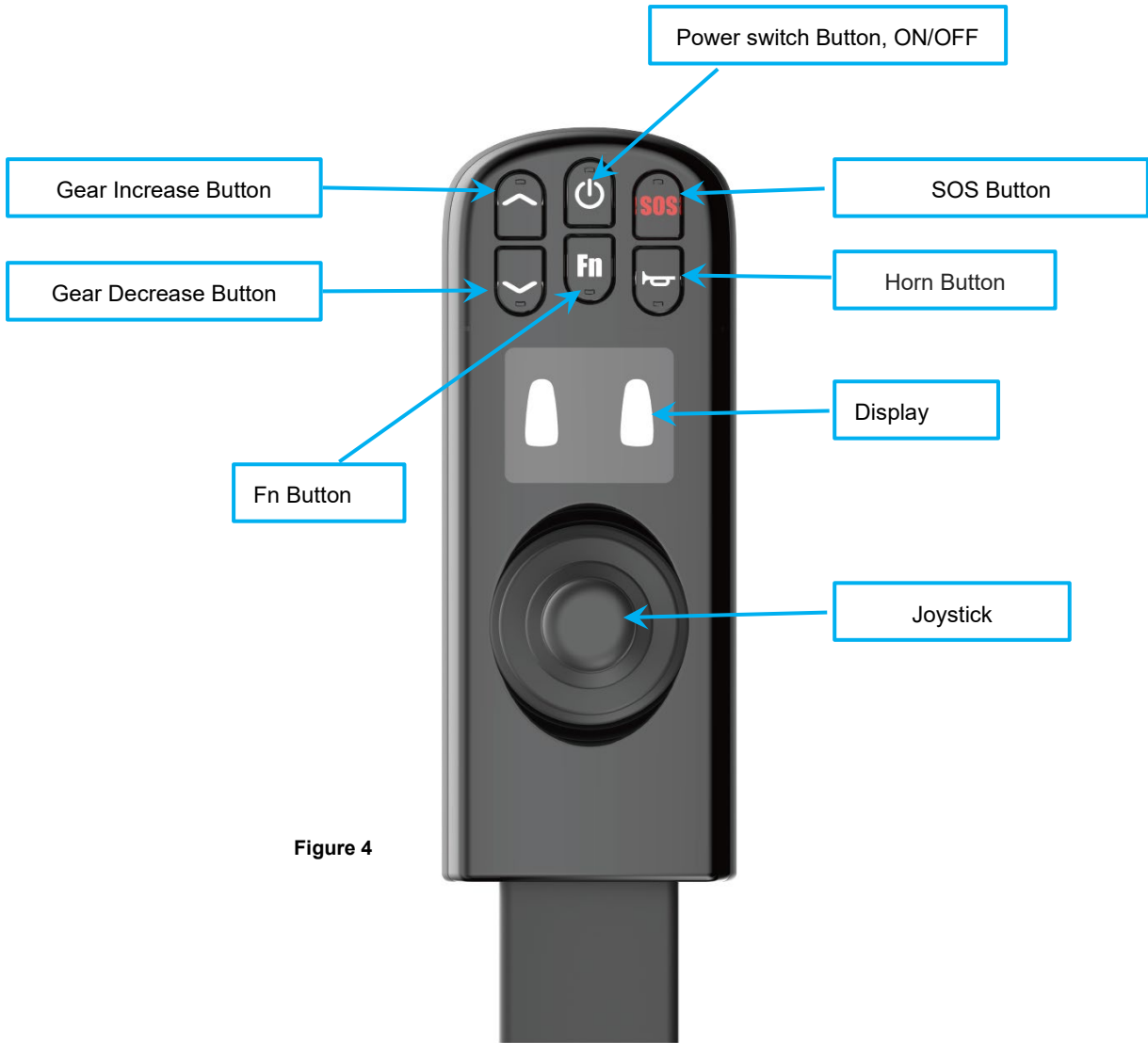
Figure 4

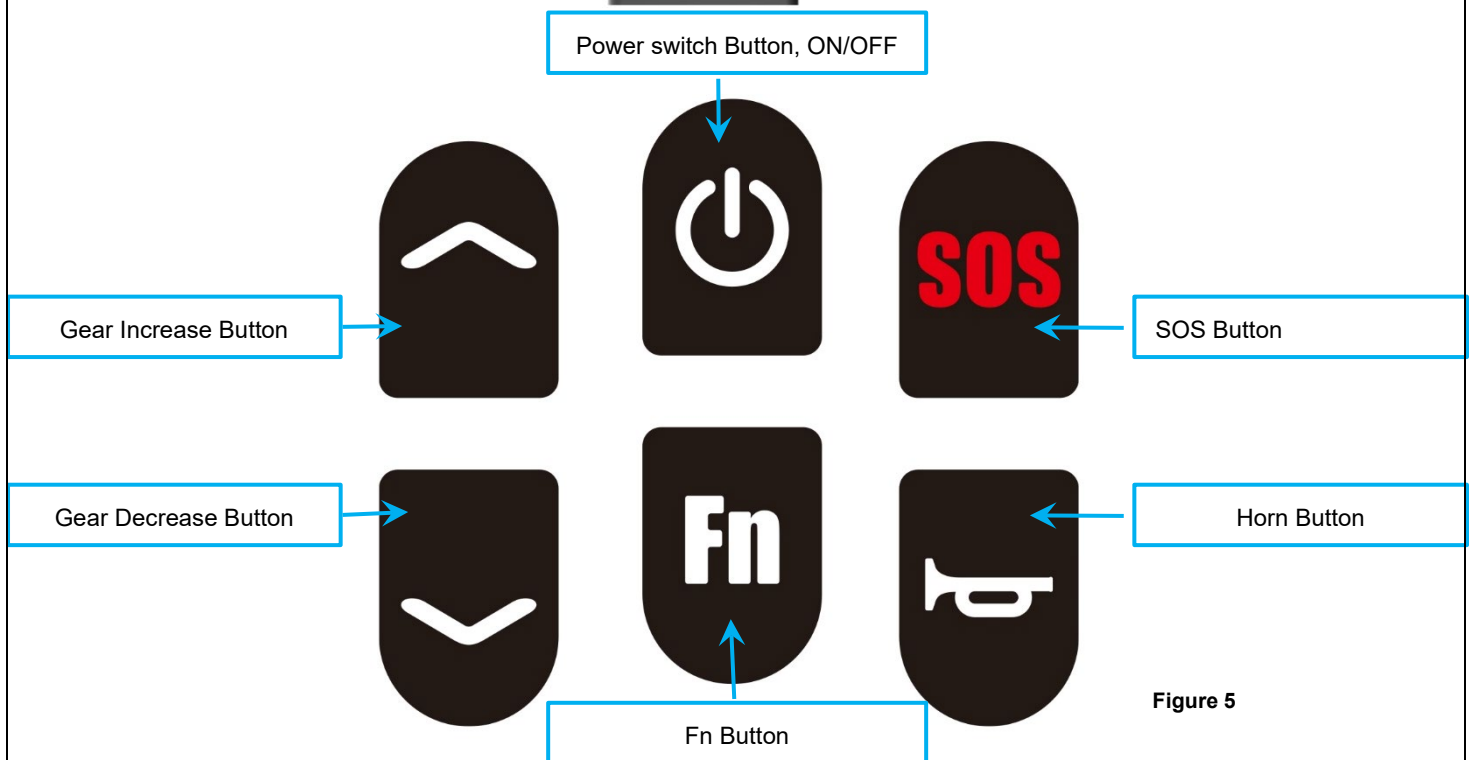
Figure 5

Controller Instructions:

Joystick

By pushing the controller Joystick, the direction and speed of the Electric wheelchair can be controlled.

Power switch Button.

Long press for 3 seconds to turn on the power.

Long press for 3 seconds to turn off the power.

Speed “^” Button.

When it is pressed, a higher gear is selected. (Up to 5 gears)

Speed “v” Button.

When it is pressed, a lower gear is selected. (Minimum 1 gear);

Horn Button.

Keep a short press of the hone key, the horn will ring for once.

SOS Button.

Press the button for 3 seconds. The system activates the audible signal.

Fn Button

Press Fn to enter the “novice mode”, and on the screen will display “gear 1”, “gear 2” and the “NOVICE” icon, accompanied by the voice prompt of "Enter the novice mode". You cannot select a higher gear than 2nd . Available grears for option are only the 1st & 2nd.

Display screen menu Instructions:

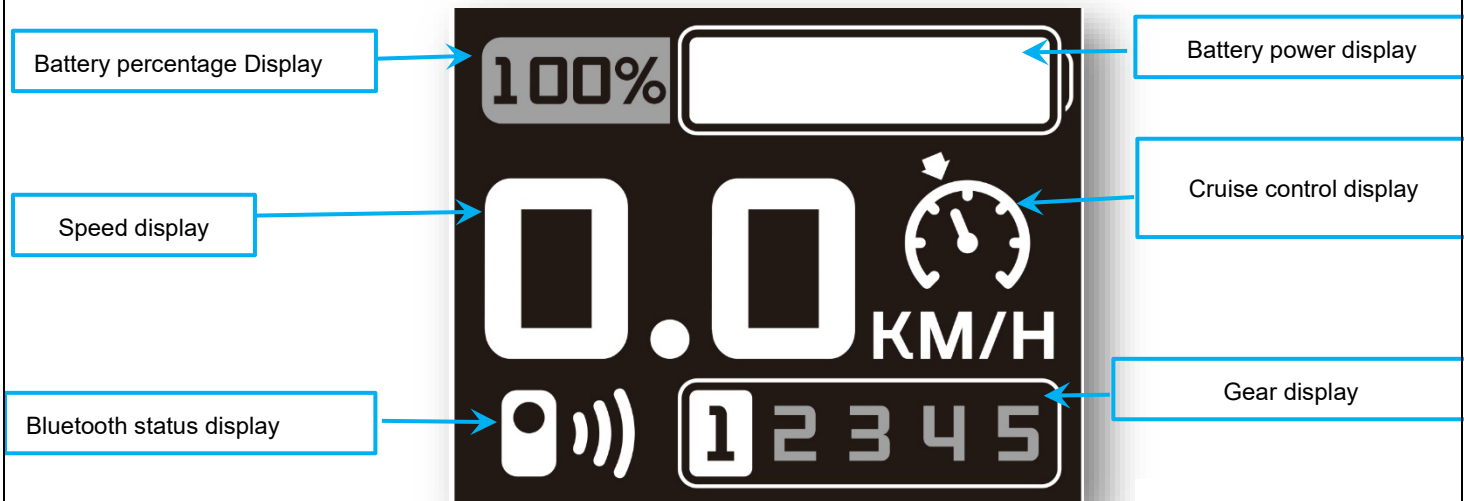
Figure 6

Battery percentage display

The upper controller screen of electric wheelchair displays the battery in %.

Speed display.

The screen displays the driving speed of the Electric wheelchair in km/h.

The precision of the displayed speed is $\pm 5\%$.

Bluetooth status display

When Bluetooth is not connected, the Bluetooth indicator on the screen is off.

When Bluetooth is connected, the Bluetooth indicator on the screen is on.

Battery power display

The upper controller displays the power state in real time.

Cruise control display

Press and hold the "Speed \wedge " button for more than 3s to enter the cruise control mode. The electric wheelchair upper controller screen displays the cruise control icon, and the voice prompts two times. Push the joystick forward to start the cruise control.

Press the key or move the joystick backwards to exit the cruise control (regardless of driving state).

The cruise control function can only be triggered when non-locked mode.

The buttons " \wedge " and " \vee " can be used to adjust the gears during the cruise control process.

Use the joystick to control the turn during operation.

After entering the cruise control mode, if it is not started within 5s, the mode will be automatically exited, and the voice prompt "With wait timeout, the cruise control mode has been automatically exited" will be announced.

Gear display

Press the "▲" button once to select the next higher gear. The 5th gear is the highest.
Press the "▼" button once to select the next lower gear. The 1st gear is the lowest.

The above functions can also be activated on the APP.

On display shows the selected gear.

Novice mode display

Click "Fn" to enter the novice mode, and the screen will display "gear 1", "gear 2" and the "NOVICE" icon, accompanied by the voice prompt of "Enter the novice mode". Speed gear limit 2 (the first and second gears are available).

Display of fault code

For the content of the codes displayed by the product's built-in diagnostic test system, see the section, "Troubleshooting".

Figure 7


Figure 8

Figure 9

SOS display

Long press SOS button for 3 seconds to start SOS signal. The screen displays SOS icon and flashes.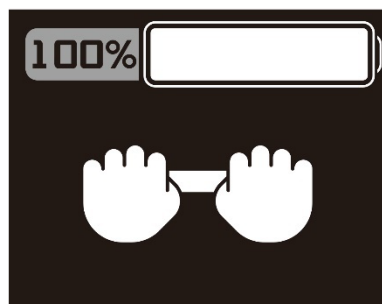

Pushing mode display

At the same time, press and hold the button "Speed ▲" and "Speed ▼" for 3s, and the pushing mode icon will be displayed on the upper controller screen of electric wheelchair.

Figure 10

Battery charging status display

After the charging port of the electric wheelchair is connected to the charger, the upper controller screen will display "CHARGING" icon, meaning that the electric wheelchair is being charged.

After the electric wheelchair is fully charged, the upper controller screen will display "FULL CHARGED" icon, meaning that the electric wheelchair has been charged.

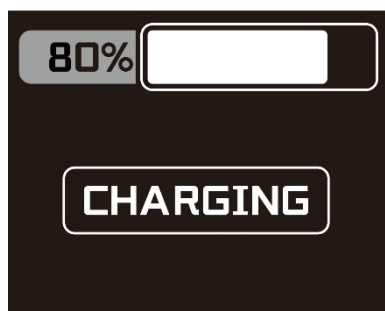
Figure 11

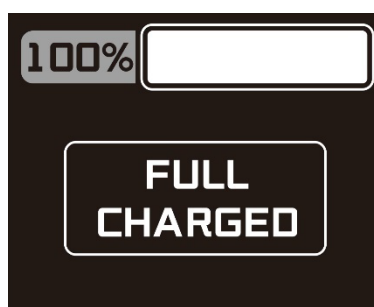
Figure 12

3.4 Battery

Battery disassembly:

Step 1: Manually rotate the left knob of the battery case to the unlocking position. Then open the battery left buckle.

Step 2: Manually rotate the right knob of the battery case to the unlocking position. Then open the battery right buckle.

Step 3: Remove the battery in the direction shown in the figure.

Figure 13

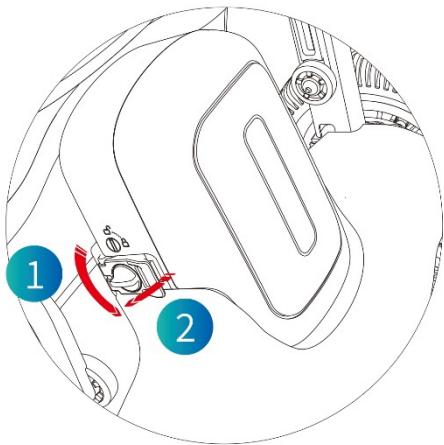
Figure 14

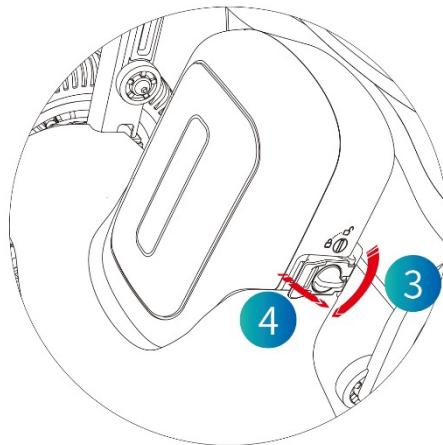
Figure 15

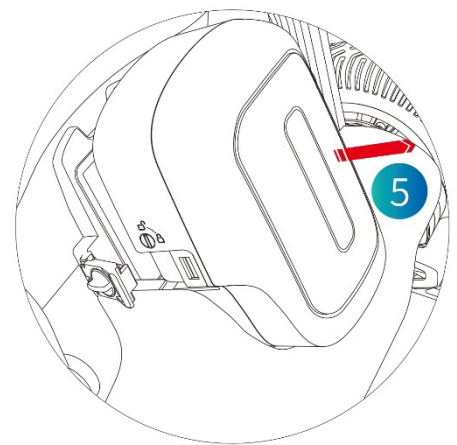
Figure 16

Battery Installation:

Step 1: Place the battery in the cell slot in the direction shown.

Step 2: Manually close the battery right buckle and rotate the right knob to the locked position.

Step 3: Manually close the battery left buckle and rotate the left knob to the locked position.

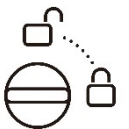
Figure 17

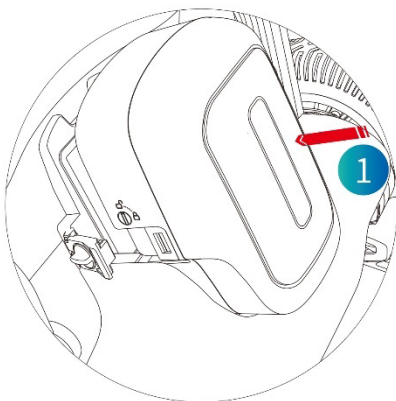
Figure 18

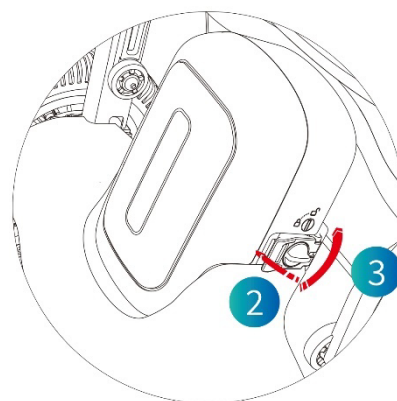
Figure 19


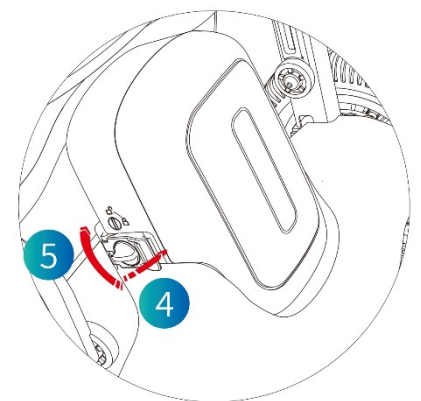
Figure 20

3.5 Switch to pushing mode.


Press “▲” key and “▼” button at the same time to enter pushing mode. You can quit from this mode by wing the joystick.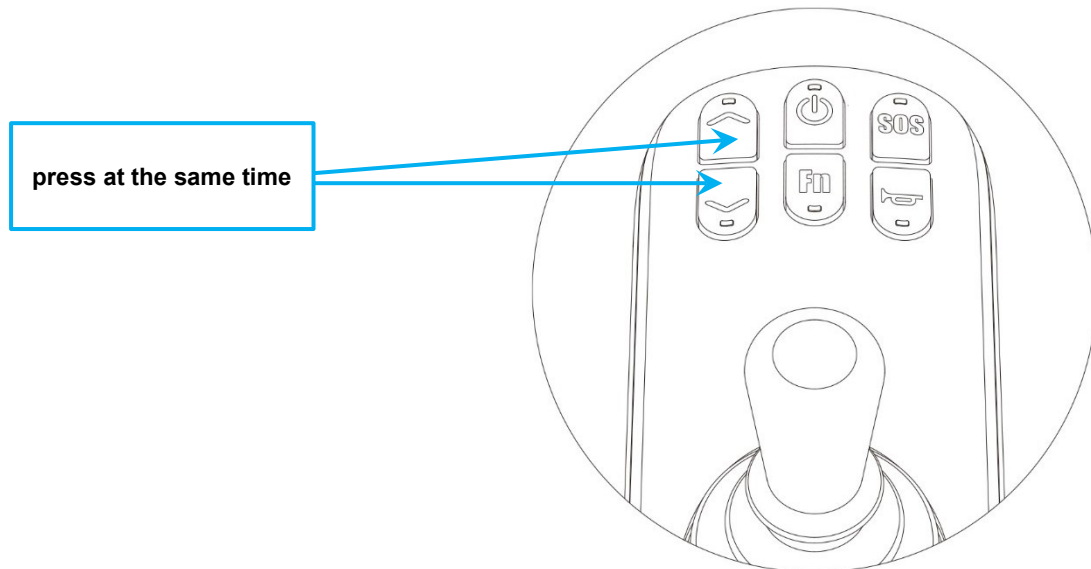


3.6 Charging of Electric wheelchair.

Charging method I: Plug the charger into the socket located under the control system firstly and then plug it into the power socket on the wall.

Charging method II: Remove the battery case and plug the charger into the charging socket on the case and then plug it into the power socket on the wall.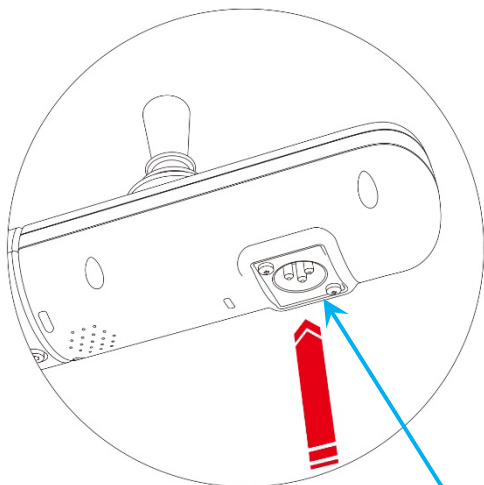

Charging socket under the control system.

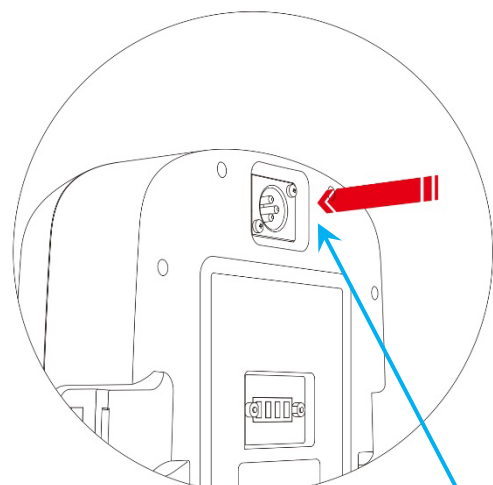Charging socket under the battery.

4. Maintenance and troubleshooting

4.1 Daily spot inspection and maintenance scheme

- Regularly (weekly or monthly, depending on the use frequency) check the connecting parts, such as screws and nuts, etc. of the frame for loosening, falling and corrosion, etc.
- Regularly check the movable joint parts for jam and falling, etc.
- Regularly check the tires for aging and serious wear, etc.
- Check the wheel pressure, If there are pneumatic wheels on the wheelchair.
- Clean the metal parts of the product with a dry cloth.
- During use of the Electric wheelchair, in case of rain or wet weather, the Electric wheelchair shall be wiped dry as soon as possible to prevent rust due to dampness.
- Exposure of the electric wheelchair to solar radiation under high temperature conditions should be avoided.
- We recommend a full review once a year to a qualified service provider.
- In case of repair, use only authentic spare parts.

4.1 Codes Fault Table

When contacting our technical support department at services@mobiak.com or our call center for a fault report or repair, please quote the error code.

Thanks to the built-in diagnostic control system of the device, the displayed error code may correspond to a problem during operation of the wheelchair.

Fault code	Situation	Description
E1	Key abnormality	The key corresponding function does not work.
20	The rocker ground line is disconnected	The rocker function does not work.
21	The rocker power cord is disconnected	The rocker function does not work.
22	The rocker X-axis is disconnected	The rocker function does not work.
23	The rocker Y-axis is disconnected	The rocker function does not work.
24	The rocker is a non-zero start	The rocker function does not work.
E6	CAN communication is abnormal	The rocker cannot control the driving of the electric wheelchair.
E7	FLASH fault	FLASH device ID cannot be detected, and the device cannot work.

F1	Left motor Hall fault	The left motor Hall is not connected or faulty, and the motor does not function.
F2	Right motor Hall fault	The right motor Hall is not connected or faulty, and the motor does not function.
F3	Flat in the electromagnetic brake of left motor	To release the electromagnetic brake handle of left motor, close the electromagnetic brake.
F4	Flat in the electromagnetic brake of right motor	To release the electromagnetic brake handle of right motor, close the electromagnetic brake.
F5	Low-voltage fault	The motor cannot run.
F6	Pre-charge fault	The motor cannot run.
84	Short-circuit fault of lower gear MOS in the right motor	The motor cannot run.
85	Short-circuit fault of upper gear MOS in the right motor	The motor cannot run.
86	Short-circuit fault of upper gear MOS in the left motor	The motor cannot run.
87	Short-circuit fault of lower gear MOS in the left motor	The motor cannot run.

4.2 Safety warnings.

The device has the following audio voices.

Warning voice	Corresponding scene description.
Enter novice mode	Press the button for novice mode
Quit novice mode	Press the button for novice mode again
Alarm	Long press of SOS button
LOCK mode	It is in default that: the product is locked after it is powered on. Switch non-locking status to locking status.
Unlock	Switch locking status to non-locking status
Start remote control	Voice prompt when the remote control or APP Bluetooth is connected successfully.
Bluetooth has been locked	Press the lock key.
Upgrade mode	When the APP detects new version, click to upgrade
Upgrade completed	End of the single-module upgrade.
Upgrading	The equipment is upgrading
Powering on tone	Press and hold the power-on key for 3s to start the power-on and the power-on sound.
Key tone	Touch the button to produce a prompt sound.
Charging	Charging

Charging completed	Unplug the charger and trigger the voice prompt of charging completed.
Band-type brake open	The band-type brake opens.
Bluetooth connected	Voice prompt when the remote control or APP Bluetooth is connected successfully.
Drive protection	Voice prompt for a drive failure.
Machine protection	Voice prompt for a machine failure.
Rocker protection	Voice prompt for a joystick failure.
Power off tone	Press and hold the power-off key for 3s to start the power-off and the power-off sound.
Bluetooth disconnected	Voice prompt when the remote control or APP Bluetooth is disconnected.
Upgrade failed	There is something wrong during upgrading.
The horn sounds	Touch the horn button.
Please charge for a low power	Trigger when the power is lower than 20%.
Bluetooth unlocking	Press the remote control to unlock.
Enter pushing mode	Long press "~" button and "<" button 3 seconds to enter the pushing mode, accompanied by the pushing mode prompt.
Quit pushing mode	Shake the rocker to quit the pushing mode.

4.3 Cleaning the product.

The user or the attendant of the wheelchair is responsible for the cleaning, inspection and maintenance.

In addition to maintenance and inspection, a general cleaning for the frame and the upholstery can be carried out.

DO NOT leave the wheelchair wet or dirty with dust or mud. The frame should be cleaned regularly.

Use a clean and soft cloth to wipe the surface of the electric wheelchair.

5. Environmental Protection Statement:

If one day you find that your product needs to be replaced or it is no longer working for you, consider protecting the environment:

- 1) Do not dispose your product along with the rest of the public waste (this is also the meaning of the shown recycling sign).
- 2) Contact your Public Authorities and they will instruct you of the Recycling centers to which your product must be disposed.
- 3) Correct disposal of your product helps the protection of the environment as well as the recycling of the product's components.
- 4) The electrical products due to their construction materials, if not correctly disposed, may lead to environmental and furthermore health hazards.

6. Incident Reporting:

Any serious incident that occurs in relation to use of the device must be reported to the manufacturer and a competent authority of the Member State in which the user and / or patient resides.

7. Declaration Of Conformity:

We are solely responsible for declaring that the Medical Devices mentioned in this statement are of Low-Risk Class (Class I) and comply with the requirements of the European Regulation 745/2017 and where appropriate, the standards and legislation referred to.

8. Terms of Warranty:

The product is guaranteed for a period of two (2) years warranty on the frame and six (6) months the battery, from the purchasing date.

The warranty apply on factory defects & DOES NOT cover damages resulting from misuse, poor maintenance, modification, overuse, or non-compliance with the product's operating instructions. Also DOES NOT cover the parts that can wear out during use or over time, like the upholstery, wheels, wheel tire, safety belt, footrests or brakes.

Damages or defects caused by: natural disasters, unauthorized maintenance or repair, power supply problems (where applicable) & carriers are not covered by the warranty.

No service or spare part is covered unless it has been inspected by an authorized service department or the manufacturer of the product. Shipping costs for goods of warranty are to be paid by the user-customer or by dealer.

When the user or the provider apply on warranty request, provide a valid proof of purchase and the completed warranty form, with a photo of the product serial number and a photo or video of the problem that has occurred. In the event the user cannot provide the above documents, if we can confirm that the product or part is still within the warranty period, we may be able to honor the user's request for coverage under warranty, but we are not obligated to do so.

Warranty Form

BUYERS DETAILS:			
FULL NAME:			
PHONE:			
PURCHASING DATE:		Invoice Number:	
SERIAL NUMBER:			
LOT:			
DEALER DETAILS:			
FULL NAME:			
ADDRESS:			
PHONE:			
			DEALER SIGN AND SIGNATURE

MOBIAK S.A
ΚΑΘΙΑΝΑ ΑΚΡΩΤΗΡΙΟΥ, ΧΑΝΙΑ - ΚΡΗΤΗ
KATHIANA AKROTIRIOU CHANIA - CRETE
T: +30 28210 63222 | WEB: www.mobiakcare.com

9. Disclaimer:

The content of the product manual is the intellectual property of MOBIAK S.A. (hereinafter referred to as the "Company") and the copyright is reserved by the Company. Without the written consent of the Company, no one may copy, export or translate the manual or any content included in any form into other languages without the written consent of the Company. The Company carefully compiles the product manual with a sense of responsibility for users but does not guarantee the completeness or correctness of its contents. The product manual, as a purely technical document, does not include implied content or content with oblique reference to third parties. The Company assumes no responsibility for any typographical errors that may cause ambiguities. In the event of direct or indirect loss of information or business interruption due to the product manual or the product information contained therein, the Company shall not be liable.

When using all MOBIAK S.A. products, users must strictly follow the product manual and the instructions for use available on the Company's official website (www.mobiakcare.com) for the handling of the equipment and not violate any warning marked in the product manual when using the equipment. In case of injury to users or other persons, or material damage caused by the handling of the equipment without following the instructions for use or by violating the warnings marked in the product manual, the persons concerned shall bear the respective legal responsibilities.

As the Company continues to improve and update its products, it is essential that you regularly log on to its official website to refer to the most up-to-date instructions and conditions of use or contact your distributor for the latest user manual. The Company reserves the right to revise the product manual without notice.

The Company reserves the right of final interpretation.

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