EL EN

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PRODUCT DESCRIPTION

Air cushions can reduce & cure bed ulcers in chronic diseased patients or for people who sit on wheelchair or armchairs for long periods. They can be used for home care use. They distribute evenly pressure on the body to prevent edema.

The air pump alternates the air pressure on the cushion with a feeling of light massage & activates immobilized cells due to the prolonged sitting.

This product is indicated for patients who spend more than 15 hours a day in a chair, armchair or wheelchair.

OPERATION USE

For air cushion:

- ☐ Check for sharp objects on the patient's seat (wheelchair, armchair, chair, etc.), and then place the air cushion on. Make sure the air inlet pipes are on the same side of the pump and are not folded.
- ☐ Attach the air cushion to the pump by using the hoses in the package and install correctly.

For the pump


- ☐ Use the hooks behind the pump to hang the pump onto the wheelchair armrest. Connect the pump to the nearest wall outlet (not an extension cord) and adjust the air flow according the weight of the patient with the rotary selector on the top of the device.

SPARE PARTS

REF. NUMBER	DESCRIPTION
0806297	STRIPED AIR CUSHION
0806272	PUMP FOR ANTI-DECUBITUS BUBBLE MATTRESS MINOA
0806299	PUMP WITH BATTERY

DESCRIPTION OF INDICATORS AND FUNCTIONS FOR THE AIR PUMP

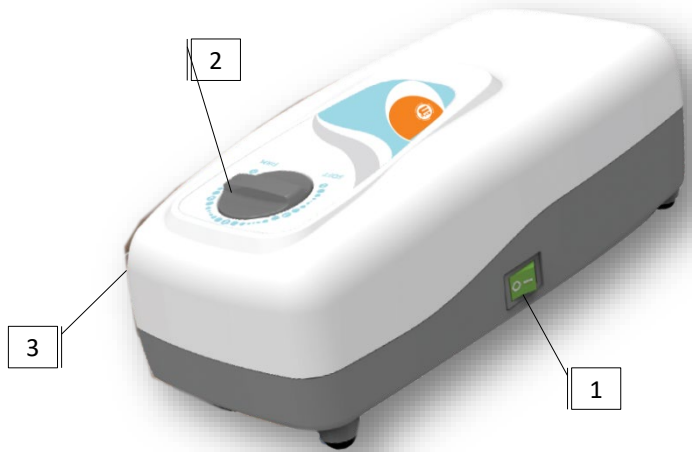
1. Power switch
2. Rotary pressure switch.
3. Air outlets

PUMP	Voltage (V) - Frequency (Hz)	AC220-240V/ 50HZ
	Air Output	≥ 5-6 lit/min
	Pressure Range	50mmHg – 110mmHg
	Cycle Time	12min
	Power Consumption	≤ 7 W ± 10%
	Noise	≤ 30 dB ± 10%
	Dimensions (LxWxH)	22,5 x 10,5 x 8,1 cm
	Pump Weight	1 Kg

AIR CUSHION	Max. patient weight (Kg)	120 Kg
	Color	Blue
	Material	Nylon TPU & Stretch PU for the cover
	Cushion height (cm)	10 cm
	Dimension (LxWxH)	43 x 43 x 10 cm
	Air tubing length (cm)	75 cm
	Weight (Kg)	2 Kg

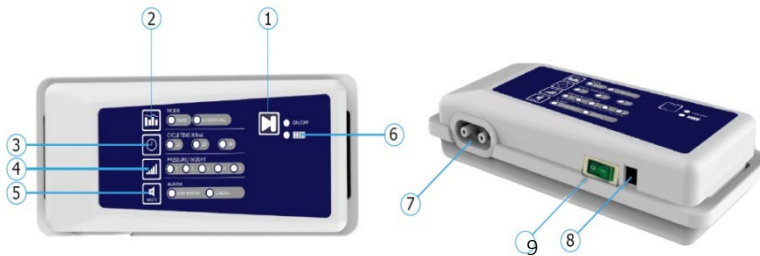DESCRIPTION OF INDICATORS AND FUNCTIONS FOR THE BATTERY PUMP

1. Start button (press for two seconds)
2. Operation mode selection key
3. Circle time selection key
4. Pressure adjustment knob
5. "Mute" key
6. Battery charging indicator
7. Air outlets
8. Charger input
9. Power switch

AIR PUMP	Voltage (V) – Frequency (Hz) -	100-220 VAC/ 50Hz - Li-Battery 11.1V 2200 mAh
	Pump autonomy (h)	12 hours (Pressure level 1)
	Air outlet (lit/min)	> 4 liters / min
	Pressure range (mmHg)	30 ~ 70 mmHg
	Time cycle (min)	10 / 15 / 20 min
	Noise (dB)	≤ 30 dB ± 10%
	Dimensions (LxWxH)	34,5 x 11 x 5,5 cm
	Pump weight (Kg)	1,111 Kg

The air cushion pump has two functions: **Alternative or Static.**

ALTERNATIVE:

In ALTERNATING mode, the tubes inflate and deflate, and change the patient's contact area.

STATIC:

In STATIC mode, all tubes are inflated simultaneously to give the patient uniform support throughout the contact area.

• Switching time selection.

When ALTERNATING is selected, the user can choose whether they want the tubes to rotate faster or slower during the time of an alternating cycle. There are 3 selection cycles such as 10 minutes, 15 minutes or 20 minutes.

• Pressure level adjustment

The pressure level can be adjusted from soft (1) to hard (5). The user can select the most comfortable pressure by using the pressure adjustment key.

(The duration of each step depends on battery capacity)

- MUTE

If the alarm is on, the pump will make BEEP sounds to alert the user for the alarm.

However, the user can mute the BEEP by pressing the mute button.

The Mute function will be deactivated by pressing the mute button again (5).

- Charging the battery

Step 1-> Connect the power plug to the power input.

Step 2-> Connect the power adapter to an A / C wall outlet.

Step 3-> LED 6 lights up and starts charging.

Step 4-> During the charging of the battery a light indicator is on.

Step 5-> the light indicator turns off when the battery is fully charged.

Step 6-> when the battery is fully charged, make sure you unplug the power adapter from the pump before unplugging it from the A / C socket.

(Disconnecting in reverse order can cause serious damage to the pump).

NOTICE

1. Make sure you have fully charged the battery before the first use.

2. Do not use the battery when the capacity is too low.

(LED LOW BATTERY flashes red).

3. The pillow will not work if the battery is not properly inserted.

4. It is normal if the battery heats up after charging or use.

5. It is highly recommended to use only the batteries provided by the pump manufacturer.

Using other batteries can cause serious unforeseen risks.

STORAGE

- Switch off the device, unplug it from the power supply and disconnect the air-cushion from the pump.
- Place the air-cushion on a flat surface and roll or fold it from top to bottom.
- Allow air slowly escape the cushion to reduce storage space
- Close the protective cap cover on the quick connector.
- Place air-cushion & the pump in the carrying bag (if available) and store it in a moisture-free place.

IMPORTANT (!)

In case of damage or malfunction of the pump, please contact your authorized and trained agent.

To clean the cushion use a soft cloth or sponge & water with mild soap. ATTENTION:

- ☐ Do not use acidic & caustic dilutions.
- ☐ Do not smoke & do not use any flame near the cushion.
- ☐ Do not have the cushion in contact with direct heat sources.
- ☐ If you need to save the cushion, empty of air, by folding it
- ☐ Keep or store the pump in a clean, dry and well ventilated environment.
- ☐ Before you clean the pump, disconnect from the power supply.
- ☐ Clean the pump with a dry cloth.

ATTENTION:

Do not bring the pump in contact with liquids.

IMPORTANT (!)

CAUTIONS

- ☐ Do not disconnect the hoses while the pump is on.
 - ☐ Do not use sharp objects near the cushion.
 - ☐ Do not exceed the maximum load (kg) recommended by the manufacturer.
 - ☐ Do not use, while bathing the patient.
 - ☐ Protect the product by oils, lotions and creams, by covering it with the cotton wool fabric, or a bath towel.
 - ☐ Do not use alcohol-based cleaning products to clean the cushion. This could damage the material of the layer.
 - ☐ Clean the upholstery after each daily use.
- We recommend to clean the upholstery with a mild detergent to protect it from oil stains and dust. A common practice is to use a spray bottle with a mixture of anti-bacterial soap and water. Spray the surface and then wipe with a clean towel. The best solution is to be cleaned with warm water mixed with a mild soap and then thoroughly dried.
- ☐ Protect the layer from scratches, cuts and punctures.
 - ☐ Do not leave the product on, if it is not in a use.

CAUTION (!)

Do not use the product for a purpose not indicated in this manual

Only qualified person or trained for the treatment and prevention of bedsores, can operate with the device

MOBIAC S.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the product.

MOBIAC S.A reserves the right to change the information contained in this document without previous notice.

EC Declaration of Conformity

We are solely responsible for declaring that the Medical Devices mentioned in this statement are of Low-Risk Class (Class I) and comply with the requirements of the European Regulation 745/2017 and where appropriate, the standards and legislation referred to.

ENVIRONMENTAL PROTECTION

If one day you find that your product needs to be replaced or it is no longer working for you, consider protecting the environment:

- 1) Do not dispose your product along with the rest of the public waste (this is also the meaning of the shown recycling sign).
- 2) Contact your Public Authorities and they will instruct you of the Recycling centers to which your product must be disposed.
- 3) Correct disposal of your product helps the protection of the environment as well as the recycling of the product's components.
- 4) The electrical products due to their construction materials, if not correctly disposed, may lead to environmental and furthermore health hazards

NORTON SCALE DETAILS

The Norton Scale was developed in the 1960s and is widely used to assess the risk for pressure ulcer in adult patients. The five subscale scores of the Norton Scale are added together for a total score that ranges from 5-20. A lower Norton score indicates higher levels of risk for pressure ulcer development.

SCORE	RISK FOR PRESSURE ULCER DEVELOPMENT
>18	LOW
18-14	MEDIUM
14-10	HIGH
< 10	VERY HIGH

Please be noted that the Norton Scale is a tool only to be used by trained healthcare professionals, and only as a supplement to a thorough patient state evaluation.

MBK CODE	SUITABLE FOR NORTON SCALE SCORE
0806296 0806246	19 – Low Risk

WARRANTY

The product has warranty for two (2) years in pump, six (6) months on the cushion and three (3) months for the battery from purchase date.

The warranty covers factory defects and not damages resulting from improper use / maintenance of the product.

The warranty apply factory defects & DOES NOT apply damages resulting from improper use , poor maintenance, modification, abuse or non-compliance with the product's operating instructions.

Damage or defects caused by natural disasters, unauthorized maintenance or repair, power supply problems (where applicable) and transportation companies are not covered by the warranty.

Warranty Form

Buyers Data
Name:
Date of purchase:S/N.....
LOT:..... Phone:

Resellers Data
Name :
.....
Address:.....
Phone:

Reseller Stamp

INCIDENT REPORTING:

Any serious incident that occurs in relation to use of the device must be reported to the manufacturer and a competent authority of the Member State in which the user and / or patient resides.

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Description of symbols that have been printed on the label, Master Carton and or User Manual

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