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**INTENDED USE:**

The Monkey Pole is mainly intended for people who are bedridden and need help to get up from a lying position in a sitting position and reverse. It is an accessory of the bed which is offered separately, it is optional and is intended for indoor use only. No additional accessories are required to attach the product on the wall or bed. It is not designed to support the total body weight of the patient. It is not designed to move a person in and out of bed, it should be use for assistance only.

**CAUTION (!):**

- In case of damage or malfunction of your product DOT NOT use the product and please contact the authorized and trained representative.
- Do not use the product for a purpose not indicated in this manual.
- MOBIAS S.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration on the frame of the product.
- MOBIAS S.A reserves the right to change the information contained in this document without previous notice.
- There is a  $\pm 3\%$ . Deviation for the Dimensions of the Products.
- It is always recommended to follow the procedures designed for emergency response.
- Improper use of the product can cause injury to the patient.
- Always consult your doctor.

**GENERAL WARNINGS:**

- If you don't read this user manual, it is preferable not to use this product or another available part.
- If you don't understand the usage or the precautions, please contact with the dealer or the proper technical person before you use the product because may be caused damage.
- READ carefully the entire manual before using the product.
- Save this document for future reference.
- The product should be used only by qualified personnel.
- Keep away the product from heat sources.
- Be careful when children are nearby and do not allow the children to play or operate with the product.
- Special care is always required where there are moving parts that can cause limb entrapment or injury.

**BEFORE EVERY USE:**

1. Check if the frame is damaged in order to guarantee a safe use of the product (there are no cracks or fractures in the frame).
2. In case of damage do not use the product and contact your dealer for further instructions.
3. Check, if the product is properly assembled and all screws are secure and well screwed on.
4. Always check the condition of wear for the mechanical parts to ensure the absolute safe use of the product for people and objects.
5. Periodically check its sturdiness and the stability.
6. Make sure that the floor surface that will support product is flat.
7. Always assist the user during the operation of the product

**CLEANING AND DISINFECTION:**

- Use only a wet cloth and mild soap on surfaces where they need cleaning. Then dry thoroughly before you use the product.
- Dry away from direct sunlight.
- DO NOT leave the product wet.
- Wipe with a dry, clean, soft cloth.
- Do not use chemical cleaners to clean the frame. This could cause damage to the surface of the product.
- Never use acids, alkalis or solvents such as acetone
- If you need to disinfect the product, use a common, mild disinfectant.
- Protect the item from scratches, cuts and punctures.

**ENVIRONMENTAL PROTECTION:**

If one day you find that your product needs to be replaced or it is no longer working for you, consider protecting the environment:

- 1) Do not dispose your product along with the rest of the public waste (this is also the meaning of the shown recycling sign).
- 2) Contact your Public Authorities and they will instruct you of the Recycling centers to which your product must be disposed.
- 3) Correct disposal of your product helps the protection of the environment as well as the recycling of the product's components.

**INCIDENT REPORTING:**

Any serious incident that occurs in relation to use of the product must be reported to the manufacturer and a competent authority of the Member State in which the user and / or patient resides.

**WARRANTY:**

The product is guaranteed for a period of two (2) years from the purchasing date.

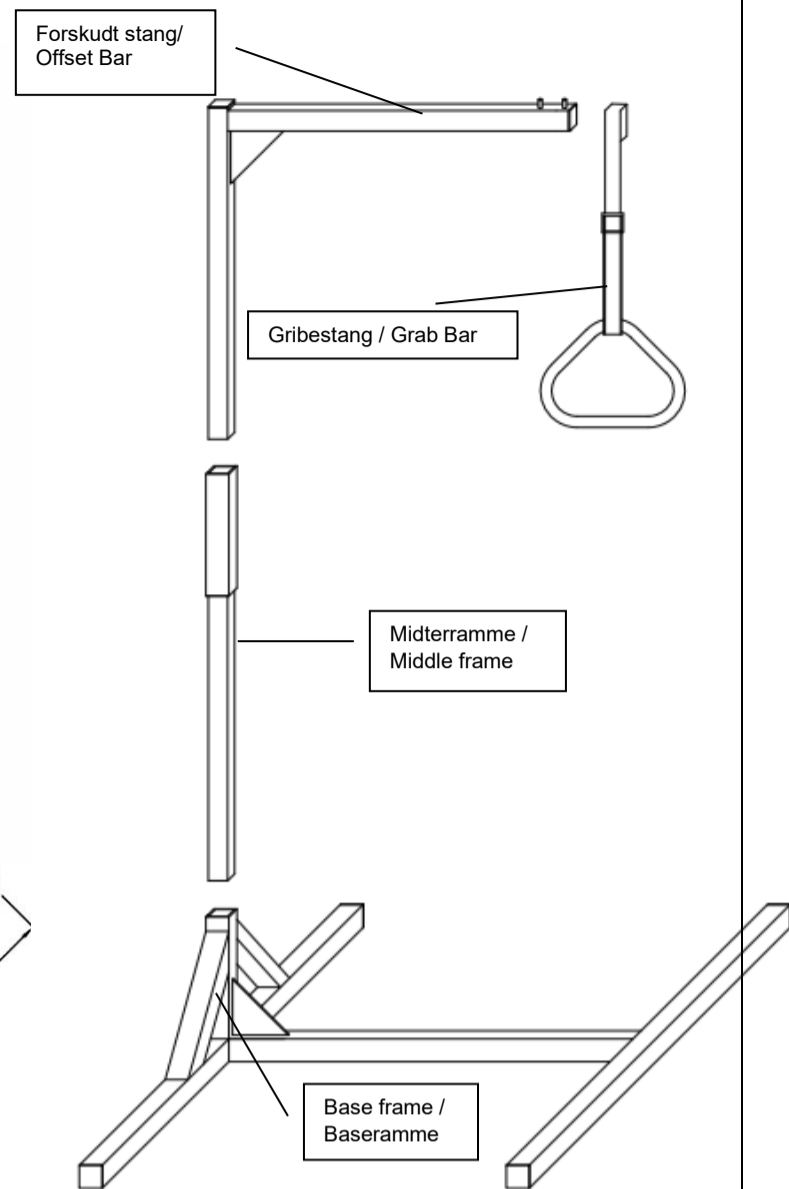
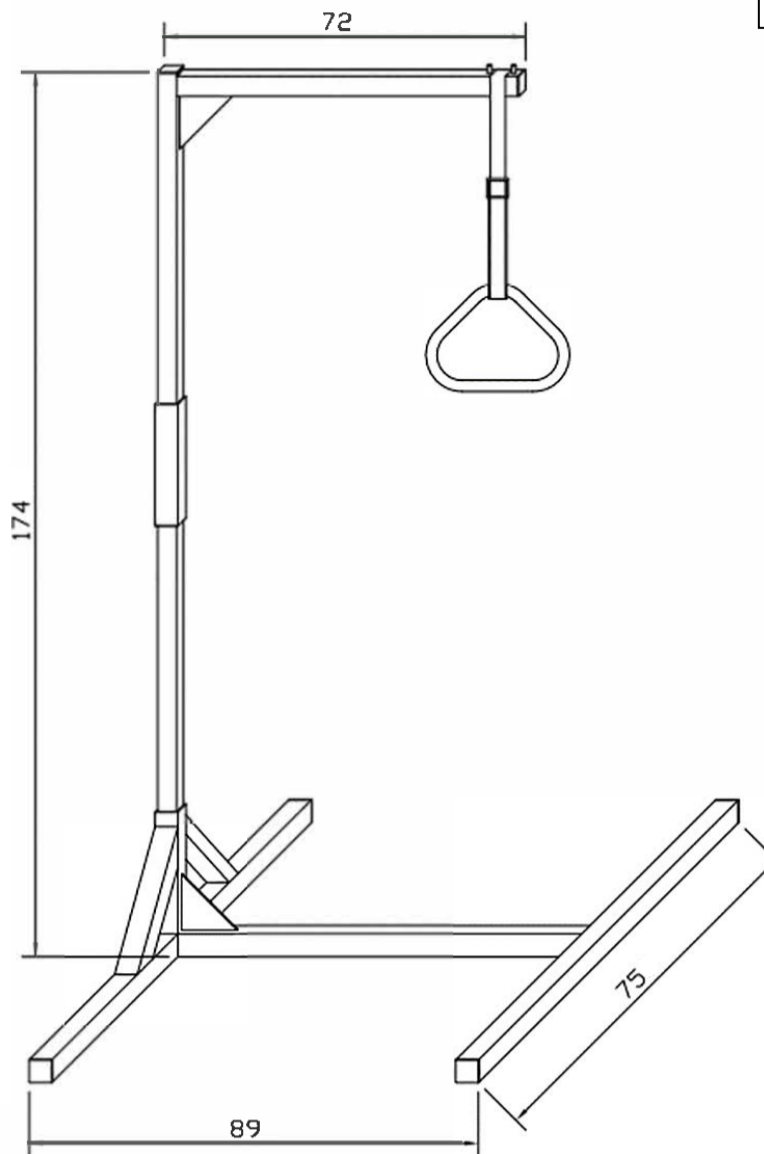
The warranty apply on factory defects & DOES NOT apply on damages resulting from misuse, poor maintenance, modification, overuse, or non-compliance with the product's operating instructions.

Also DOES NOT apply on the parts, that can wear out during use or over time. Damages or defects caused by natural disasters, unauthorized maintenance or repair, power supply problems (where applicable) & carriers are not covered by the warranty.

No service or spare part is applied unless it has been inspected by an authorized service department or the manufacturer of the product. Shipping costs for goods of warranty are to be paid by the end user- customer or by dealer.

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**MOBIAK**<sup>®</sup>  
MEDICAL • GASES • FIREFIGHTING est. 1977

BRUGSANVISNING: LØFTEGALGE MED BASE II  
USER MANUAL: LIFTING POLE WITH BASE II

**REF** : 0810650

MBK CODE / KODE	GTIN-14	DA BESKRIVELSE	EN DESCRIPTION
0810650	05213006907289	LØFTEGALGE MED BASE II	LIFTING POLE WITH BASE II

## IMPORTED BY

MOBIAK S.A  
KAGIANA AKROTIRIOY, XANIA - KPHH  
KATHIANA AKROTIRIOU CHANIA - CRETE  
T: +30 28210 63222 | WEB: www.mobiakcare.com