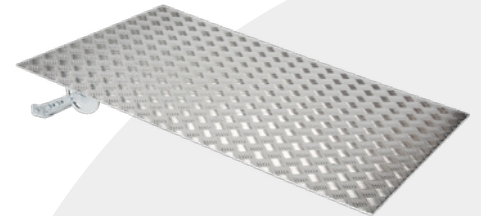
Mobilex "Terrace" T80 ramps Art. T80060, T80075 & T80125

Montage- und Gebrauchsanleitung
Montage- og brugervejledning
User Manual
Manuale d'uso e manutenzione

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1. Introduction

This user manual contains a description of the medical device and important guidances to ensure a correct and safe usage of the product. It is important prior to use to read this manual carefully. It is especially important to read the safety requirements and follow these.

We continuously develop our products and we reserve the right without further notice to change the specifications and functions of products.

Intended use of the device

The Mobilex T ramps are medical devices which is intended to be used only to assist patients in wheelchairs or scooters. They enable the passage of elevated points (e.g. Stairs) with or without the assistance of a helper. Mobilex Ramps thereby enables patients, sitting in a wheelchair because of invalidity, to enter or leave buildings or cars, buses etc. The ramps can be used as well inside as outside. Any other use is prohibited

By questions please take contact to your dealer or directly to Mobilex A/S.

Mobilex A/S Grønlandsvej 5 DK-8660 Skanderborg Denmark	Phone: +45 87 93 22 20 Telefax: +45 87 93 17 77 E-Mail: info@mobilex.dk
------------------------------------------------------------------------	-------------------------------------------------------------------------------

2. Safety requirements

- Make sure that this user manual is read by all persons using the device. Manufacturer doesn't take any responsibility for damages and/or injuries, caused by the fact that the user manual has not been followed.
- Only use the product in flawless condition.
- If defects or errors are detected you must immediately contact the dealer.
- Follow the information on the product label.
- Only use the product for the described purpose.
- Avoid making constructive changes on the device, unless you have the manufacturer's written acceptance for such modifications.
- During different adjustments which are possible on the device the user must take care not to squeeze the fingers or other body parts between two moving elements of the device
- The device must only be used on a stable surface.
- Ramps must not be loaded with more than the "max load" described in Technical Data and on the label.
- The ramps can be used at maximum inclination of:
 - T80060 12 cm**
 - T80075 15 cm**
 - T80125 25 cm**
- Only use the ramp for the purpose it is intended for.
- Avoid making constructive changes on the ramp, unless you have the producers written accept.
- The ramp must only be used on a stable surface.
- Be careful during folding or unfolding of the ramp. There is a risk of finger squeeze or cut between parts of the ramp.

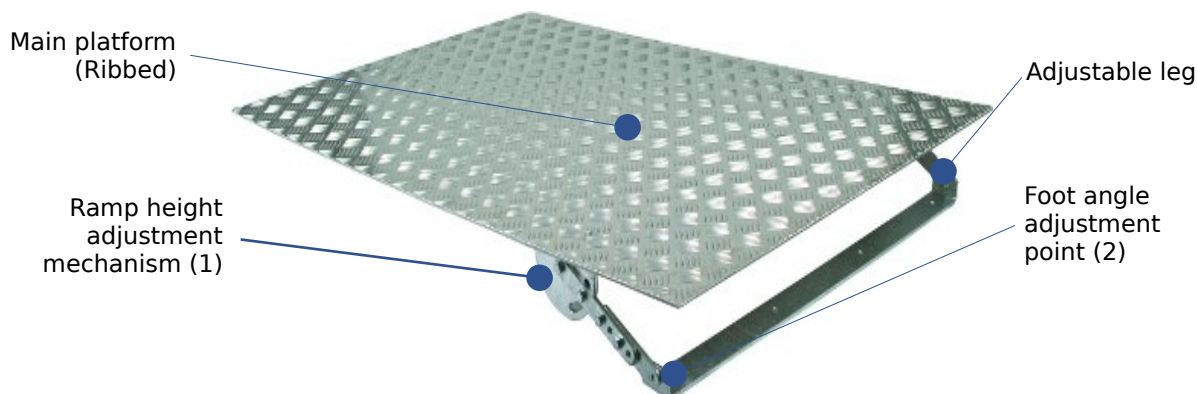
3. Description

Product description

Mobilex T-Ramps are made of aluminium. The surface is specially ribbed for anti-slipping purposes. The ramp consists of main platform and wide supporting leg with adjustable angle/height.

T-Ramps are supplied in cartons. The contents are:

- ✓ 1 Ramp
- ✓ 1 User manual

Set Up

T-Ramp comes fully assembled and ready for usage. The only set up which must be made is adjusting the ramp height and angle of ramp foot to the height of the obstacle you'd like to pass.

To make the adjustment loosen the bolts located on the height adjustment mechanism (1) and on the foot angle adjustment point (2). You must loosen bolts on both sides of the ramp. Adjust the leg so the ramp is on the desired height and tie the bolts (1). Then adjust the angle of the ramp foot, so it lies flat on the surface and tie the bolts (2).

Function description

T-Ramps are specially dedicated for help with wheelchair overpassing of elevated doorsteps, f.e. terrace & garden doors. The ramp leg must be adjusted to the height of doorstep which you'd like to pass. The ramp must be placed on a stable surface. Check that the ramp is stable before driving on it. It is possible to fix ramp to the surface (concrete, stone, tiles etc), screwing the ramp foot to the floor using provided holes located on the foot.

Transport recommendations

Alligator ramps can be folded for transport. Always assure that ramp during transportation is safely secured against unintended movements.

For the largest ramp size (T80125) it is recommended that 2 persons are needed to move ramp from one usage place to another. Stand at opposite sides of the ramp facing each other, grab the edges of the ramp with both hands at shoulder width and move ramp to the required location.

Accessories

Side protection is available for the T-ramps, which can be mounted in the side where applicable

4. Technical data

Mobilex T80 ramps	T80060	T80075	T80125
Total length	60 cm	75 cm	125 cm
Width	80 cm	80 cm	80 cm
Max. superable height	12 cm	15 cm	25 cm
Total weight	8,00 kg	12,00 kg	21,50 kg
Max. load	225 kg	225 kg	225 kg
Height adjustment	5-9 cm	8-13 cm	11-28 cm

Product label

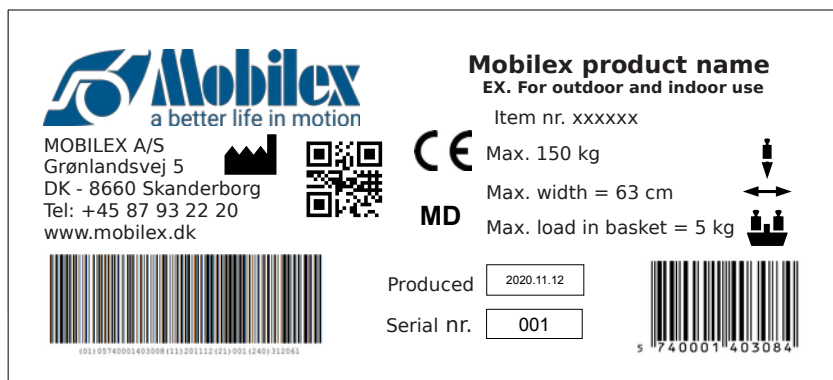
Image of the product label (not original)

5. Maintenance and cleaning

Normal dirt can be removed with standard cleaning agents and a sponge or soft rag. Check the specific product information and use only commercial cleaning products which are suitable for cleaning and disinfection (do not use alcohol-containing products and no solvents or abrasives).

Despite the solid construction and the use of resistant materials the product is subject to wear. It is therefore recommended to let check the product at regular intervals by a professional

- Lighter dirt and dust can be removed with a wet cloth.
- Heavier types of dirt can be removed with a non-corroding, mild washing agent.
- You can also use garden hose to clean the ramp.

Recommended service life: Normally 10 years depending on use.

Warning

 Please note that if the frame is bent or broken, the product must not be recycled but must be replaced

6. Reuse

The product is suitable for recycling. Before recycling, where a new user is to take over the product, the product must be checked according to point 5.

7. Most common problems and solutions

<i>Symptoms</i>	<i>possible cause</i>	<i>what to do?</i>
Ramp is not stable	The surface under the ramp is not flat	Check the position of the ramp. Put it on flat, smooth surface
Ramp is not stable	Ramp is putted on the surface upside down	Check if the A label is in the upper end and B label in the lower end of the ramp. Correct position if needed.
Ramp doesn't open	The lock strap is not loosen	Loosen the lock strap

8. Disposal of the product

The product can not be disposed of with household waste but must be brought to the local recycling centre.

9. Guarantee terms

Mobilex A/S offers 2 years of warranty for damages caused by errors in production or material errors. The warranty is not valid by not-intended repair or usage of the product. Parts subject to normal wear are not covered by the warranty, unless wear is caused by a manufacturing fault. If product changes are made without our written acceptance the CE-marking and product warranty will not be valid. In the event of product damages covered by this warranty, please notify your dealer or Mobilex A/S directly. The warranty does not cover transport costs and does not include compensation for personal injuries or non-use of the product during repair. The warranty does not cover damages due to the user manual not being followed.

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Mobilex A/S, Grønlandsvej 5, DK-8660 Skanderborg, Denmark,